


# Do's



- ✔ Ensure all components are fully dry before use/autoclaving. Esp. the powder chambers and handpieces.
- ✔ Store the machine in a cool, dry, and dust-free environment.
- ✔ Use only **EMS** consumables, accessories, and parts.
- ✔ Train all users in proper handling and maintenance procedures.
- ✔ Report and log any malfunctions or anomalies immediately to Unicorn Service helpline (7065558558).
- ✔ Make sure to only use Dry – compressed air with the machine.

# Airflow Prophylaxis Master Essential Do's, Don'ts & Care Guidelines

# Don'ts

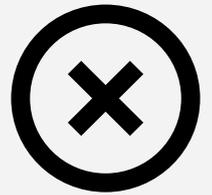

- ✘ Don't use the machine on patients with Woodpecker or so-called compatible powders.
- ✘ Don't operate the device with wet hands or on wet surfaces.
- ✘ Don't attempt unauthorized repairs or modifications.
- ✘ Don't expose the machine to extreme temperatures or humidity.



# Airflow Prophylaxis Master

## Guidelines for Machine Maintenance:

1. Flush the Airflow handpiece using easy clean tool from both ends – front 2 times, back 5 times. Use the 3-way syringe (air only) to gently flush out any excess water from the small opening on the backside.
2. Flush the Perioflow handpiece using easy clean tool from back side – 5 times.
3. Don't mix and match the powders – use the specifically dedicated jars for their respective powders only.
4. Only use Distilled water with the machine.
5. Use a mild surface disinfectant instead of spirit to clean the machine and handpieces, as this helps prevent corrosion and long-term yellowing of the machine surface.
6. Do the night cleaning protocol every night when you use the machine.
7. Perform the daytime cleaning protocol each morning following the night cleaning procedure, and after every patient throughout clinic hours.
8. Don't put the handpieces in glass bead sterilizer – Only use B-class autoclave.
9. Use only Dry-Compressed air with the device.
10. Use only **EMS Scaler Tips** on the scaler handpiece; otherwise, it will damage the threads of the handpiece.
11. Do not use erythritol, sodium bicarbonate powders, or so-called 'compatible' powders from other brands, as they will damage the handpiece, powder jars, and the machine itself.
12. Always remember, **Airflow Prophylaxis Master + Airflow Max Handpiece + Airflow Plus Power = AIRFLOWING.**
13. Please read the '*Instructions for Use*' and '*Treatment Recommendations*' booklets that come with the machine thoroughly before operating it.
14. A Preventive Maintenance (PM) request must be submitted by the practitioner before the end of both the first and second year after reaching 50 patient surveys to avail extended warranty on the machine. Our service team will perform the PM at no additional cost.



### Declaration

I, confirm that I have read and understood the document titled Essential Do's, Don'ts & Care Guidelines."

I agree to follow all instructions, precautions, and care protocols mentioned within. I take full responsibility for proper usage and maintenance of the equipment and will ensure my clinic staff is also informed accordingly.

**Customer Name:** .....

**Date:** ..... **Place:** .....

**Signature:** .....

Connect with



Unicorn DenMart