OBJUSER GUIDE Genie Always by your side

www.genieconnect.co.uk



GenieConnect® enables the delivery of more flexible and better quality care to combat loneliness and improve independence.

Contents

3	Т	urnina	Genie	On	and	Off
		3				

- 4 Changing the volume
- **5 6** Connecting to Wifi
- 7 8 Contacting the Care Centre
- **9 10** Contacting Family and Friends
- **11 12** Accessing the Radio
- 13 18 Adding a calendar event
- **19 20** Adding a recurring calendar event
 - **21 22** Deleting a calendar event
 - 23 Deleting a recurring calendar event
 - 24 Using voice commands



Turning Genie on and off



1. To turn Genie on press and hold the power button for a few seconds. Release the button when the screen lights up.



2. To turn Genie off press and hold the power button for a few seconds. A notification will appear on the Screen to confirm power off. Tap 'power off'.

Changing the volume



1.The volume controls are on the shoulders of Genie and are marked by a little sticker



2. When looking face on to Genie, the right hand side will increase the volume. The left hand side will decrease the volume.



3. You will know if the volume is changing when a bar appears across the top of the Genie screen with a circle moving to the left or right



Connecting to WiFi

When there is no WiFi connection a blue bar will appear at the bottom of the screen.



1.To connect to the WiFi tap 'CONNECT'.



 Select the WiFi you want to connect to and enter the password.



3. Once you have
entered the correct
WiFi password tap
'DONE'. Once
connected press
'Home'.

Connecting to the WiFi



4. If you need to connect to a different WiFi, from the home screen tap on the settings wheel.



5. You can then select WiFi settings and repeat steps 2-3.



Contacting the Care Centre



1. Tap the middle of the screen

1:13 pm Wednesday, 14 July 2021	16 °C	Bristol
Calling Image: Constraint of the second	Orge	Contemport

2. From the home screen tap 'Call Care Centre' and a video call will begin.

Contacting the Care Centre



3. Another way to call the care centre is by using the red button on the top of Genie.



4. Press the red button to call your care centre.



8

Contacting family and friends



1. Tap the middle of the screen



2. Tap 'Calling'

Contacting family and friends

		0		
1:11 pr Wednesday, 14	n July 2021 My co	ntacts		
Family	Carers	Friends	Call History	
Back Off	Call Care	e Centre	Home	
	8			

3. Select who you would like to call 'Family' 'Carers' or 'Friends'



4. Select the person you would like to call. This will start a video call. **10**



Accessing the radio



1.Tap the middle of the screen



2. Tap on 'Entertainment'



3. Tap 'Radio'

Accessing the radio



4. Tap 'BBC Sounds'



5.You can then swipe left or right to select a station





1.Tap the middle of the screen







3. Tap 'Calendar'



4. Tap on the plus symbol



5. To change the category of the event tap on the coloured square.



6. Choose the category of the event



← New Event	
Appointment	
Title	
Location	
Description	
() All-day	

7. To add a title tap on 'Title'



8. Type in the title of your event then tap 'NEXT'



9. If required add in a location and then tap 'NEXT'



10. If required add in further details and then tap 'DONE'



11. Scroll down by moving your finger upwards on the screen. Next you can change the time and date of the event.



12. To change the start time of the event tap on the time.



lew Event						
day		11		59		
y 14 (V		12	:	00		
y 14 (V		13		01		
our be	Cancel				ок	
d another remin	nder					

13. Change the time by scrolling up or down on the hour and minute sections. Once happy tap 'OK'



14. Repeat this step for the finishing time



15. Next you can add a reminder for this event.Start by tapping the time next to the alarm bell icon



16. Select the reminder you require



17. Once you areready to add thisevent to the calendar,tap the tick in the topright hand side.



Adding a recurring calendar event



18. To add areoccurring event tothe calendar simplyfollow steps 1-17.Once you have addedreminder, tap on therepetition section.

19. Select the duration of repetition

ewevent	
/ 14 O No repetition	- 1
/ 14 O Daily	- 1
O Weekly	- 1
Monthly	- 1
Yearly	- 1
repetition	



20. Tap on the date to select which date you would like to end the repeating event

Adding a recurring calendar event



21. And then tap here



You can now select the date you would like the reoccurring event to end on. Once selected, tap ok and then finish by tapping the tick in the top right hand side.



Deleting a calendar event



1. To delete an event from the calendar start by selecting the day that you would like to delete an event from



21

2. Then select the event

Deleting a calendar event



3. To delete the event, tap on the bin icon in the top right corner



Tap 'YES' to continue with the deletion.
 The event will now be deleted from the calender.

22



Deleting a reoccurring calendar event



5. To delete a reoccurring event follow the previous steps 1-4

6. When you tap on the bin icon you will be asked which event you would like to delete.



7. Once you have decided which option you would like, tap 'YES'. The selected events will then be deleted from the calendar
 23

Using voice commands

COMMANDS

Start all commands with "Genie..."

"WHAT IS THE TIME?" Genie will tell you the time

"WHAT IS THE WEATHER TODAY?" Genie will tell you the weather

"OPEN ORGANISER" Opens organiser to view calendar, alarms and tasks

"OPEN CALENDAR" Opens calendar to view and set appointments and reminders

"OPEN ALARMS" Opens Alarm to sets alarms, timer and stopwatch

> **"OPENS TASKS"** Opens tasks to make a 'to-do' list

"OPEN CONTACTS" Opens contacts to make a call

"OPEN BBC SOUNDS" Open BBC sounds to play your favourite radio station

Listening to a voice command can also be activated by pressing the middle part of Genie's body









www.genieconnect.co.uk