



---THE BUNK POLICE---

MORRIS

TEST-KIT-GUIDE

INSTRUCTIONS

1. POWDERIZE YOUR SAMPLE AND PLACE IT ON A WHITE CERAMIC PLATE ***DO NOT USE A TEST TUBE***
2. DRIP ONE DROP FROM BOTTLE "A" ON THE SAMPLE
3. DRIP ONE DROP FROM BOTTLE "B" ON THE SAMPLE
4. STIR THE MIXTURE WITH A TOOTHPICK OR THE SHARP POINT OF A KNIFE FOR A FULL 30 SECONDS
5. COMPARE COLOR WITH REACTIONS IN THIS BOOK

SAMPLE SIZE

POWDER: USE 5 MILLIGRAMS - ABOUT THE SIZE OF A DROP OF WATER

PILL: SCRAPE OFF A SMALL AMOUNT - AROUND 1/20TH OF A PILL

WARNING! THE TESTING LIQUID IS HAZARDOUS
DO NOT GET IT ON YOUR SKIN OR IN YOUR EYES - DO NOT INGEST
USE IN A WELL VENTILATED AREA AND WEAR GLOVES & EYE PROTECTION
IF POSSIBLE. IF THE BOTTLE BECOMES BLOCKED, DO NOT ATTEMPT TO
CLEAR BY SQUEEZING. INSTEAD CLEAR THE BLOCKAGE BY SHAKING
VIGOROUSLY OR INSERTING A PAPERCLIP / SIMILAR OBJECT.

KEEP OUT OF HEAT/SUNLIGHT


DO NOT REMOVE ABSORBENT POWDER FROM CONTAINER

INSTRUCTIONS


THIS TEST IS DIFFERENT FROM EVERY OTHER KIT PRODUCED BY THE BUNK POLICE - TWO STEPS ARE CRUCIAL TO ACHIEVE THE DESIRED RESULT - 1. CRUSH / POWDERIZE YOUR SAMPLE COMPLETELY BEFORE TESTING 2. STIR THE SAMPLE FOR A FULL 30 SECONDS BEFORE READING THE RESULT.

WARNING


WHEN YOU FIRST ADD THE DROP FROM BOTTLE "A" - BLUE SPECS WILL APPEAR. IGNORE THIS COLOR AND WAIT UNTIL YOU 1. HAVE ADDED THE DROP FROM BOTTLE "B" 2. HAVE FINISHED STIRRING FOR A FULL 30 SECONDS 3. THE COLOR HAS STOPPED CHANGING - BEFORE READING THE REACTION.

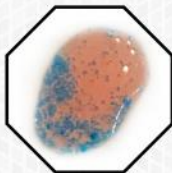


COLOR AFTER ADDING DROP "A" - IGNORE

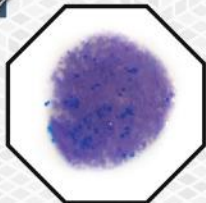


KETAMINE

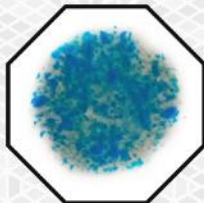

COCAINE

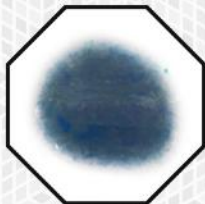

DCK & ZF-DCK

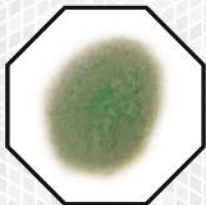

NO REACTION

THE FOLLOWING SUBSTANCES DO NOT REACT
WITH THE MORRIS TEST KIT - MEANING THE COLOR
THAT IS PRESENTED WILL BE GREEN AS SEEN
ABOVE - THIS LIST IS NOT COMPREHENSIVE.
RESEARCH IS STILL ONGOING.

PCP • 2-OXO-PCE • 3-MeO-PCP
3-Cl-PCP • 3-MeO-PCE • O-PCE
MXPr • MXE • EPE • DPD

USE THESE KITS TO TEST
AMPHETAMINE

MARQUIS

COLOR
MATCHES
SPECTRUM

MECKE

ANY
OTHER
COLOR

NOT
AMPHETAMINE

COLOR
MATCHES
SPECTRUM

SIMON'S

ANY
OTHER
COLOR


NOT
AMPHETAMINE

ZERO
REACTION

AMPHETAMINE

BRIGHT
BLUE

METHAMPHETAMINE

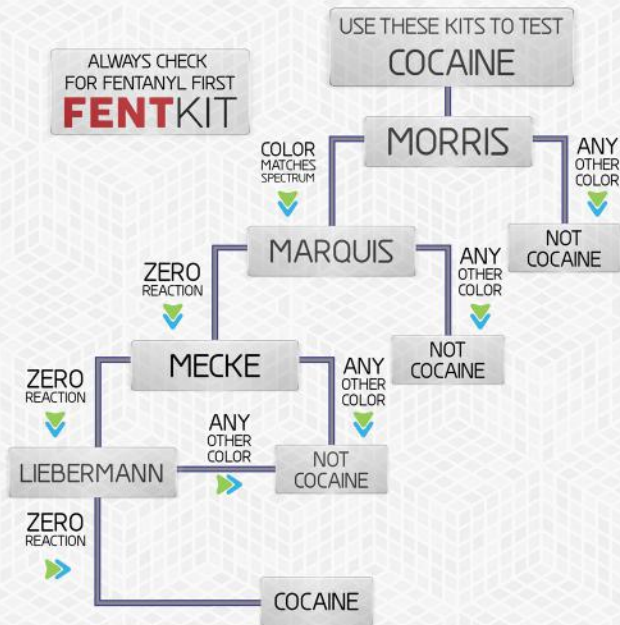

USE THESE KITS TO TEST
DMT

HOFMANN

COLOR
MATCHES
SPECTRUM



ANY
OTHER
COLOR



MECKE

NOT
DMT

COLOR
MATCHES
SPECTRUM




ANY
OTHER
COLOR




DMT

NOT
DMT

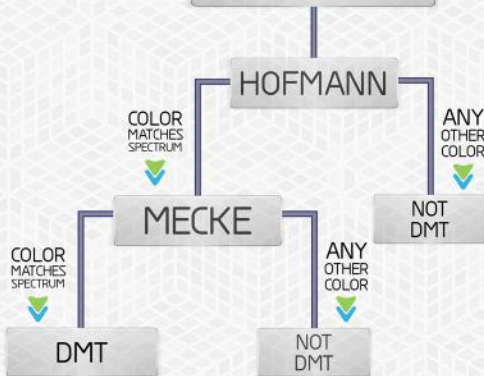

ALWAYS CHECK
FOR FENTANYL FIRST
FENTKIT

USE THESE KITS TO TEST
HEROIN & PAIN PILLS

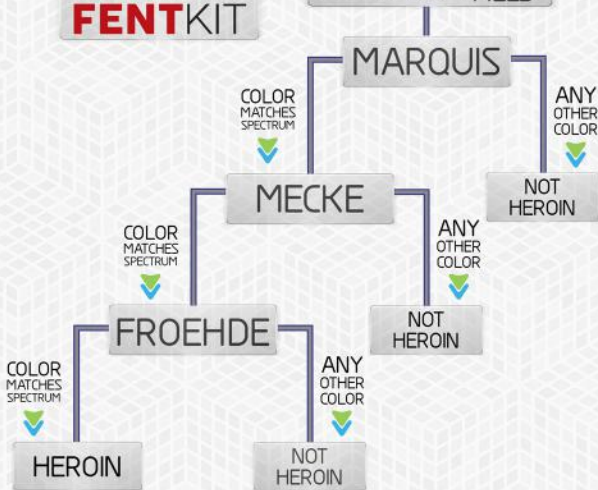

USE THESE KITS TO TEST
KETAMINE

MORRIS

COLOR
MATCHES
SPECTRUM



ANY
OTHER
COLOR



NOT
KETAMINE

MECKE

ZERO
REACTION



ANY
OTHER
COLOR



NOT
KETAMINE

LIEBERMANN

COLOR
MATCHES
SPECTRUM



ANY
OTHER
COLOR



ANY
OTHER
COLOR




NOT
KETAMINE


FROEHDE


ZERO
REACTION



KETAMINE

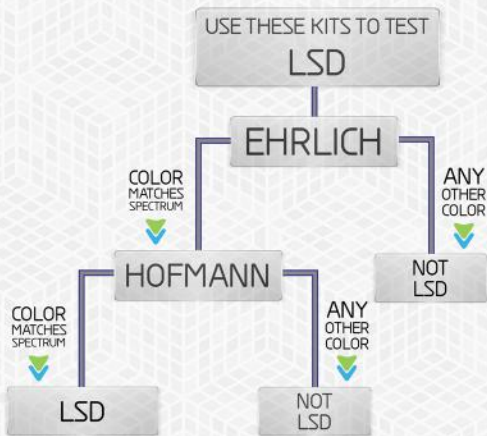

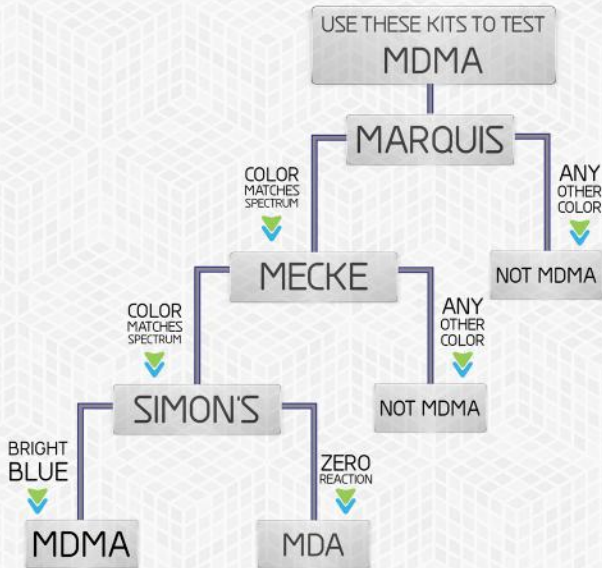

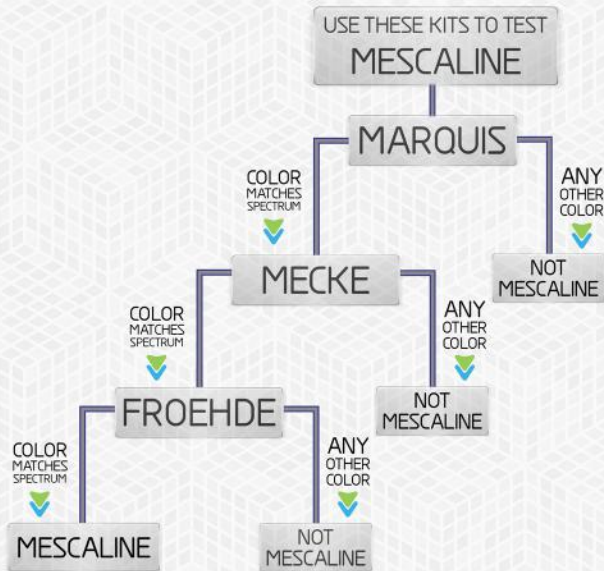

USE THESE KITS TO TEST
METHAMPHETAMINE

MARQUIS

COLOR
MATCHES
SPECTRUM

MECKE

ANY
OTHER
COLOR

NOT
METHAMPHETAMINE

COLOR
MATCHES
SPECTRUM

SIMON'S

ANY
OTHER
COLOR

NOT
METHAMPHETAMINE

BRIGHT
BLUE

METHAMPHETAMINE

ZERO
REACTION

AMPHETAMINE



SAFETY & STORAGE

WARNING! THE TESTING LIQUID IS A STRONG ACID

AVOID SKIN AND EYE CONTACT - DO NOT INGEST

CONDUCT TESTS IN A WELL VENTILATED AREA

USE GLOVES IF POSSIBLE & WASH HANDS AFTER USE

IF THE BOTTLE BECOMES BLOCKED, DO NOT ATTEMPT TO CLEAR IT BY SQUEEZING. INSTEAD CLEAR THE BLOCKAGE BY SHAKING THE BOTTLE VIGOROUSLY OR INSERTING A PAPERCLIP OR SIMILAR OBJECT IF NECESSARY.

KEEP REFRIGERATED - FROZEN IF POSSIBLE
(IF USED OUTDOORS KEEP SHADED OR IN A COOLER)

SHELF LIFE DEPENDS ON STORAGE CONDITIONS
IF KEPT REFRIGERATED YOUR KIT CAN LAST SIX MONTHS TO A YEAR
(WE DO NOT RECOMMEND USING KITS OVER ONE YEAR OLD)

TIPS & TRICKS

1. USE MORE THAN ONE TYPE OF KIT FOR INCREASED ACCURACY
2. MAKE SURE TO CONDUCT TESTS IN A WELL LIT AREA
3. CRUSH YOUR SAMPLE INTO A POWDER BEFORE TESTING
4. THE FIRST 1 - 3 SECONDS OF THE REACTION ARE THE MOST IMPORTANT - PAY CLOSE ATTENTION DURING THIS TIME OR RECORD THE REACTION ON YOUR PHONE SO YOU CAN REPLAY AND ANALYZE IT AS MANY TIMES AS YOU LIKE.
5. THERE ARE MANY SUBSTANCES THAT DO NOT CHANGE COLOR OR REACT IN ANY WAY. **SOME OF OUR KITS TAKE ADVANTAGE OF THIS, ALLOWING YOU TO SEE THE REACTIONS OF UNWANTED SUBSTANCES (ADULTERANTS) THAT MIGHT BE IN YOUR SAMPLE ALONG WITH THE DESIRED SUBSTANCE.**
6. OUR REACTION CHARTS ARE MADE USING TIME-LAPSE PHOTOS OF VERIFIED PURE SAMPLES AS THEY REACT. BECAUSE YOU'RE LOOKING AT REAL PHOTOS OF A REACTION, YOU CAN USE OTHER INFORMATION BEYOND JUST COLOR AND TIMING TO IDENTIFY YOUR SUBSTANCE. DID THE POWDER DISSOLVE QUICKLY OR SLOWLY? WERE THERE LARGE BUBBLES OR SMALL RAPID ONES? ETC..

THE VIDEOS USED TO MAKE OUR CHARTS ARE AVAILABLE ON OUR VIMEO / ANDROID APP / AND BUNKPOLICE.COM



...QUESTION EVERYTHING...

@BUNKPOLICE / BUNKPOLICE.COM