



Patient-friendly Compression



Catalogue | Issue 8



HADDENHAM®

< September >

| | | | | | | |
|----|----|----|----|----|----|----|
| 30 | 31 | 1 | 2 | 3 | 4 | 5 |
| 6 | 7 | 8 | 9 | 10 | 11 | 12 |
| 13 | 14 | 15 | 16 | 17 | 18 | 19 |
| 20 | 21 | 22 | 23 | 24 | 25 | 26 |
| 27 | 28 | 29 | 30 | 31 | 1 | 2 |



Clinical Support Service


Visit...



<https://hadhealth.com.au/support>

Book a time that suits you to discuss your cases with our clinical advisors.
They have a wealth of knowledge to support possible compression therapy options.

- Assistance with measuring and sizing for compression garments and wraps
- Discuss the rationale around the choice of garment and treatment
- Develop effective strategies through guided clinical reasoning

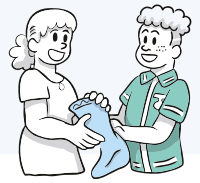

Head-to-toe compression

Which area of the body do you need to treat?

Chest

Page

| | |
|-----------------------|-------|
| Comfiwave Breast Band | 116 ▶ |
| Eto Bras & Vests | 128 ▶ |
| Prairie Bras | 138 ▶ |
| Kinesio Tape | 156 ▶ |

Abdomen

| | |
|--------------------------|-------|
| Abdominal Wrap | 157 ▶ |
| Comfiwave Abdominal Band | 104 ▶ |

Genitals

| | |
|----------------------|-------|
| Comfiwave Shorts | 102 ▶ |
| Eto Shorts | 134 ▶ |
| Eto Scrotal Supports | 133 ▶ |
| Veni Shorts | 26 ▶ |

Legs

| | |
|----------------------|-------|
| Circular Knit Socks | 24 ▶ |
| Flat Knit Socks | 18 ▶ |
| Comfiwave Below Knee | 96 ▶ |
| easywrap Leg | 74 ▶ |
| Steve Donning Aids | 150 ▶ |

Head & Neck

Page

| | |
|----------------|-------|
| Comfiwave Neck | 118 ▶ |
| Eto Garments | 131 ▶ |
| Kinesio Tape | 156 ▶ |

Arms

| | |
|-------------------|-------|
| Comfiwave Sleeves | 106 ▶ |
| Armsleeves | 48 ▶ |
| easywrap Arm | 86 ▶ |

Hands


| | |
|------------------|-------|
| Microfine Glove | 64 ▶ |
| Flat Knit Gloves | 52 ▶ |
| Comfiwave Glove | 110 ▶ |
| easywrap Hand | 84 ▶ |

Thigh

| | |
|-------------------------|-------|
| Circular Knit Stockings | 24 ▶ |
| Flat Knit Stockings | 18 ▶ |
| Comfiwave Thigh | 98 ▶ |
| easywrap Thigh + Knee | 78 ▶ |
| Eto Grace | 136 ▶ |

Feet

| | |
|---------------------|-------|
| Microfine Toecap | 34 ▶ |
| Custom Made Toecaps | 35 ▶ |
| Comfiwave Toecap | 100 ▶ |
| easywrap Foot | 76 ▶ |

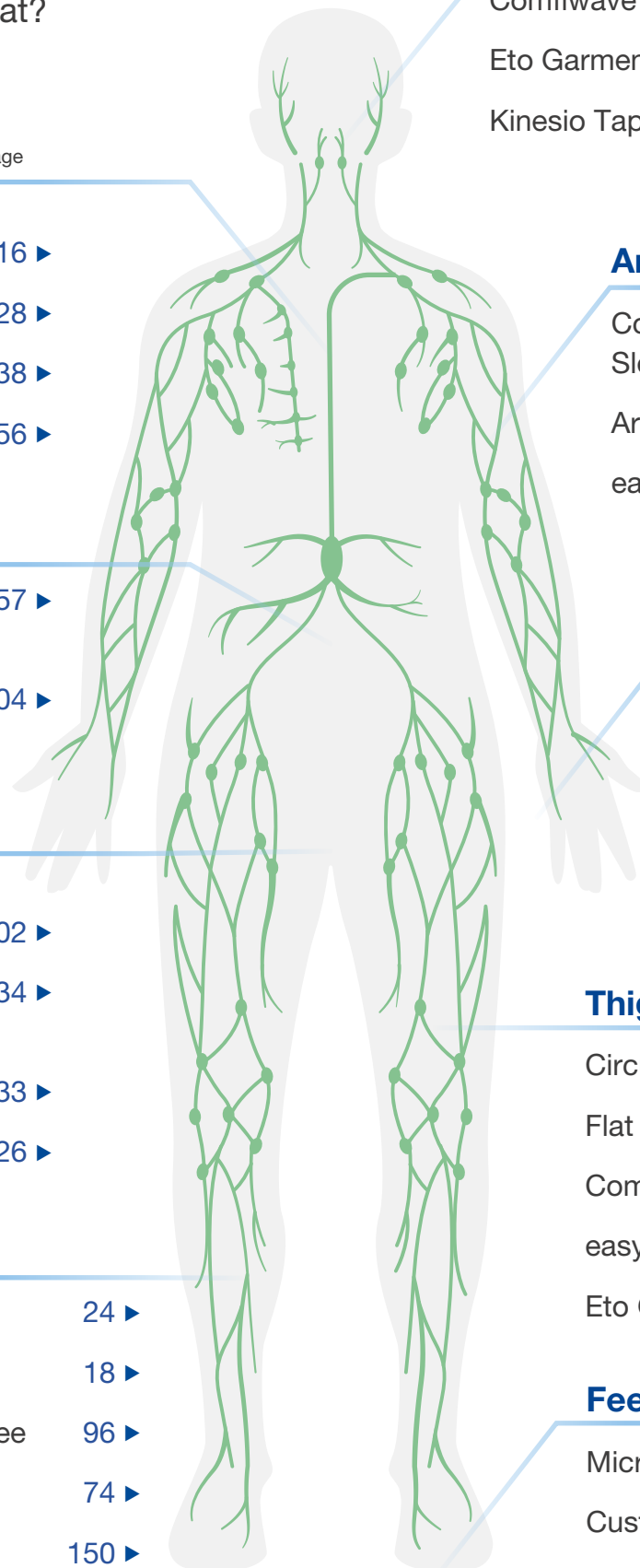

Patient-friendly Compression

Haddenham Healthcare is a family-owned, UK-based medical device company with a global outlook. We specialize in compression therapy for lymphoedema and related conditions, designing products that integrate seamlessly into patients' daily lives.



**Supporting
Therapists**

Page 4 ▶



**Supporting
Patients**

Page 5 ▶



Our Ethos

Page 6 ▶



**Conditions
We Treat**

Page 8 ▶

About us

How we work

We listen to patients and healthcare professionals, using real-life feedback for research and development.

We innovate with purpose, creating lasting solutions. We partner long-term, ensuring everyone is supported.



Who we are

We foster a friendly culture where ideas can be shared freely.

We care about our communities, donating generously from our retained earnings to charities annually.

We grow responsibly, committed to our employees, partners, and the environment.



Why it matters

Our ethos drives us to deliver life-changing products. We measure success by whether we improve people's lives.

“ We strive to be innovative, trustworthy and sincere.”

Supporting therapists...

Sales

E-mail

Send your order/query to sales@hadhealth.com.au

Call us

We'd be happy to take your call 9am - 5pm, Mon-Fri.
[03 9544 5515](tel:0395445515).

Portal

Order via our user friendly e-commerce portal. Contact us for more information.
portal.hadhealth.com.au

Support

Clinical support service

Book an appointment with our clinical team. They have a wealth of knowledge to support compression therapy solutions.

www.hadhealth.com.au/contact

Learning

Online modules, webinars and face to face training on our products and more.
www.hadhealth.com.au/learning

Resources

Order forms

Download measurement forms and charts.

www.hadhealth.com.au/forms

Social

Connect to our social media channels and mailing list to stay up to date with the latest news and products.



hadhealth.com/subscribe



facebook.com/hadhealth



instagram.com/hadhealth



youtube.com/@haddenham

...and their patients



www.Lymphshop.com

Compression


Online store for patient ordering of **Extra Garments** - encouraging supervised self-management. For example spare pairs of stockings for taking on holiday, or extra colours to suit their wardrobe.

Accessories

Garment glue, kinesiology tape and more.

Resources

Patient-friendly books on Lymphoedema, "Compression Matters" blog and mailing list with offers and promotions.

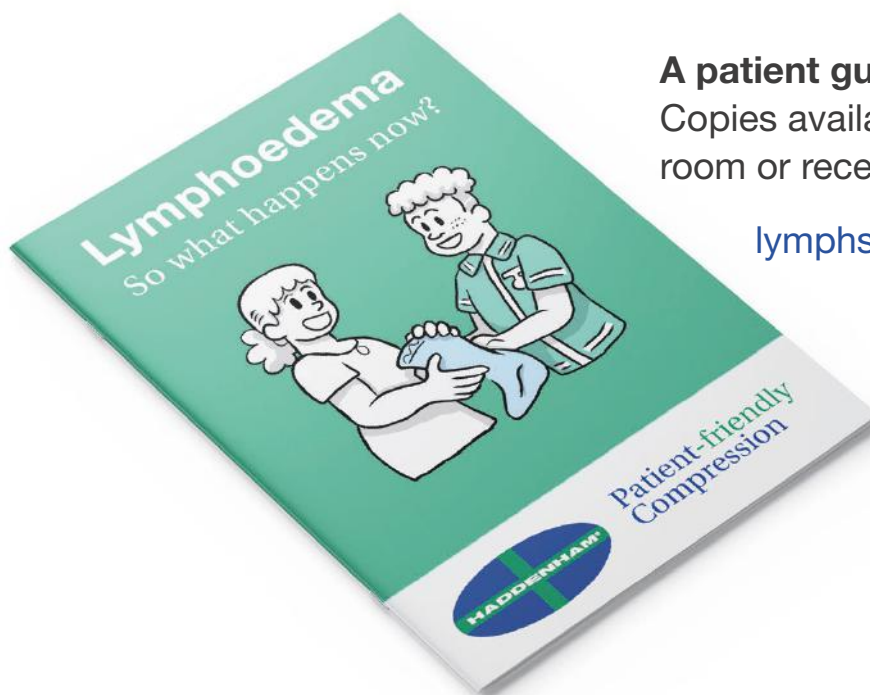

A patient guide for the recently diagnosed
Copies available to request for your waiting room or reception.

lymphshop.com/what-is-lymphoedema

Our Ethos

Approachability & sincerity

We put people at the heart of everything we do - from having a real person answer the phone, to offering free training sessions led by experienced clinicians.

Our open-door policy ensures every employee or customer feels acknowledged and valued.



Innovation for real-world needs

We develop products that are thoughtfully engineered to address patient and clinical needs. Feedback from healthcare professionals and patients keep us responsive and forward-thinking.


Long-term relationships

We regard our customers and suppliers as partners, collaborating to refine solutions that enhance patient care and outcomes.

Trust is built on reliability, honesty, and genuine collaboration—the values that define us as a family business.

A place people want to work

We foster a friendly, encouraging culture, where continuous improvement is valued. Employees benefit from work-life balance and opportunities for growth, making Haddenham a pleasant place to work.

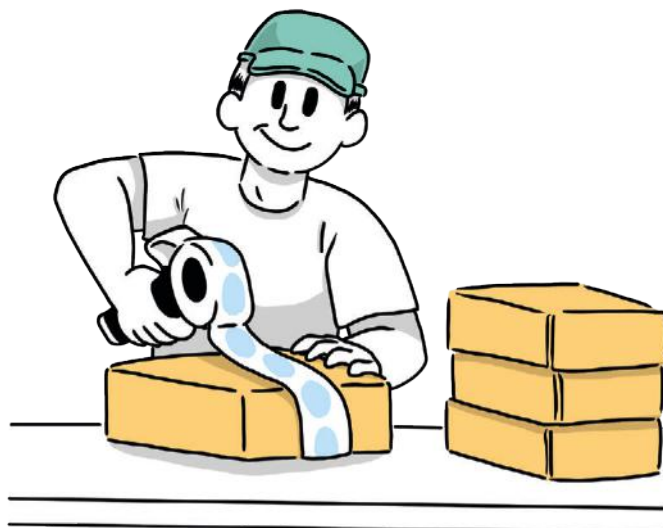

Social & environmental responsibility

We contribute a percentage of our retained earnings to charitable causes, both locally and internationally to help build a better future for all.

We adhere to recognised best practices in environmental management, ensuring we minimise our impact throughout our operations.

Why it matters

Our goal is to be a trusted innovator in compression therapy, delivering products that patients prefer to wear and which healthcare professionals can depend on.

“ By combining innovation, trust and sincerity, we aim to leave a lasting, positive mark on the world benefiting patients, communities, and our shared future.”

Conditions we treat

Lymphoedema / Chronic Oedema

What is the lymphatic system

The lymphatic system is a network of vessels and glands throughout the body that helps fight infection and remove excess fluid. It is responsible for collecting all fluid in the tissues and returning it to the circulatory system, failure of the lymphatics leads to chronic oedema.

What can go wrong?


Chronic oedema is an umbrella term used to describe swelling of all causes and occurs when fluid accumulates in the tissues and is present for at least 3 months. This can be caused by an underlying problem with the venous or lymphatic systems preventing fluid balance in the tissues.

The terms chronic oedema and lymphoedema are sometimes used interchangeably, both describe failure of the lymphatics but there are some key differences.




Lymphoedema is a long-term (chronic) condition that causes swelling in the body's tissues. It can affect any part of the body, but usually develops in the arms or legs.

Unhealthy lymphatics

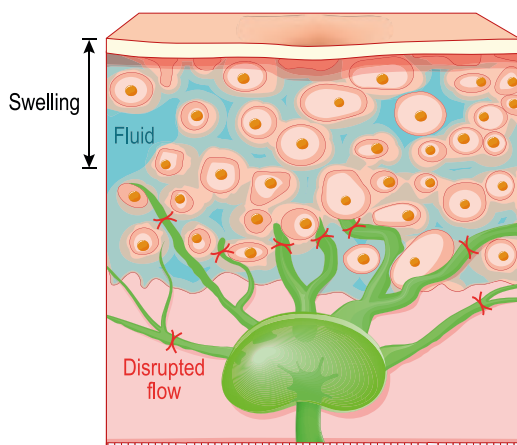

Healthy/compressed lymphatics

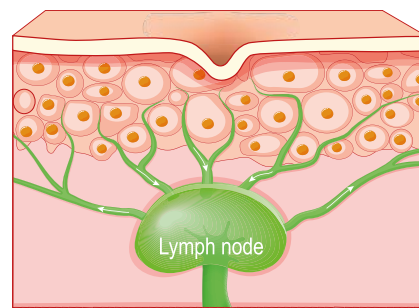

How can compression help?

When the lymphatic system is compromised, pressure within the system causes the vessels to dilate and results in backflow of lymph into the tissues.

The external pressure created when compression is applied, moves fluid from the tissues into the initial lymphatics, increasing lymph transport. Where there is lymphatic failure compression will help to push fluid through the tissues towards areas where drainage is functional.

Vascular Disorders


What is the Venous/Circulatory System?

It comprises of the heart, arteries supplying oxygenated blood to our tissues and organs, the veins return deoxygenated blood back to the lungs and heart. Capillaries are the intersection between arteries and veins where high pressure arterial blood transfers to the venous system causing fluid to leak out of the blood vessels into the interstitium containing nutrients, other substances and debris.

What can go wrong?

When the veins become damaged the valves within are unable to close and return the blood back to the lungs & heart. This results in venous reflux, and venous hypertension.

Prolonged venous hypertension leads to leakage of proteins and fluid into the skin, creating inflammation and subsequent tissue breakdown, a venous leg ulcer.

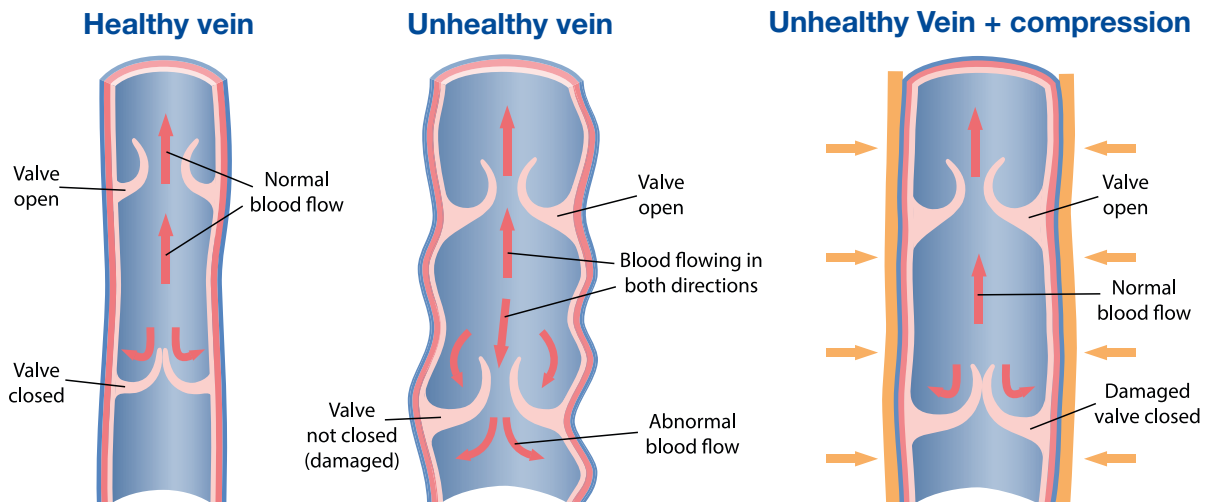

How can compression help?

Compression works by applying external pressure on the skin, thus compressing the veins. This force helps to improve the function of the valves within the veins, which reduces the amount of fluid that leaks out of the veins and into the tissues.

Compression also supports the muscles, which helps to create a force to push blood through the venous system.

CEAP Classification of venous disorders

Clinical Manifestations, Etiology, Anatomic Distribution, Pathophysiology

| | |
|------------|--|
| C0 | No visible or palpable signs of venous disease |
| C1 | Telangiectasias or reticular veins |
| C2 | Varicose Veins |
| C2r | Recurrent varicose veins |
| C3 | Oedema |
| C4a | Pigmentation or eczema |
| C4b | Lipodermatosclerosis or atrophie blanche |
| C4c | Corona phlebectatica |
| C5 | Healed venous ulcer |
| C6 | Active venous ulcer |
| C6r | Recurrent active venous ulcer |

Lipoedema

What is Lipoedema

Lipoedema is a condition that causes a symmetrical build-up of adipose (fat) often in the lower limbs but can also affect the arms, usually affecting women. It typically spares the hands and feet and is often mistaken for lymphoedema or obesity. The affected areas may feel tender, bruise easily, and appear disproportional. While some people experience pain or discomfort, others may have few symptoms.

How can compression help?

Compression therapy is a key part of conservative management for lipoedema. While it does not reduce the underlying fat tissue, compression can help alleviate symptoms such as pain, heaviness, and tenderness. It supports the tissues, promotes comfort, and can improve mobility. When secondary lymphatic involvement occurs, compression also assists in managing oedema by helping to reduce fluid accumulation and support lymphatic return.

General guidance

You should always aim to treat your patient in line with best practice guidelines. The recommendations in this document are based on best practice guidelines from national and international expert groups, and are within published documents.

Remember that this guide does not replace your clinical judgement. If you are unsure then do seek advice from your peers, or directly from the clinical team at Haddenham Healthcare (clinical@hadhealth.com.au).


There may be instances where you are unable to confer. In such cases the compression prescribed should be one that is tolerated well by the patient and applies a suitable level of compression for the symptoms being treated.

Hosiery



Medical Compression Hosiery

Haddenham medical compression hosiery utilises high-quality, long-lasting yarns that deliver RAL standard, graduated compression to the lower limb.

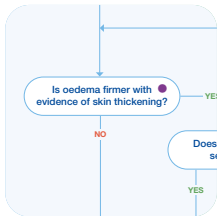

Which fabric do I need?

Use our pathway to guide your clinical judgement [Page 16](#) ▶



What makes it so patient-friendly?

Turn the page to discover all features [Page 12](#) ▶

Sized Flat Knits



Pertex Light
[Page 18](#) ▶

Sized Circular Knits



Veni
[Page 24](#) ▶



Stride
[Page 32](#) ▶

Custom Made



Hosiery
[Page 36](#) ▶



Pertex
[Page 20](#) ▶



Star Cotton
[Page 28](#) ▶



Doktus
[Page 30](#) ▶



Toecaps
[Page 35](#) ▶



Goldpunkt
[Page 22](#) ▶



Microfine Toecap
[Page 34](#) ▶

Hosiery Appendix



Sizing information, measuring guide, and full range of options & add-ons. [Page 39](#) ▶



Freedom
to choose

Graduated compression

Works with the body's natural circulation - concentrating the highest compression at the ankle and gradually decreasing up the limb.

The perfect fit

The widest range of sized garments available, with only a few simple measurements required.

- 3 leg lengths
- 3 foot lengths
- Up to 12 sizes
- Extra wide option for every size



Sized

Our sized Hosiery is **made-to-order**, with the care and quality that we believe each patient deserves.

We also retain **stock** of frequently ordered variants for an even faster turnaround.

or

Custom Made

We aim to be flexible wherever possible by providing a made-to-measure service that offers unparalleled customisation and fit.

7 versatile fabrics

Flat and Circular knit garments suitable for treatment of all stages of Lymphoedema, Venous disorders and Lipoedema.

Fabric selection guide

See page 16 ▶




Quality

We only use high-quality, durable, long-lasting yarns which are guaranteed for 6 months of use.

Compression levels are standardised using the RAL classification system.

Reliable

Stockings that can keep up with the most active of lifestyles, thanks to conforming fabrics and a variety of grip top options.

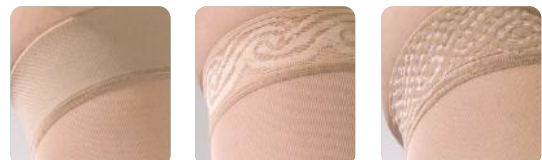

Natural colours

A wide variety of colours to choose, from a versatile palette of natural tones.



Subtle patterns

Our most versatile fabric Veni also offers stylish pattern options for added texture and additional aesthetic combinations.



Veni



Doktus

Configurable

Comprehensive options to tailor the garment to the patient's needs. Style like active-wear with **Footless** finish, add a **Fly** to tights, or enhance donnability with **Velcro closures**.

For full range of options see [Hosiery appendix Page 39](#) ▶

Which fabric?

Please use the below maintenance phase pathway as a guide to which of our fabrics would be most suitable for your patient based on their presentation and symptoms.

How would you describe the level of your patient's oedema?

Early accumulation of fluid which subsides with elevation, with or without pitting.

ISL Stage 0-1

Mild

Tissue swelling rarely reduces with elevation and pitting is present.

ISL Stage 2

Moderate

Pitting is absent with skin thickening fibrosis and excess subcutaneous fat.

ISL Late Stage 2 - Stage 3

Severe

What is the level of shape distortion? 

MILD


EXTENSIVE

MODERATE

Is the oedema soft and watery?

YES

NO

How would you describe the texture of the patient's tissue? 

WOODY

PUTTY LIKE

What is the condition of the patient's skin? 

What is the level of pitting? 

Are warty overgrowths with fat lobules present?

NO

YES

HEALTHY SKIN

SENSITIVE

FRAGILE SLIGHT*

MODERATE

DEEP

veni

STAR
Cotton

Peritex
Light

STRIDE
Below Knee

Doktus

Peritex

GOLDPUNKT

Page 24 ▶

Page 28 ▶

Page 18 ▶

Page 32 ▶

Page 30 ▶

Page 20 ▶

Page 22 ▶

Signs and symptoms

▲ Skin conditions

Fragile Thin skin prone to skin tears

Sensitive the presence of skin conditions where the skin is prone to reactions such as redness and itching.

◆ Textures

Watery soft pliable tissues.

Putty putty consistency represents early fibrotic changes, will pit with firm pressure.

Woody firm woody texture is consistent with advanced fibrotic tissue changes and does not pit.

● Pitting Levels

Slight 1+ 2mm barely detectable immediate rebound

Moderate 2+ 4mm deep <10 second rebound

Deep 3+ 6 mm deep <30 second rebound

Very deep 4+ up to 8mm deep >30 second rebound

● Shape distortion



Mild

Moderate

Extensive

How much compression?

Graduated compression* is highest at the ankle and gradually decreases as the yarns travel up the limb.

When deciding how much compression to apply also consider these 3 factors:

1. The nature of the fabric yarns and the way in which they are knitted together.
2. The fabric's ability to retain its structure under pressure. Measured as the increase in compression, per centimetre increase in the circumference of the limb (mmHg/cm).
3. The resultant force exerted on the limb measured in mmHg.

* Read more about how pressure is distributed on [Page 45](#) ►



Stiffness mmHg/cm

✓ Low = 0-2 | ✓ Medium = 2-4 | ✓ High = 4-6

| Pressure | Circular Knit | | | | Flat Knit | | |
|-----------------------------------|---------------|-------------|----------|----------|--------------|----------|-----------|
| | veni | STAR Cotton | Dokus | STRIDE | Pertex Light | Pertex | GOLDPUNKT |
| RAL Class 1 18 - 21mmHg | ✓ 1.1 | ✓ 1.4 | | ✓ 1.2 | ✓ 1.5 | | |
| RAL Class 2 23-32mmHg | ✓ 1.3 | ✓ 1.5 | ✓ 1.6 | ✓ 1.7 | | ✓ 2.1 | ✓ 2.8 |
| RAL Class 3 34-46mmHg | ✓ 1.8 | | ✓ 1.8 | | | ✓ 4.3 | ✓ 3.3 |
| RAL Class 4 49+ mmHg | | | | | | | ✓ 4.7 |

[Page 24](#) ►

[Page 28](#) ►

[Page 30](#) ►

[Page 32](#) ►

[Next page](#) ►

[Page 20](#) ►

[Page 22](#) ►

Pertex Light

Below Knee, Thigh High & Tights

Material Composition: Polyamide, Elastane

A unique stretchy flat knit that stretches with ease and hugs gently – ideal for fragile or sensitive skin.

Haddenham medical compression hosiery utilises high-quality, long-lasting yarns that deliver RAL standard graduated compression to the lower limb.

Effortless

Easy application

Especially for elderly or carers.

Conforming

Great at bridging limb shape discrepancies

Skin kind

Gentle on fragile or sensitive skin



Compression

| Class | mmHg | Stiffness | Well suited for | | |
|----------------|-------|-----------|--|--|---|
| Class 1 | 18-21 | Low | LYMPHOEDEMA Soft watery, slightly pitting oedema with minimal shape distortion. Dependency oedema. | VENOUS DISEASE No visible or palpable signs of venous disease | LIPOEDEMA Smooth skin with an increase of enlarged subcutaneous fat tissue |

Codes

| |  |  |  |  | | |  | | | | |
|----------------|---|---|---|--|------|-----------|---|--------|-----------|--------|--------|
| | Anklet* | Below Knee | Thigh High | Thigh High + Waistband | | | Tights | | | | |
| | | + GRIP TOP | + GRIP TOP | LEFT LEG | BOTH | RIGHT LEG | LEFT LEG | BOTH | RIGHT LEG | | |
| Class 1 | 090 | 901 | 900 | 902 | 903 | 905 | 904 | 906 | 908 | 907 | 909 |
| Unit of Sale | Pair | Pair | Pair | Pair | Pair | Single | Single | Single | Single | Single | Single |

Stock Options

All other variants made-to-order in 12-14 days




| Class | Grip Top | Sizes | Widths | Leg | Foot | Toes | Colour |
|-------|-----------|-------|----------|---------|---------|----------------|--------|
| 1 | 3cm Plain | 7-15 | Standard | Regular | Regular | Closed Open | Beige |

Sizing

Extra wide sizes are shown in green

| Circumferences cm | 4 | 5 | 6 | 7 | 8 | 9 | 10 | 11 | 12 | 13 | 14 | 15 |
|----------------------|----------------|----------------|----------------|----------------|----------------|----------------|----------------|----------------|----------------|----------------|-----------------|-----------------|
| t Waist | 65-75 | 70-80 | 70-80 | 75-85 | 75-85 | 80-90 | 80-90 | 85-95 | 85-95 | 90-105 | 90-105 | 95-110 |
| j Buttocks | 85-105 | 90-110 | 90-110 | 95-115 | 95-115 | 100-120 | 100-120 | 105-125 | 105-125 | 110-130 | 110-130 | 115-135 |
| g Thigh Top | 46-65 44-53 | 48-68 46-56 | 50-71 48-59 | 52-74 50-62 | 54-77 52-65 | 56-81 54-69 | 58-85 56-73 | 60-89 58-77 | 62-93 60-81 | 64-98 62-86 | 66-103 64-91 | 68-108 66-96 |
| d Below Knee | 29-40 28-32 | 31-42 30-34 | 32-44 31-36 | 34-46 33-38 | 35-48 34-40 | 37-50 36-42 | 38-52 37-44 | 40-54 39-46 | 41-56 40-48 | 43-59 41-51 | 44-62 43-54 | 46-65 44-57 |
| c Widest Calf | 31-39 29-33 | 33-41 31-35 | 34-43 33-37 | 36-45 35-39 | 37-47 36-41 | 39-49 38-43 | 40-51 39-45 | 42-53 41-47 | 43-55 42-49 | 45-58 44-52 | 46-61 45-55 | 48-64 47-58 |
| b Ankle | 19-21 | 20-22 | 21-23 | 22-24 | 23-25 | 24-26 | 25-27 | 26-28 | 27-29 | 28-31 | 29-33 | 30-36 |

Measurements falling outside of the above chart?
See our **Custom Made** solutions on Page 36 ▶

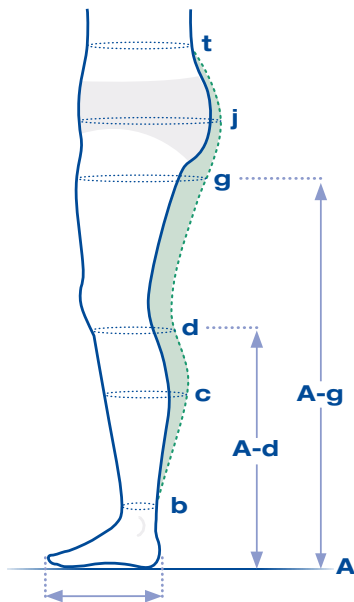

Length

Straight Distance cm

| | SHORT LEG | REGULAR LEG | LONG LEG |
|-------------------------------|-----------|-------------|----------|
| A-g Floor to Thigh Top | 57-67 | 68-80 | 81-90 |

| | | | |
|--------------------------------|-------|-------|-------|
| A-d Floor to Below Knee | 33-37 | 38-43 | 44-48 |
|--------------------------------|-------|-------|-------|

* Leg length options are unavailable for anklets.

Foot

Straight Distance cm

| | SHORT FOOT | REGULAR FOOT | LONG FOOT |
|-----------------------------------|------------|--------------|-----------|
| Heel to End of Longest Toe | 21-23 | 23-26 | 26-29 |

Standard Options

Standard options must be specified when placing an order.

| | | | |
|-----------------|-------------------------------------|-------------------------------------|----------|
| FOOT | Closed toe | Open toe | Footless |
| GRIP TOP | 3cm Strong Plain Below Knee Default | 5cm Strong Plain Thigh High Default | |
| COLOUR | Beige | Black | |

Special Options



All our special options are fully documented in the **Hosiery Appendix** Page 39 ▶

Increase **panty compression**, switch to **male gusset**, add **fastenings** to aid donning, change the **grip top style**, or adjust the **heel and knee stitching** based on your patients activity levels.

Pertex

Below Knee, Thigh High & Tights

Material Composition: Polyamide, Elastane

Firm knit with soft touch making it a go-to option for reliable medium compression needs.

Haddenham medical compression hosiery utilises high-quality, long-lasting yarns that deliver RAL standard graduated compression to the lower limb.

Strong

Robust yet soft

Suitable for moderate to severe oedema, but with a soft, medium-density structure.

Post-surgical solution

Ideal post-decongestive or post-liposuction garment



Compression

| Class | mmHg | Stiffness | Well suited for | VENOUS DISEASE | LIPOEDEMA |
|--------------------|-------|-----------|---|--|--|
| RAL Class 2 | 23-32 | Medium | LYMPHOEDEMA Moderate shape distortion, deep to very deep pitting. | Telangiectasia, thread, reticular or broken veins. Varicose veins/ recurrent varicose veins. | Uneven skin with indentations in the fat tissue and lipomas. |
| RAL Class 3 | 34-46 | High | Obesity-related lymphoedema or lymphoedema where there is excess subcutaneous fibrosis and fat, post liposuction for lymphoedema. | Pigmentation or eczema, Lipodermatosclerosis or atrophie blanche. Corona phlebectatica. Healed Venous Ulcer. | Large deformations of fat, lipolymphoedema, large fat nodules and lipomas. |

Codes


| |  |  |  |  | | |  | | |
|--------------------|---|---|---|--|--------|-----------|---|--------|-----------|
| | Anklet* | Below Knee + GRIP TOP | Thigh High + GRIP TOP | LEFT LEG | BOTH | RIGHT LEG | LEFT LEG | BOTH | RIGHT LEG |
| RAL Class 2 | 092 | 921 920 | 922 923 | 925 | 924 | 926 | 928 | 927 | 929 |
| RAL Class 3 | 093 | 931 930 | 932 933 | 935 | 934 | 936 | - | - | - |
| Unit of Sale | Pair | Pair | Pair | Single | Single | Single | Single | Single | Single |

Sizing

Extra wide sizes are shown in green

| Circumferences cm | 4 | 5 | 6 | 7 | 8 | 9 | 10 | 11 | 12 | 13 | 14 | 15 |
|----------------------|----------------|----------------|----------------|----------------|----------------|----------------|----------------|----------------|----------------|----------------|----------------|----------------|
| t Waist | 65-75 | 70-80 | 70-80 | 75-85 | 75-85 | 80-90 | 80-90 | 85-95 | 85-95 | 90-105 | 90-105 | 95-110 |
| j Buttocks | 85-105 | 90-110 | 90-110 | 95-115 | 95-115 | 100-120 | 100-120 | 105-125 | 105-125 | 110-130 | 110-130 | 115-135 |
| g Thigh Top | 46-48 44-46 | 48-50 46-48 | 50-52 48-50 | 52-54 50-52 | 54-56 52-54 | 56-58 54-56 | 58-60 56-58 | 60-62 58-60 | 62-64 60-62 | 64-66 62-64 | 66-68 64-66 | 68-70 66-68 |
| d Below Knee | 29-31 28-30 | 31-33 30-31 | 32-34 31-33 | 34-36 33-34 | 35-37 34-36 | 37-39 36-37 | 38-40 37-39 | 40-42 39-40 | 41-43 40-41 | 43-45 41-43 | 44-47 43-45 | 46-48 44-47 |
| c Widest Calf | 31-33 29-31 | 33-35 31-33 | 34-36 33-35 | 36-38 35-36 | 37-39 36-38 | 39-41 38-39 | 40-42 39-41 | 42-44 41-42 | 43-45 42-44 | 45-47 44-45 | 46-49 45-47 | 48-50 47-49 |
| b Ankle | 19-20 | 20-21 | 21-22 | 22-23 | 23-24 | 24-25 | 25-26 | 26-27 | 27-28 | 28-29 | 29-30 | 30-31 |

Measurements falling outside of the above chart?
See our **Custom Made** solutions on Page 36 ▶

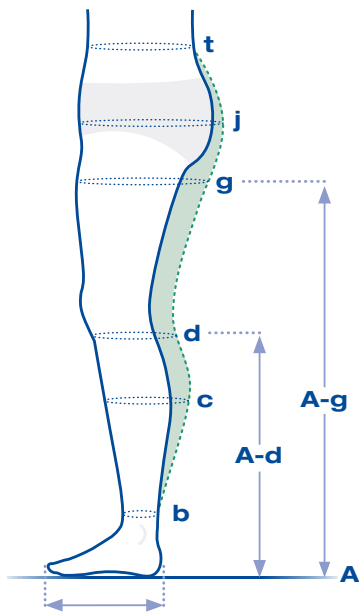

Length

Straight Distance cm

A-g Floor to Thigh Top

SHORT LEG

REGULAR LEG

LONG LEG

60-67

68-75

76-82

A-d Floor to Below Knee

33-37

38-42

43-46

* Leg length options are unavailable for anklets.

Foot

Straight Distance cm

Heel to End of Longest Toe

SHORT FOOT

REGULAR FOOT

LONG FOOT

21-23

23-26

26-29

Standard Options

Standard options must be specified when placing an order.

FOOT



Closed toe

Open toe

Footless

GRIP TOP



3cm Strong Plain Below Knee Default



5cm Strong Plain Thigh High Default

COLOUR



Beige



Black

Special Options



All our special options are fully documented in the **Hosiery Appendix Page 39** ▶

Increase **panty compression**, switch to **male gusset**, add **fastenings** to aid donning, change the **grip top style**, or adjust the **heel and knee stitching** based on your patients activity levels.

Goldpunkt

Below Knee, Thigh High & Tights

Material Composition: Polyamide, Elastane

Made for stubborn and rebound oedema support, with silky low friction on the inside to make it easy to apply.

Tough

High density body yarns

For high stiffness and oedema control

Smooth inner texture

For easier donning at high compression



Compression

| Class | mmHg | Stiffness | Well suited for | VENOUS DISEASE | LIPOEDEMA |
|--------------------|-------|-----------|--|--|--|
| RAL Class 2 | 23-32 | Medium | LYMPHOEDEMA Obesity-related lymphoedema or lymphoedema with moderate shape distortion and deep pitting. | Telangiectasia, thread, reticular or broken veins. Varicose veins/ recurrent varicose veins. | Uneven skin with indentations in the fat tissue and lipomas. |
| RAL Class 3 | 34-46 | Medium | Obesity-related lymphoedema or lymphoedema with moderate to extensive shape distortion,woody and/or very deep pitting tissues. | Pigmentation or eczema, Lipodermatosclerosis or atrophie blanche. Corona phlebectatica. Healed venous ulcer. | Large deformations of fat. |
| RAL Class 4 | 49+ | High | Obesity-related lymphoedema or lymphoedema with extensive shape distortion, fat depositions, fibrosis and warty overgrowths, indicating elephantiasis. | Suitable for use in venous disorders where oedema and shape distortion is extensive. | Large deformations of fat, lipolymphoedema, large fat nodules and lipomas. |


Codes

| |  |  |  |  |  |
|--------------------|---|---|---|--|---|
| | Anklet* | Below Knee + GRIP TOP | Thigh High + GRIP TOP | Thigh High + Waistband | Tights |
| | | | | LEFT LEG PAIR RIGHT LEG | LEFT LEG SINGLE RIGHT LEG |
| RAL Class 2 | 094 | 941 940 | 942 943 | 945 944 946 | |
| RAL Class 3 | 096 | 961 960 | 962 963 | 965 964 966 | Available as Custom Made |
| RAL Class 4 | 098 | 981 980 | 982 983 | 985 984 986 | |
| Unit of Sale | Pair | Pair Pair | Pair Pair | Single Single Single | Single Single Single |

Sizing

Extra wide sizes are shown in green

| Circumferences cm | 4 | 5 | 6 | 7 | 8 | 9 | 10 | 11 | 12 | 13 | 14 | 15 |
|----------------------|----------------|----------------|----------------|----------------|----------------|----------------|----------------|----------------|----------------|----------------|----------------|----------------|
| t Waist | 65-75 | 70-80 | 70-80 | 75-85 | 75-85 | 80-90 | 80-90 | 85-95 | 85-95 | 90-105 | 90-105 | 95-110 |
| j Buttocks | 85-105 | 90-110 | 90-110 | 95-115 | 95-115 | 100-120 | 100-120 | 105-125 | 105-125 | 110-130 | 110-130 | 115-135 |
| g Thigh Top | 46-48 44-46 | 48-50 46-48 | 50-52 48-50 | 52-54 50-52 | 54-56 52-54 | 56-58 54-56 | 58-60 56-58 | 60-62 58-60 | 62-64 60-62 | 64-66 62-64 | 66-68 64-66 | 68-70 66-68 |
| d Below Knee | 29-31 28-30 | 31-33 30-31 | 32-34 31-33 | 34-36 33-34 | 35-37 34-36 | 37-39 36-37 | 38-40 37-39 | 40-42 39-40 | 41-43 40-41 | 43-45 41-43 | 44-47 43-45 | 46-48 44-47 |
| c Widest Calf | 31-33 29-31 | 33-35 31-33 | 34-36 33-35 | 36-38 35-36 | 37-39 36-38 | 39-41 38-39 | 40-42 39-41 | 42-44 41-42 | 43-45 42-44 | 45-47 44-45 | 46-49 45-47 | 48-50 47-49 |
| b Ankle | 19-20 | 20-21 | 21-22 | 22-23 | 23-24 | 24-25 | 25-26 | 26-27 | 27-28 | 28-29 | 29-30 | 30-31 |

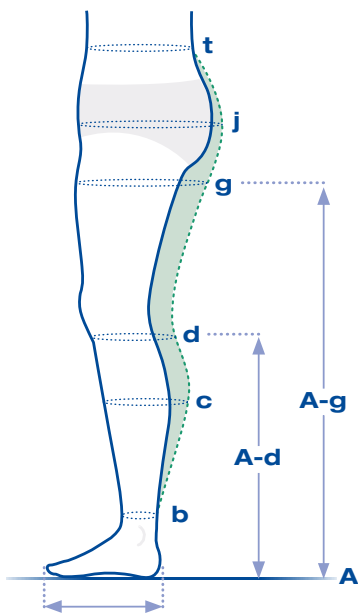

Measurements falling outside of the above chart?
See our **Custom Made** solutions on Page 36 ►

Length

Straight Distance cm

A-g Floor to Thigh Top

SHORT LEG

REGULAR LEG

LONG LEG

60-67

68-75

76-82

A-d Floor to Below Knee

33-37

38-42

43-46

* Leg length options are unavailable for anklets.

Foot

Straight Distance cm

Heel to End of Longest Toe

SHORT FOOT

REGULAR FOOT

LONG FOOT

21-23

23-26

26-29

Standard Options

Standard options must be specified when placing an order.

| | | | |
|-----------------|---|---|---|
| FOOT |  |  |  |
| | Closed toe | Open toe | Footless |
| GRIP TOP |  | 3cm Strong Plain Below Knee Default |  |
| | | | 5cm Strong Plain Thigh High Default |
| COLOUR |  |  | |
| | Beige | Black | |

Special Options



All our special options are fully documented in the **Hosiery Appendix** Page 39 ►

Increase **panty compression**, switch to **male gusset**, add **fastenings** to aid donning, change the **grip top style**, or adjust the **heel and knee stitching** based on your patients activity levels.

Below Knee, Thigh High & Tights

Material Composition: Polyamide, Elastane

Our most versatile fabric—soft, stretchy, and easy to don and doff, with wide colour and pattern options.

Haddenham medical compression hosiery utilises high-quality, long-lasting yarns that deliver RAL standard graduated compression to the lower limb.



Versatile




Conforming

Conforms to limb shape for comfort

Stylish




Available in 10 colours and 5 patterns

Compression

| Class | mmHg | Stiffness | Well suited for | LYMPHOEDEMA | VENOUS DISEASE | LIPOEDEMA |
|--|-------|-----------|--|-------------|--|---|
|  Class 1 | 18-21 | Low | Subclinical Oedema. Mild watery oedema. Post lymphatic surgery. | | Tired aching legs | Smooth skin with an increase of enlarged subcutaneous fat tissue. |
|  Class 2 | 23-32 | Low | Swelling that subsides overnight. Slight tissue pitting. | | Telangiectasia, thread, broken or reticular veins, varicose veins, recurrent varicose veins. | Uneven skin with indentations in the fat tissue and lipomas. |
|  Class 3 | 34-46 | Low | Swelling rarely reduces with elevation. Mild shape distortion with healthy skin. | | Healed venous ulcer. | N/A |

Codes



| | Anklet* | Below Knee | Thigh High | Thigh High + Waistband | Tights | | | | | | |
|--|---------|------------|------------|------------------------|--------|-----------|----------|--------|-----------|--------|--------|
| | | + GRIP TOP | + GRIP TOP | LEFT LEG | BOTH | RIGHT LEG | LEFT LEG | BOTH | RIGHT LEG | | |
|  Class 1 | 050 | 501 | 500 | 505 | 509 | 517 | 513 | 521 | 527 | 525 | 529 |
|  Class 2 | 053 | 533 | 532 | 537 | 541 | 549 | 545 | 553 | 559 | 557 | 561 |
|  Class 3 | 056 | 565 | 564 | 569 | 573 | 581 | 577 | 585 | 591 | 589 | 593 |
| Unit of Sale | Pair | Pair | Pair | Pair | Pair | Single | Single | Single | Single | Single | Single |

Stock Options

All other variants made-to-order in 12-14 days




| Class | Grip Top | Sizes | Widths | Leg | Foot | Toes | Colour |
|-------|----------|-------|----------|---------|---------|------|---------|
| 2 | None | 1-8 | Standard | Regular | Regular | Open | Caramel |

Sizing

Extra wide sizes are shown in green

| Circumferences cm | 1 | 2 | 3 | 4 | 5 | 6 | 7 | 8 |
|----------------------|----------------|----------------|----------------|----------------|----------------|----------------|----------------|----------------|
| t Waist | 65-75 | 70-80 | 75-85 | 80-90 | 85-95 | 90-105 | 95-110 | 100-120 |
| j Buttocks | 85-105 | 90-110 | 95-115 | 100-120 | 105-125 | 110-130 | 115-135 | 120-140 |
| g Thigh Top | 46-55 43-48 | 48-59 45-51 | 52-63 49-55 | 56-66 53-59 | 60-69 57-63 | 64-72 61-66 | 67-75 64-69 | 70-78 67-72 |
| c Widest Calf | 31-35 29-33 | 33-37 31-35 | 35-39 33-37 | 37-42 35-39 | 39-45 37-42 | 41-47 39-45 | 43-52 41-48 | 45-56 43-51 |
| b Ankle | 18-20 | 20-22 | 22-24 | 24-26 | 26-28 | 28-30 | 30-32 | 32-34 |

Measurements falling outside of the above chart?
See our **Custom Made** solutions on Page 36 ▶

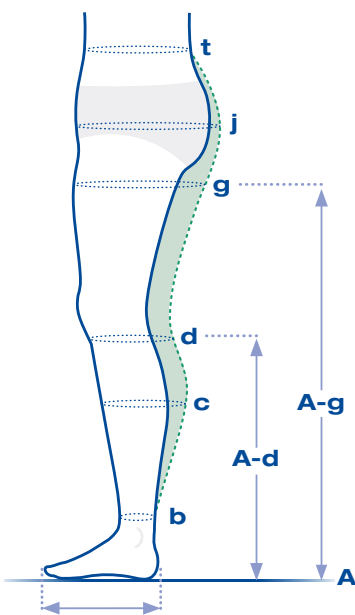

Length

Straight Distance cm

| | SHORT LEG | REGULAR LEG | LONG LEG |
|-------------------------------|-----------|-------------|----------|
| A-g Floor to Thigh Top | 57-67 | 68-80 | 81-90 |

| | SHORT LEG | REGULAR LEG | LONG LEG |
|--------------------------------|-----------|-------------|----------|
| A-d Floor to Below Knee | 33-37 | 38-43 | 44-48 |

* Leg length options are unavailable for anklets.

Foot

Straight Distance cm

| | SHORT FOOT | REGULAR FOOT | LONG FOOT |
|-----------------------------------|------------|--------------|-----------|
| Heel to End of Longest Toe | 21-23 | 23-26 | 26-29 |

Standard Options

Standard options must be specified when placing an order.

| | | | | | | |
|-------------------------------------|--|--|-----------|------------|-----------|---------|
| FOOT | Closed toe | Open toe | Footless | | | |
| GRIP TOP | 3cm Strong Plain Below Knee Default | 5cm Strong Plain Thigh High Default | | | | |
| COLOUR | Beige | Black | Burgundy | Caramel | Mocha | |
| | Navy | Pearl | Slate | Smoke | White | |
| PATTERNS Class 1 & 2 only | None | Diamond | Honeycomb | Narrow Rib | Pinstripe | Zig-zag |

Special Options

All our special options are fully documented in the **Hosiery Appendix Page 39** ▶

Increase **party compression**, change **waistband type**, change the **grip top style**, and browse reference **photos of colours and patterns**.



Veni

Compression Shorts

Material Composition: Polyamide, Elastane

Veni compression shorts are made from specially sized Veni RAL Class 2 fabric, to deliver a stiffer and higher compression than standard compression tights.

Haddenham medical compression hosiery utilises high-quality, long-lasting yarns that deliver RAL standard graduated compression to the lower limb.


Versatile

Compressive Panty

For effective treatment of genital oedema

Graduated Leg Compression

Keeps legs comfortable

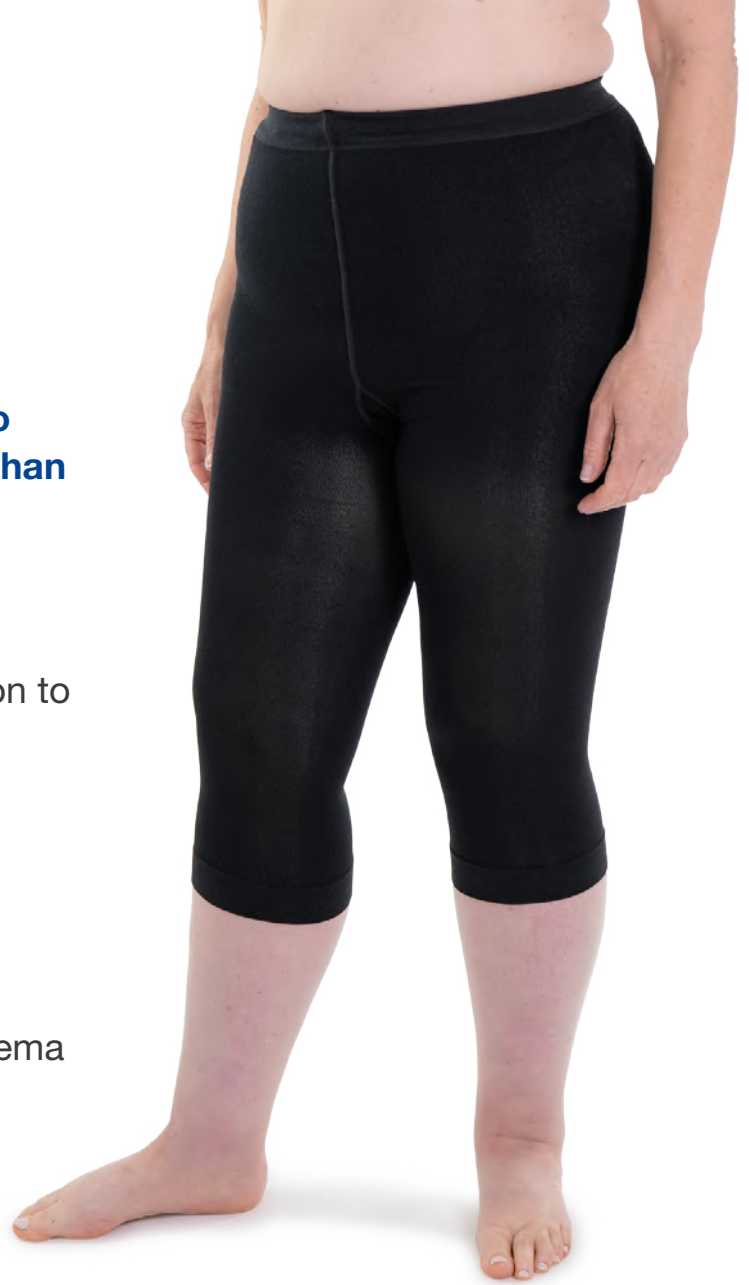

Compression

| Class | mmHg | Stiffness | Recommended for |
|-----------------------------|-------|-----------|---|
| RAL Class 2 | 23-32 | Low | Truncal and genital oedema, lipoedema affecting the hips and buttocks |

Codes All variants held in stock.



Cycling Shorts




Capri Shorts

| | | |
|-----------------------------|-----|-----|
| RAL Class 2 | 558 | 560 |
|-----------------------------|-----|-----|

| | | |
|--------------|--------|--------|
| Unit of Sale | Single | Single |
|--------------|--------|--------|

Sizing

| Circumferences cm | XS | S | M | L | XL | XXL | XXXL |
|----------------------|-----------|----------|----------|----------|-----------|------------|-------------|
| t Waist | 65-75 | 75-85 | 85-95 | 95-105 | 105-115 | 115-125 | 125-135 |
| j Buttocks | 85-95 | 95-105 | 105-115 | 115-125 | 125-135 | 135-145 | 145-155 |
| g Thigh Top | 43-51 | 47-55 | 49-56 | 55-63 | 59-67 | 63-71 | 67-74 |
| f Mid-Thigh | 43-48 | 47-52 | 49-54 | 55-61 | 59-65 | 63-69 | 67-72 |
| d Below Knee | 25-28 | 27-30 | 29-33 | 31-35 | 35-39 | 37-41 | 40-44 |
| c Widest Calf | 27-30 | 29-32 | 33-37 | 36-40 | 39-43 | 42-46 | 45-51 |

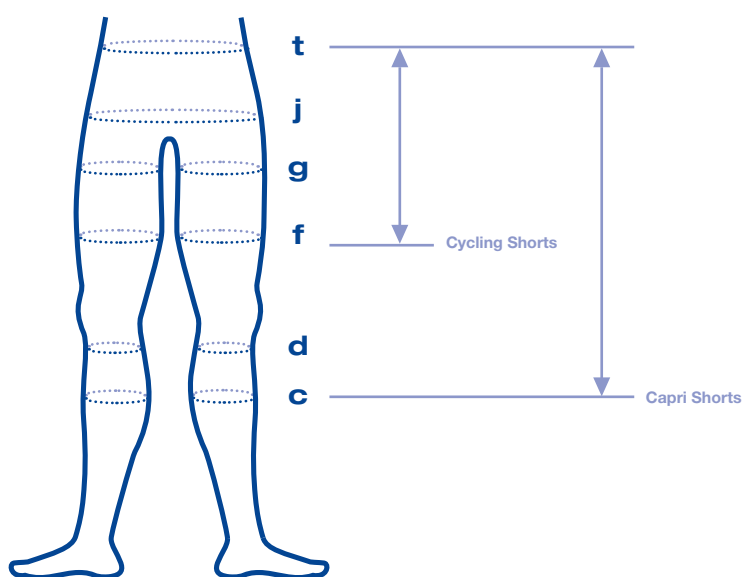

Options

Colour



Beige



Black

Star Cotton

Below Knee, Thigh High & Tights

Material Composition: Polyamide, Elastane, Cotton

100% cotton fabric in contact with skin – perfect for sensitive skin with superb comfort and support.

Haddenham medical compression hosiery utilises high-quality, long-lasting yarns that deliver RAL standard graduated compression to the lower limb.

Comfort

High quality cotton

Soft and breathable for all day comfort

Opaque body yarns

Similar in appearance to active-wear or socks



Compression

| Class | mmHg | Stiffness | Well suited for | VENOUS DISEASE | LIPOEDEMA |
|--------------------|-------|-----------|---|--|---|
| RAL Class 1 | 18-21 | Low | LYMPHOEDEMA Subclinical Oedema. Mild watery oedema with sensitive skin conditions | Subclinical venous disorders / at risk of venous disorders with sensitive skin conditions. | Smooth skin with an increase of enlarged subcutaneous fat tissue. |
| RAL Class 2 | 23-32 | Low | Swelling rarely reduces with elevation. Mild shape distortion with sensitive skin conditions. | Pigmentation or eczema, Lipodermatosclerosis or atrophie blanche. Corona phlebectatica. | Uneven skin with indentations in the fat tissue and lipomas. |


Codes

| |  |  |  |  |  |  |  | | | | |
|--------------------|---|---|---|--|---|---|---|--------|-----------|--------|--------|
| | Anklet* | Below Knee | Thigh High | Thigh High + Waistband | Tights | | | | | | |
| | | + GRIP TOP | + GRIP TOP | LEFT LEG | BOTH | RIGHT LEG | LEFT LEG | BOTH | RIGHT LEG | | |
| RAL Class 1 | 034 | 341 | 340 | 345 | 349 | 357 | 353 | 361 | 367 | 365 | 369 |
| RAL Class 2 | 074 | 741 | 740 | 742 | 743 | 747 | 746 | 748 | 750 | 749 | 751 |
| Unit of Sale | Pair | Pair | Pair | Pair | Pair | Single | Single | Single | Single | Single | Single |

Sizing

Extra wide sizes are shown in green

| Circumferences cm | 1 | 2 | 3 | 4 | 5 | 6 | 7 | 8 |
|----------------------|----------------|----------------|----------------|----------------|----------------|----------------|----------------|----------------|
| t Waist | 65-75 | 70-80 | 75-85 | 80-90 | 85-95 | 90-105 | 95-110 | 100-120 |
| j Buttocks | 85-105 | 90-110 | 95-115 | 100-120 | 105-125 | 110-130 | 115-135 | 120-140 |
| g Thigh Top | 46-55 43-48 | 48-59 45-51 | 52-63 49-55 | 56-66 53-59 | 60-69 57-63 | 64-72 61-66 | 67-75 64-69 | 70-78 67-72 |
| c Widest Calf | 31-35 29-33 | 33-37 31-35 | 35-39 33-37 | 37-42 35-39 | 39-45 37-42 | 41-47 39-45 | 43-52 41-48 | 45-56 43-51 |
| b Ankle | 18-20 | 20-22 | 22-24 | 24-26 | 26-28 | 28-30 | 30-32 | 32-34 |

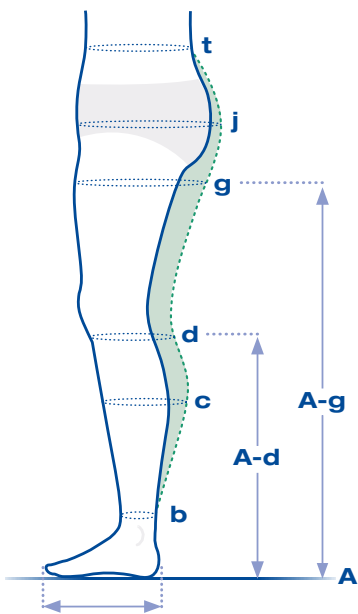

Measurements falling outside of the above chart? See our **Custom Made** solutions on Page 36 ▶

Length

Straight Distance cm

A-g Floor to Thigh Top

SHORT LEG

REGULAR LEG

LONG LEG

60-67

68-75

76-82

A-d Floor to Below Knee

33-37

38-42

43-46

* Leg length options are unavailable for anklets.

Foot

Straight Distance cm

Heel to End of Longest Toe

SHORT FOOT

REGULAR FOOT

LONG FOOT



21-23

23-26

26-29

Standard Options

Standard options must be specified when placing an order.

| | | | |
|-----------------|---|---|---|
| FOOT |  |  |  |
| | Closed toe | Open toe | Footless |
| GRIP TOP |  | 3cm Strong Plain Below Knee Default |  |
| | | | 5cm Strong Plain Thigh High Default |
| COLOUR |  |  | |
| | Beige | Black | |

Special Options

All our special options are fully documented in the **Hosiery Appendix Page 39** ▶

Increase **panty compression**, change **waistband type**, change the **grip top style**.



Doktus

Below Knee, Thigh High & Tights

Material Composition: Polyamide, Elastane

Robust circular knit for moderate lymphoedema that's soft and comfortable to wear, with the appearance of active-wear.

Haddenham medical compression hosiery utilises high-quality, long-lasting yarns that deliver RAL standard graduated compression to the lower limb.



Durable

Thicker, opaque body yarns

Similar in appearance to active-wear or leggings

Stiffer circular knit

Suitable for patients needing stronger compression

Compression

| Class | mmHg | Stiffness | Well suited for | VENOUS DISEASE | LIPOEDEMA |
|--------------------|-------|-----------|--|--|--|
| RAL Class 2 | 23-32 | Low | LYMPHOEDEMA Moderate oedema which rarely reduces overnight with moderate pitting tissues. | Telangiectasia, thread, broken or reticular veins. Varicose veins, recurrent varicose veins. | Uneven skin with indentations in the fat tissue and lipomas. |
| RAL Class 3 | 34-46 | Low | Moderate oedema which rarely reduces overnight and is prone to rebound. | Healed venous ulcer. | N/A |

Codes


| |  |  |  |  | | |  | | |
|--------------------|---|---|---|--|--------|-----------|---|--------|-----------|
| | Anklet* | Below Knee + GRIP TOP | Thigh High + GRIP TOP | LEFT LEG | BOTH | RIGHT LEG | LEFT LEG | BOTH | RIGHT LEG |
| RAL Class 2 | 070 | 701 700 | 702 703 | 707 | 706 | 708 | 717 | 709 | 718 |
| RAL Class 3 | 072 | 721 720 | 722 723 | 727 | 726 | 728 | 737 | 729 | 738 |
| Unit of Sale | Pair | Pair | Pair | Single | Single | Single | Single | Single | Single |

Sizing

Extra wide sizes are shown in green

| Circumferences cm | 1 | 2 | 3 | 4 | 5 | 6 | 7 | 8 |
|----------------------|----------------|----------------|----------------|----------------|----------------|----------------|----------------|----------------|
| t Waist | 65-75 | 70-80 | 75-85 | 80-90 | 85-95 | 90-105 | 95-110 | 100-120 |
| j Buttocks | 85-105 | 90-110 | 95-115 | 100-120 | 105-125 | 110-130 | 115-135 | 120-140 |
| g Thigh Top | 46-55 43-48 | 48-59 45-51 | 52-63 49-55 | 56-66 53-59 | 60-69 57-63 | 64-72 61-66 | 67-75 64-69 | 70-78 67-72 |
| c Widest Calf | 31-35 29-33 | 33-37 31-35 | 35-39 33-37 | 37-42 35-39 | 39-45 37-42 | 41-47 39-45 | 43-52 41-48 | 45-56 43-51 |
| b Ankle | 18-20 | 20-22 | 22-24 | 24-26 | 26-28 | 28-30 | 30-32 | 32-34 |

Measurements falling outside of the above chart?
See our **Custom Made** solutions on Page 36 ▶

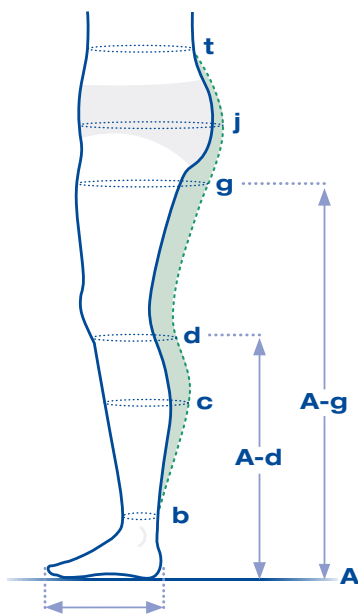

Length

Straight Distance cm

A-g Floor to Thigh Top

SHORT LEG

REGULAR LEG

LONG LEG

60-67

68-75

76-82

A-d Floor to Below Knee

33-37

38-42

43-46

* Leg length options are unavailable for anklets.

Foot

Straight Distance cm

Heel to End of Longest Toe

SHORT FOOT

REGULAR FOOT

LONG FOOT

21-23

23-26

26-29

Standard Options

Standard options must be specified when placing an order.

| | | | |
|-----------------|---|---|--|
| Foot |  Closed toe |  Open toe |  Footless |
| Grip Top |  3cm Strong Plain Below Knee Default |  5cm Strong Plain Thigh High Default | |
| Colour |  Beige |  Black |  Navy |

Special Options

All our special options are fully documented in the **Hosiery Appendix Page 39** ▶

Increase **party compression**, switch to **male gusset**, change the **grip top style**, and browse reference **photos of colours and patterns**.



Stride

Socks

Material Composition: Polyamide, Elastane

Stride is a circular knit below knee garment knitted in the likeness of a sports sock, combining a pile knit sole with medical grade compression yarns up the leg.

Haddenham medical compression hosiery utilises high-quality, long-lasting yarns that deliver RAL standard graduated compression to the lower limb.



Compression

| Class | mmHg | Stiffness | Well suited for | |
|--------------------|-------|-----------|--|--|
| LYMPHOEDEMA | | | | VENOUS DISEASE |
| RAL Class 1 | 18-21 | Low | Subclinical or Mild swelling with slight pitting and healthy skin. | Subclinical venous disorders /at risk of venous disorders with healthy skin. |
| RAL Class 2 | 23-32 | Low | Moderate swelling with slight pitting and healthy skin. | Telangiectasia, thread, broken or reticular veins. Varicose veins, recurrent varicose veins. |

Codes



Below Knee

| | |
|--------------------|------|
| RAL Class 1 | 820 |
| RAL Class 2 | 821 |
| Unit of Sale | Pair |

Stock Options

All other variants made-to-order in 12-14 days



Class

2

Sizes

XXS - XXL

Leg

Regular

Foot

Regular

Colour

Black

Active

Pile knit sole

Comfortable cushioning for people on their feet, or those keeping active.

Maximise movement

Helps maximise the benefits of movement to the lymphatic system.


Comfort

Four foot lengths for the perfect fit.



Sizing

| Circumferences cm | XXS | XS | S | M | L | XL | XXL |
|----------------------|-------|-------|-------|-------|-------|-------|-------|
| c Widest Calf | 29-33 | 31-35 | 33-37 | 35-39 | 37-42 | 39-45 | 41-48 |
| b Ankle | 18-20 | 20-22 | 22-24 | 24-26 | 26-28 | 28-30 | 30-32 |

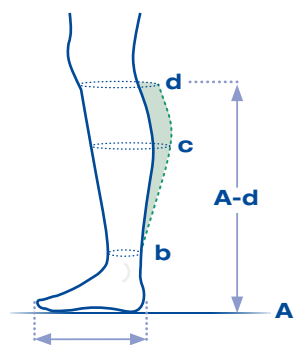

Length

Straight Distance cm

A-d Floor to Below Knee

SHORT LEG

REGULAR LEG

33-37

38-43

Foot

Straight Distance cm

Heel to End of Longest Toe

SHORT FOOT

REGULAR FOOT

LONG FOOT

EXTRA LONG

21-23

23-26

26-29

29-31

Options

Colour



Black



Grey



Navy



White

Microfine

Toecap

Material Composition: Polyamide, Elastane

Haddenham Microfine is an ultra-thin and smooth fabric that trims to fit without fraying.

Microfine Toecaps are an easy to measure, time-saving solution for oedema that extends to the toes.



Versatile

Time saving

No need to bandage toes

Trim to fit

Easily modified to desired toe length

Compression

mmHg

Well suited for

20-36


LYMPHOEDEMA

Positive Stemmer sign, fibrosis in the toes.
Mild to moderate shape distortion.

VENOUS DISEASE

In patients who present with oedema and are unable to wear closed toe hosiery. Prevention of toe oedema.

Sizing

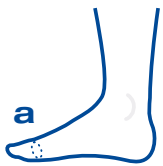

Circumferences cm

XXS

XS

S

M

L

XL

a Ball of Foot

19-24

21-23

23-25

25-27

27-29

29-31

Codes

All variants held **in stock**.

XXS

XS

S

M

L

XL



Sand

218S-0

218S-1

218S-2

218S-3

218S-4

218S-5



Black

218B-0

218B-1

218B-2

218B-3

218B-4

218B-5

Unit of Sale

Single

Single

Single

Single

Single

Single

Custom Made

Toecaps & Toemits

We aim to be flexible wherever possible by providing a made-to-measure service that offers unparalleled customisation and fit.



Bespoke

Made to Measure

Custom made toecaps are knitted to a patient's exact measurements, for comfort and conformity.

Download the [Custom Made Toecap](http://www.hadhealth.com.au/F0006a) form
www.hadhealth.com.au/F0006a ►

Special Options



All our special options are fully documented in the **Hosiery Appendix Page 39** ►

Add a **heel strap**, or **customise toes**.

Codes



Toecap



Toe mit

| | PERTEX LIGHT | PERTEX | GOLDPUNKT | MICROFINE | PERTEX LIGHT | PERTEX | GOLDPUNKT | MICROFINE |
|--------------------|--------------|----------|-----------|-----------|--------------|-------------|-------------|-----------|
| RAL Class 1 | PX-TOE-1 | - | - | - | PX-TOEMIT-1 | - | - | - |
| RAL Class 2 | - | PX-TOE-2 | GP-TOE-2 | - | - | PX-TOEMIT-2 | GP-TOEMIT-2 | - |
| RAL Class 3 | - | PX-TOE-3 | GP-TOE-3 | - | - | PX-TOEMIT-3 | GP-TOEMIT-3 | - |
| 20-36mmHg | - | - | - | MF-TOE | - | - | - | MF-TOEMIT |
| Unit of Sale | Single | Single | Single | Single | Single | Single | Single | Single |

Custom Made Medical Compression Hosiery

We aim to be flexible wherever possible by providing a made-to-measure service that offers unparalleled customisation and fit.

Bespoke

The widest choice

Select from our widest range of styles, fabric and options



Special Options



All our special options are fully documented in the **Hosiery Appendix Page 39** ▶






Increase **panty compression**, change **waistband type**, add **fastenings** to aid donning, change the **grip top style**, or adjust the **heel and knee stitching** based on your patients activity levels.



Download the Custom Made Hosiery form
www.hadhealth.com.au/F0001 ▶

Custom Made Flat Knit Codes

Pertex Light

| |  |  |  |  | | |  | | | | |
|--------------------|---|---|---|--|----------|--------|---|---------------|----------|--------|-----------|
| | Anklet | Below Knee + GRIP TOP | Thigh High + GRIP TOP | Thigh High + Waistband | LEFT LEG | BOTH | RIGHT LEG | Tights | LEFT LEG | BOTH | RIGHT LEG |
| RAL Class 1 | 091CM | 901CM 900CM | 902CM 903CM | 905CM | 904CM | 906CM | 908CM | 907CM | 909CM | | |
| Unit of Sale | Single | Single Single | Single Single | Single | Single | Single | Single | Single | Single | Single | Single |

Pertex

| |  |  |  |  | | |  | | | | |
|--------------------|---|--|--|---|----------|--------|--|---------------|----------|--------|-----------|
| | Anklet | Below Knee + GRIP TOP | Thigh High + GRIP TOP | Thigh High + Waistband | LEFT LEG | BOTH | RIGHT LEG | Tights | LEFT LEG | BOTH | RIGHT LEG |
| RAL Class 2 | 092CM | 921CM 920CM | 922CM 923CM | 925CM | 924CM | 926CM | 928CM | 927CM | 929CM | | |
| RAL Class 3 | 093CM | 931CM 930CM | 932CM 933CM | 935CM | 934CM | 936CM | 938CM | 937CM | 939CM | | |
| Unit of Sale | Single | Single Single | Single Single | Single | Single | Single | Single | Single | Single | Single | Single |

Goldpunkt

| |  |  |  |  | | |  | | | | |
|--------------------|---|---|---|--|----------|--------|---|---------------|----------|--------|-----------|
| | Anklet | Below Knee + GRIP TOP | Thigh High + GRIP TOP | Thigh High + Waistband | LEFT LEG | BOTH | RIGHT LEG | Tights | LEFT LEG | BOTH | RIGHT LEG |
| RAL Class 2 | 094CM | 941CM 940CM | 942CM 943CM GP-32 GP-32 + CM90 | 945CM | 944CM | 946CM | 948CM | 947CM | 949CM | | |
| RAL Class 3 | 096CM | 961CM 960CM | 962CM 963CM GP-33 GP-33 + CM90 | 965CM | 964CM | 966CM | 968CM | 967CM | 969CM | | |
| RAL Class 4 | 098CM | 981CM 980CM | 982CM 983CM | 985CM | 984CM | 986CM | 988CM | 987CM | 989CM | | |
| Unit of Sale | Single | Single Single | Single Single | Single | Single | Single | Single | Single | Single | Single | Single |

Custom Made Circular Knit Codes

Veni

| |  |  |  |  | | |  | | |
|--------------------|---|---|---|--|--------|-----------|---|--------|-----------|
| | Anklet | Below Knee | Thigh High | Thigh High + Waistband | | | Tights | | |
| | | + GRIP TOP | + GRIP TOP | LEFT LEG | BOTH | RIGHT LEG | LEFT LEG | BOTH | RIGHT LEG |
| RAL Class 1 | 050CM | 501CM 500CM | 505CM 509CM | 517CM | 513CM | 521CM | 527CM | 525CM | 529CM |
| RAL Class 2 | 053CM | 533CM 532CM | 537CM 541CM | 549CM | 545CM | 553CM | 559CM | 557CM | 561CM |
| RAL Class 3 | 056CM | 565CM 564CM | 569CM 573CM | 581CM | 577CM | 585CM | 591CM | 589CM | 593CM |
| Unit of Sale | Single | Single Single | Single Single | Single | Single | Single | Single | Single | Single |

Star Cotton

| |  |  |  |  | | |  | | |
|--------------------|---|--|--|---|--------|-----------|--|--------|-----------|
| | Anklet | Below Knee | Thigh High | Thigh High + Waistband | | | Tights | | |
| | | + GRIP TOP | + GRIP TOP | LEFT LEG | BOTH | RIGHT LEG | LEFT LEG | BOTH | RIGHT LEG |
| RAL Class 1 | 034CM | 341CM 340CM | 345CM 349CM | 357CM | 353CM | 361CM | 367CM | 365CM | 369CM |
| RAL Class 2 | 074CM | 741CM 740CM | 742CM 743CM | 747CM | 746CM | 748CM | 750CM | 749CM | 751CM |
| Unit of Sale | Single | Single Single | Single Single | Single | Single | Single | Single | Single | Single |

Doktus

| |  |  |  |  | | |  | | |
|--------------------|---|---|---|--|--------|-----------|---|--------|-----------|
| | Anklet | Below Knee | Thigh High | Thigh High + Waistband | | | Tights | | |
| | | + GRIP TOP | + GRIP TOP | LEFT LEG | BOTH | RIGHT LEG | LEFT LEG | BOTH | RIGHT LEG |
| RAL Class 2 | 070CM | 701CM 700CM | 702CM 703CM | 707CM | 706CM | 708CM | 717CM | 709CM | 718CM |
| RAL Class 3 | 072CM | 721CM 720CM | 722CM 723CM | 727CM | 726CM | 728CM | 737CM | 729CM | 738CM |
| Unit of Sale | Single | Single Single | Single Single | Single | Single | Single | Single | Single | Single |

Appendix

Contents

Full information regarding sizing and the numerous special options available.

Measuring



Sizing


Page 40 ▶

Special Options



Foot Options


Page 41 ▶

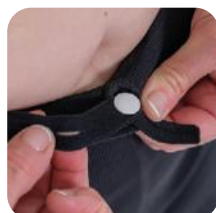

Waistbands

Page 42 ▶

Veni Exclusives

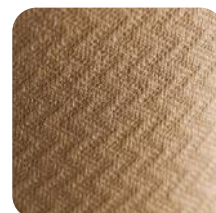

Patterns

Page 44 ▶



Closures

Page 41 ▶



Panty Options

Page 43 ▶




Colours

Page 44 ▶



Grip Tops

Page 42 ▶

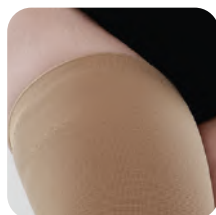

Other Options

Page 43 ▶

Special Option Codes

Page 45 ▶

Sizing



Foot Length

| | | SHORT FOOT | REGULAR FOOT | LONG FOOT |
|----------------------------|--------------|------------|--------------|-----------|
| HEEL TO END OF LONGEST TOE | Centimetres | 21-23 | 23-26 | 26-29 |
| | UK Shoe Size | 2-5 | 5-9 | 9-11 |
| | EU Shoe Size | 35-38 | 39-42 | 43-46 |

Reference sizing for Open Toe

| | | | | |
|---------------------------------|-------------|---------|---------|---------|
| HEEL TO BALL OF FOOT AT BIG TOE | Centimetres | 13-15cm | 15-18cm | 18-21cm |
|---------------------------------|-------------|---------|---------|---------|

Foot Options



Open Toe

Open toe finish on the foot.



Closed Toe

Closed toe finish on the foot.



Slant Toe

Open toe slant option. Please include inside and outside foot measurements.

Flat Knit Only



Footless

You can choose to finish your footless tights at any length. Keep in mind that without anchorage, footless garments might ride up so choose your finishing point with consideration of the leg shaping.



y-Heel

A y-heel is the **default fit**, offering a soft, contoured shape that accommodates natural movement around the ankle and heel.



t-Heel

A t-heel design redirects fabric tension across the front of the ankle to reduce pressure and irritation during upward foot movement while ensuring a secure, contoured fit around the heel.

Flat Knit Only

Closures



Zips

Zips can be added in any location but remember that this will alter the fabric properties at that location. For example, it won't stretch longitudinally where the zip is sewn in.

Flat Knit Only



Velcro Fastenings

The Velcro runs vertically down the length of the garment to oppose the two sides and close the garment.

Flat Knit Only



Velcro Straps

Velcro straps can be added in any location along the length of the leg. This however does not allow the garment to perform the same function as a wrapping device. The two edges need to be carefully aligned to prevent puckering.

Flat Knit Only

Grip Tops



Strong Plain

3cm / 5cm



Strong Lace

5cm




Fine Lace

5cm



Non-Silicone

4cm

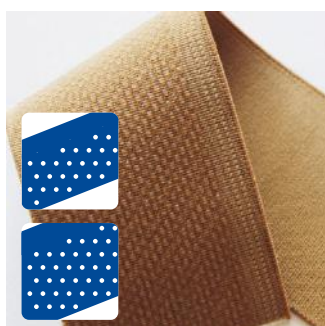

Strong Plain (Veni)

3cm / 5cm

Microdot design, only available on Veni garments.

Waistbands



Drawstring Waist

The adjustable drawstring waist is a good option if you need to accommodate wider hips and waist.



Elastic Waistband


Flat knit default, non-silicon elastic, 4cm wide waistband is made to fit the waist circumference. This option may not be suitable for wider hips and waist.



Knitted Waistband

Circular knit default, knitted ribbed waist designed to stretch.
Circular Knit Only

Panty Finishing

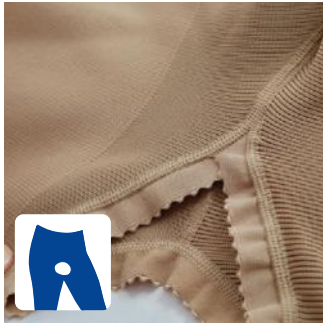

No Gusset


Open pubis.



Fly/Male Gusset

Closed pubis with horizontal or vertical fly at front.

Larger gusset to increase comfort for male genitals.

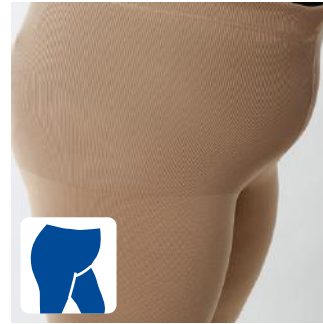

Loose Fit

Fully panty but made more generously with two seams to accommodate larger abdomens or to allow for extra comfort if someone is intolerant of compression over the abdomen.

Panty Compression



No Compression

Unless another option is specified panty section will be made without compression.



Half Compression

The panty section will be knitted with half compression. That is, every second row will have a compression inlay threads.



Full Compression

The garments will continue to be made with graduated compression all the way to the waist with the same thread and tension as what you have asked for in the legs.

Other Options



Pocket/Padding/Lining

To provide additional protection to delicate tissues.

Custom Made Only



Malleolus Pad

Padding at either side of the ankle for additional support and comfort.

Custom Made Only




Knee Shaping Flat Knit Only

No knee shaping is default.

90° knee reduced bunching when the leg remains in a fixed position, such as those who are wheelchair bound.

60° knee offering reduced bunching for those who are active but have day to day roles where sitting a lot.

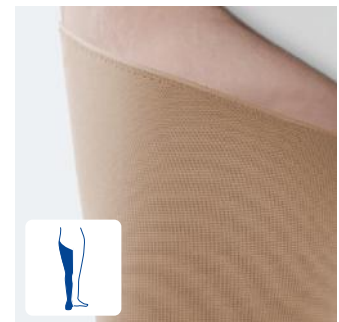

Slant Top Flat Knit Only

Slant finish to accommodate more garment coverage on the lateral side of the limb. Please include inside and outside measurements.

Custom Made Only

Veni Patterns

Available in Class 1 and Class 2 only.



Honeycomb



Zig Zag



Pin Stripe



Narrow Rib



Diamond

Veni Colours



Beige



Black



Pearl



Mocha*

*Class 1 and 2 only



Burgundy*

*Plain Grip Top Only
*Class 1 and 2 only



Slate



White



Smoke




Navy

Special Option Codes

| Category | Description | Sized Code | Custom Made Code |
|------------------------|-------------------------|------------|------------------|
| Foot | Footless | NF | NF-CM |
| | T-Heel | TH | TH-CM |
| Padding | Partial Lining | SPECIAL | LIN-CM |
| | Pocket | SPECIAL | POC-CM |
| | Malleolus Pads | SPECIAL | MAL-CM |
| | Pad | SPECIAL | PAD-CM |
| Grip Top | Non-Standard/Additional | SPECIAL | GT-CM |
| Closures | Zip | SPECIAL | ZIP-CM |
| | Velcro Fastening | SPECIAL | VELF-CM |
| | Velcro Strap | SPECIAL | VELS-CM |
| Finishing | Slant Foot | SF | SF-CM |
| | Slant Top | ST | ST-CM |
| | Heel Strap | HEEL | HEEL-CM |
| Tights | No Gusset | NG | NG-CM |
| | Fly Opening | FY | FY-CM |
| | Open Front | OF | OF-CM |
| | Loose Fit | LO | LO-CM |
| | Male Gusset | MG | MG-CM |
| | Elastic Waistband | EWB | - |
| | Half Compression Panty | HC | HC-CM |
| Full Compression Panty | FC | FC-CM | |

Compression

Residual pressure refers to the gradually decreasing pressure above the ankle, expressed as a percentage of the ankle pressure, and varies by compression class.

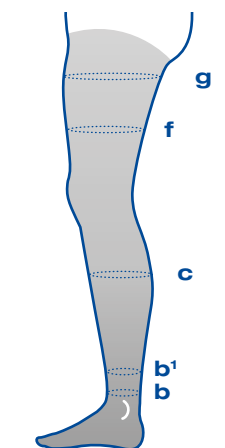

| Compression Class | mmHg | Residual Pressure Ratio | | |
|-------------------|-------|--|---------------|--------------------------------|
| | | b ¹ Start of calf muscle | c Mid-calf | f / g Mid-thigh / Thigh-top |
| Class 1 | 18-21 | 70-100% | 50-80% | 20-60% |
| Class 2 | 23-32 | 70-100% | 50-80% | 20-50% |
| Class 3 | 34-46 | 70-100% | 50-70% | 20-40% |
| Class 4 | 49+ | 70-100% | 50-70% | 20-40% |



Armsleeves



Haddenham medical compression armsleeves are made from high-quality, long-lasting yarns that deliver RAL standard graduated compression.



What makes it so patient-friendly?

Turn the page to discover all features [Page 48](#) ▶

Sized Flat Knits



Pertex Light

Page 52 ▶

Sized Circular Knits



Venex

Page 58 ▶



Star Cotton

Page 60 ▶

Custom Made



Armsleeves

Page 65 ▶



Pertex

Page 54 ▶



Doktus

Page 62 ▶



Microfine Glove

Page 64 ▶



Goldpunkt

Page 56 ▶

Haddenham medical compression armsleeves are made from high-quality, long-lasting yarns that deliver RAL standard graduated compression.

Freedom to choose



Pertex

6 versatile fabrics

Flat and Circular knit garments suitable for treatment of all stages of Lymphoedema, Venous disorders and Lipoedema.


[Fabric selection guide](#)

See page 50 ►

Reliable

Sleeves that can keep up with the most active of lifestyles, thanks to conforming fabrics and a variety of grip top options.



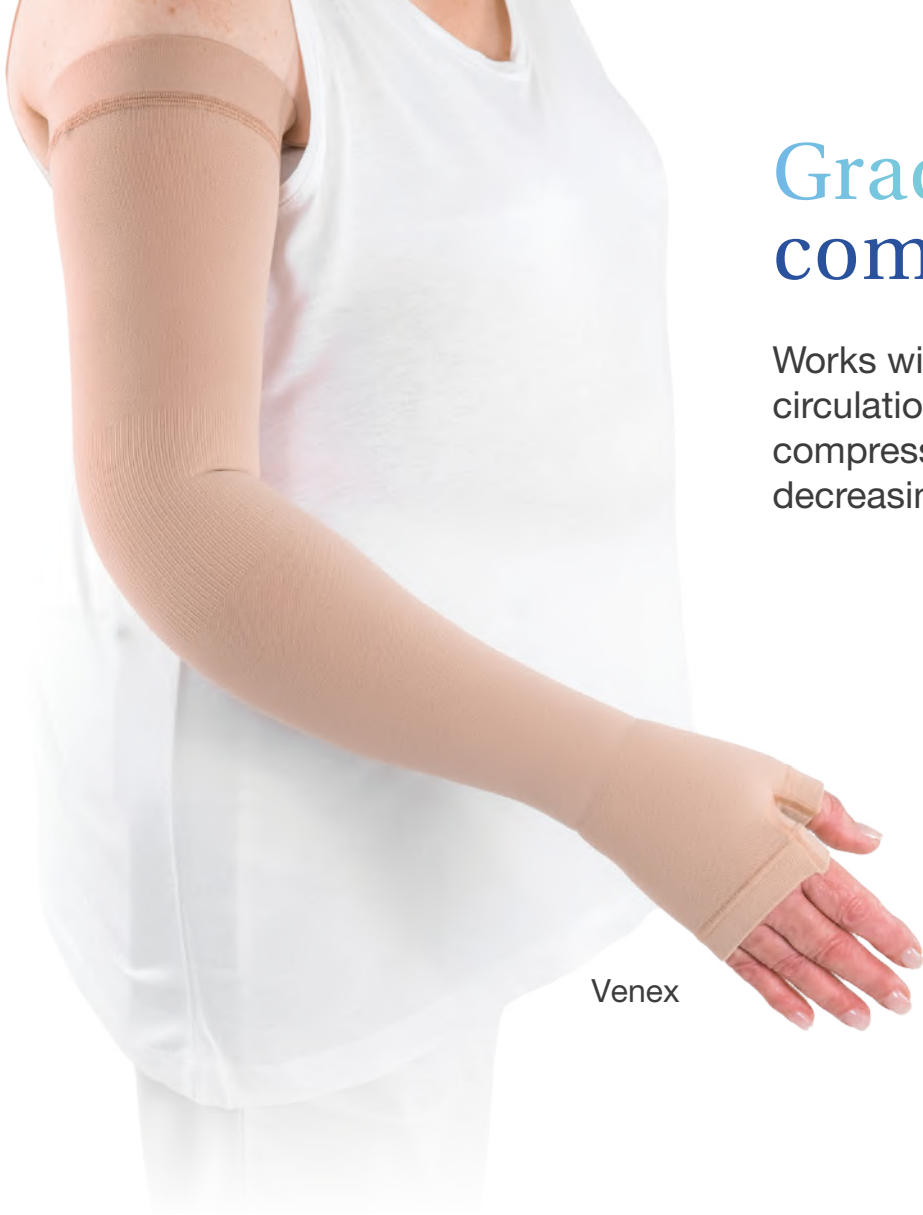

Venex

Graduated compression

Works with the body's natural circulation - concentrating the highest compression at the wrist and gradually decreasing up the limb.

Comfort

Elbow stretch zone and gentle thumb gusset for freedom in motion and maintaining dexterity.

Wide range of sizing to find the perfect fit.

Sized

Our sized armsleeves and gloves are **made-to-order**, with the care and quality that we believe each patient deserves.

We also retain **stock** of frequently ordered variants for an even faster turnaround.

_____ or _____

Custom Made

We aim to be flexible wherever possible by providing a made-to-measure service that offers unparalleled customisation and fit.



Microfine
Glove

Which fabric?

Please use the below maintenance phase pathway as a guide to which of our fabrics would be most suitable for your patient based on their presentation and symptoms.

How would you describe the level of your patient's oedema?

Early accumulation of fluid which subsides with elevation, with or without pitting.
ISL Stage 0-1

Tissue swelling rarely reduces with elevation and pitting is present.
ISL Stage 2

Pitting is absent with skin thickening fibrosis and excess subcutaneous fat.
ISL Late Stage 2 - Stage 3

Mild

Moderate

Severe

What is the level of shape distortion? ●

MILD EXTENSIVE
MODERATE

Is the oedema soft and watery? YES

NO

How would you describe the texture of the patient's tissue? ◆

WOODY PUTTY LIKE

What is the condition of the patient's skin? ▲

HEALTHY SKIN

SENSITIVE

FRAGILE

SLIGHT

What is the level of pitting? ●

MODERATE

DEEP

Are warty overgrowths with fat lobules present? NO YES

Venex

STAR Cotton

Pertex Light

Doktus

Pertex

GOLDPUNKT

References

▲ Skin conditions

Fragile Thin skin prone to skin tears

Sensitive the presence of skin conditions where the skin is prone to reactions such as redness and itching.

◆ Textures

Watery soft pliable tissues.

Putty putty consistency represents early fibrotic changes, will pit with firm pressure.

Woody firm woody texture is consistent with advanced fibrotic tissue changes and does not pit.

● Pitting Levels


Slight 1+ 2mm barely detectable immediate rebound

Moderate 2+ 4mm deep <10 second rebound

Deep 3+ 6 mm deep <30 second rebound

Very deep 4+ up to 8mm deep >30 second rebound

● Shape distortion

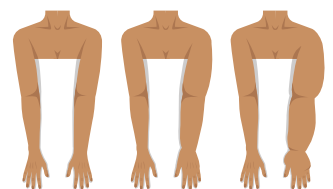

Mild

Moderate

Extensive

How much compression?

Graduated compression is highest at the wrist and gradually decreases as the yarns travel up the limb.

When deciding how much compression to apply also consider these 3 factors:

1. The nature of the fabric yarns and the way in which they are knitted together.
2. The fabric's ability to retain its structure under pressure. Measured as the increase in compression per centimetre increase in the circumference of the limb (mmHg/cm).
3. The resultant force exerted on the limb measured in mmHg.



Stiffness mmHg/cm

✓ Low = 0-2 | ✓ Medium = 2-4 | ✓ High = 4-6

| | Circular Knit | | | Flat Knit | | |
|------------------------|---------------|-------------|----------|--------------|----------|-----------|
| | Venex | STAR Cotton | Dokus | Pertex Light | Pertex | GOLDPUNKT |
| Pressure | | | | | | |
| Class 1 18 - 21mmHg | ✓ 1.2 | ✓ 1.4 | | ✓ 1.5 | | |
| Class 2 23-32mmHg | ✓ 1.3 | ✓ 1.5 | ✓ 1.6 | | ✓ 2.1 | ✓ 2.8 |
| Class 3 34-46mmHg | | | ✓ 1.8 | | ✓ 4.3 | ✓ 3.3 |

Page 58 ►

Page 60 ►

Page 62 ►

Page 52 ►

Page 54 ►

Page 56 ►

Pertex Light

Armsleeve, Glove and Mitten

Material Composition: Polyamide, Elastane

A unique stretchy flat knit that stretches with ease and hugs gently – ideal for fragile or sensitive skin.

Haddenham medical compression armsleeves utilise high-quality, long-lasting yarns that deliver RAL standard graduated compression to the upper limb.



Compression

| Class | mmHg | Stiffness | Well suited for |
|--------------------|-------|-----------|---|
| LYMPHOEDEMA | | | |
| LIPOEDEMA | | | |
| RAL Class 1 | 18-21 | Low | Mild soft watery oedema with slightly pitting tissues and fragile skin. Mild shape distortion. Palliative care. |
| | | | Smooth skin with an increase of enlarged subcutaneous fat tissue. |

Codes



Mitten



Glove



Sleeve

| | | LEFT | RIGHT | | + GRIP TOP |
|--------------------|--------|--------|--------|--------|------------|
| RAL Class 1 | 106 | 105L | 105R | 143 | 145 |
| Unit of Sale | Single | Single | Single | Single | Single |

Effortless

Stock Options

All other variants made-to-order in 12-14 days



Class

1

Grip Top

None
3cm Plain

Sizes

XS, S
M, L

Widths

Standard
Extra Wide

Length

Regular
Long

Colour

Beige

Uniquely stretchy

A stretchy knit makes for effortless donning and doffing, which means better patient compliance and self management.




Sizing

EXTRA WIDE shown in green

| Circumferences cm | XS | S | M | L |
|--------------------------|----------------|----------------|----------------|----------------|
| g Axilla | 26-32 20-26 | 28-35 21-28 | 34-48 24-34 | 40-50 30-40 |
| e Elbow Fold | 18-22 | 19-24 | 22-28 | 26-32 |
| c Wrist | 14-16 | 16-19 | 20-24 | 22-26 |
| a Base of Fingers | 17-18 | 18-20 | 20-22 | 22-24 |

| Length cm | REGULAR | LONG |
|----------------------------|----------------|-------------|
| c-g Wrist to Axilla | 40-44 | 44-50 |

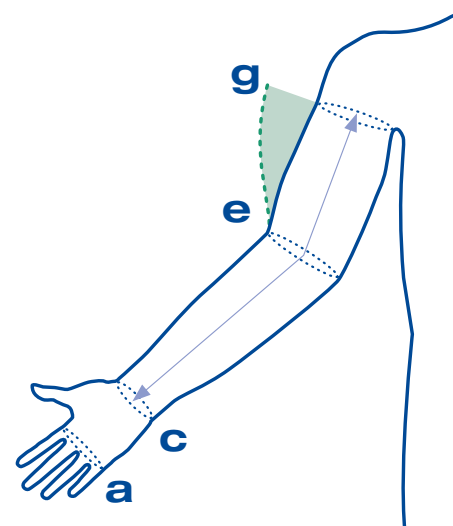

Options

Grip Top



3cm
Strong Plain



5cm
Strong Plain



5cm
Fine Lace



5cm
Strong Lace

Colour



Beige



Black

Shoulder Cap

Please state g-h measurement and left/right arm



With Velcro® Bra Attachment

Please state g-h measurement, g-t circumference and left/right arm



With Adjustable Strap

Armsleeve, Glove and Mitten

Material Composition: Polyamide, Elastane

Firm knit with soft touch making it a go-to option for reliable medium compression needs.

Haddenham medical compression armsleeves utilise high-quality, long-lasting yarns that deliver RAL standard graduated compression to the upper limb.



Compression

| Class | mmHg | Stiffness | Well suited for |
|--------------------|-------|-----------|--|
| | | | LYMPHOEDEMA Moderate oedema with moderate shape distortion and deep pitting tissues. Post liposuction for lymphoedema. |
| RAL Class 2 | 23-32 | Medium | LIPOEDEMA Uneven skin with indentations in the fat tissue and lipomas. |
| RAL Class 3 | 34-46 | High | Obesity-related lymphoedema or lymphoedema with extensive shape distortion, fat depositions, fibrosis and warty overgrowths, indicating elephantiasis. |
| | | | N/A |

Codes



Mitten



Glove



Sleeve



Sleeve with Mitten

| Class | Mitten | Glove | Sleeve | Sleeve with Mitten |
|--------------------|--------|--------------|------------|--------------------|
| RAL Class 2 | 206 | 205L 205R | + GRIP TOP | + GRIP TOP |
| RAL Class 3 | 306 | 305L 305R | | |

Custom Made only See page 65

Unit of Sale Single Single Single Single Single Single Single

Stock Options

All other variants made-to-order in 12-14 days



Class

2

Grip Top

None

Sizes

XS, S
M, L

Widths

Standard

Length

Regular

Colour

Beige

Conforming

Firm knitting

Firm consistent hold for moderate oedema



Sizing

Circumferences cm

XS

S

M

L

a Base of Fingers

18-19

19-20

21-23

23-25



Options

Grip Top



3cm Strong Plain



5cm Strong Plain



5cm Fine Lace



5cm Strong Lace

Colour



Beige



Black

Shoulder Cap

Please state g-h measurement and left/right arm



With Velcro® Bra Attachment

Please state g-h measurement, g-t circumference and left/right arm



With Adjustable Strap

Goldpunkt

Armsleeve, Glove and Mitten

Material Composition: Polyamide, Elastane

Made for stubborn and rebound oedema support, with silky low friction on the inside to make it easy to apply.

Haddenham medical compression armsleeves utilise high-quality, long-lasting yarns that deliver RAL standard graduated compression to the upper limb.



Compression

| Class | mmHg | Stiffness | Well suited for |
|--------------------|-------|-----------|---|
| RAL Class 2 | 23-32 | Medium | <p>LYMPHOEDEMA</p> <p>Obesity-related lymphoedema or lymphoedema with moderate shape distortion and very deep pitting. Post liposuction for lymphoedema.</p> <p>LIPOEDEMA</p> <p>Uneven skin with indentations in the fat tissue and lipomas.</p> |
| RAL Class 3 | 34-46 | High | <p>Obesity-related lymphoedema or lymphoedema where there is excess subcutaneous fibrosis and fat.</p> <p>N/A</p> |

Codes



Mitten



Glove



Sleeve

+ GRIP TOP



Sleeve with Mitten

+ GRIP TOP

| | | | |
|--------------------|-----|------|------|
| RAL Class 2 | 209 | 207L | 207R |
| RAL Class 3 | 309 | 307L | 307R |

Custom Made only See page 65

| | | | | | | | |
|--------------|--------|--------|--------|--------|--------|--------|--------|
| Unit of Sale | Single | Single | Single | Single | Single | Single | Single |
|--------------|--------|--------|--------|--------|--------|--------|--------|

Tough

High Compression

Effective treatment for the most stubborn oedema



Sizing

EXTRA WIDE shown in green

| Circumferences cm | XS | S | M | L |
|--------------------------|-----------|----------|----------|----------|
| a Base of Fingers | 18-19 | 19-20 | 21-23 | 23-25 |



Options

Grip Top



3cm Strong Plain



5cm Strong Plain



5cm Fine Lace



5cm Strong Lace

Colour



Beige



Black

Shoulder Cap

Please state g-h measurement and left/right arm



With Velcro® Bra Attachment

Please state g-h measurement, g-t circumference and left/right arm



With Adjustable Strap

Armsleeve

Material Composition: Polyamide, Elastane

A semi-sheer, lightweight and breathable fabric with microfibre softness.

Haddenham medical compression armsleeves utilise high-quality, long-lasting yarns that deliver RAL standard graduated compression to the upper limb.



Compression

| Class | mmHg | Stiffness | Well suited for |
|--------------------|-------|-----------|--|
| LYMPHOEDEMA | | | |
| LIPOEDEMA | | | |
| RAL Class 1 | 18-21 | Low | Mild soft watery oedema with healthy skin. |
| RAL Class 2 | 23-32 | Low | Mild soft watery oedema with slightly pitting tissues, mild shape distortion and healthy skin. |
| | | | Smooth skin with an increase of enlarged subcutaneous fat tissue. |
| | | | Uneven skin with indentations in the fat tissue and lipomas. |

Codes



Sleeve



Sleeve with Mitten

+ GRIP TOP

+ GRIP TOP

| | | | | |
|--------------------|--------|--------|--------|--------|
| RAL Class 1 | 103 | 101 | 104 | 102 |
| RAL Class 2 | 223 | 221 | 224 | 222 |
| Unit of Sale | Single | Single | Single | Single |

Stock Options

All other variants made-to-order in 12-14 days



| Class | Grip Top | Sizes | Widths | Length | Colour |
|-------|-----------|------------------|----------------------|---------------|--------|
| 2 | 3cm Plain | XXS, XS, S, M, L | Standard, Extra Wide | Regular, Long | Beige |

Gentle

Soft and stretchy
for easy application

Conforms
to limb shape for comfort




Sizing

EXTRA WIDE shown in green

| Circumferences cm | XXS | XS | S | M | L |
|--------------------------------------|----------------|----------------|----------------|----------------|----------------|
| g Axilla | 29-33 26-28 | 33-36 28-32 | 36-40 32-36 | 40-44 36-40 | 44-48 40-44 |
| e Elbow Fold | 21-23 | 23-25 | 25-27 | 28-30 | 30-32 |
| c¹ 4cm Above Wrist | 14-15 | 15-17 | 17-19.5 | 19.5-22.5 | 22.5-26 |
| a Base of Fingers | 13-16 | 14-18 | 16-21 | 18-24 | 21-27 |

| Length cm | REGULAR | LONG |
|----------------------------|---------|-------|
| c-g Wrist to Axilla | 38-44 | 44-50 |

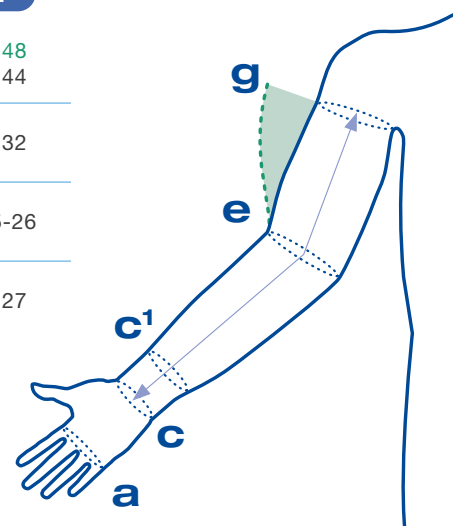

Options

Grip Top



3cm Strong Plain



5cm Strong Plain



5cm Fine Lace



5cm Strong Lace

Colour



Beige



Black



Caramel



Pearl



Slate



Smoke

Shoulder Cap

Please state g-h measurement and left/right arm



With Velcro® Bra Attachment

Please state g-h measurement, g-t circumference and left/right arm



With Adjustable Strap

Star Cotton

Armsleeve

Material Composition: Cotton, Polyamide, Elastane

Star Cotton is a circular knit fabric comprised of cotton covered yarns, which makes it an ideal choice for those with an intolerance to non-natural fibres.

Haddenham medical compression armsleeves utilise high-quality, long-lasting yarns that deliver RAL standard graduated compression to the upper limb.



Compression

| Class | mmHg | Stiffness | Well suited for LYMPHOEDEMA | LIPOEDEMA |
|---------|-------|-----------|--|---|
| Class 1 | 18-21 | Low | Mild soft watery oedema with sensitive skin. | Smooth skin with an increase of enlarged subcutaneous fat tissue. |
| Class 2 | 23-32 | Low | Mild soft watery oedema with slightly pitting tissues, mild shape distortion and sensitive skin. | Uneven skin with indentations in the fat tissue and lipomas. |

Codes



Sleeve

+ GRIP TOP



Sleeve with Mitten

+ GRIP TOP

| | | | | |
|--------------|--------|--------|--------|--------|
| Class 1 | 161 | 163 | 162 | 164 |
| Class 2 | 261 | 263 | 262 | 264 |
| Unit of Sale | Single | Single | Single | Single |

Comfort

100% Cotton against skin

Minimises skin irritation for those with an intolerance to non-natural fibres.

Soft wide cuff at wrist

To increase comfort and reduce irritation




Sizing





EXTRA WIDE shown in green
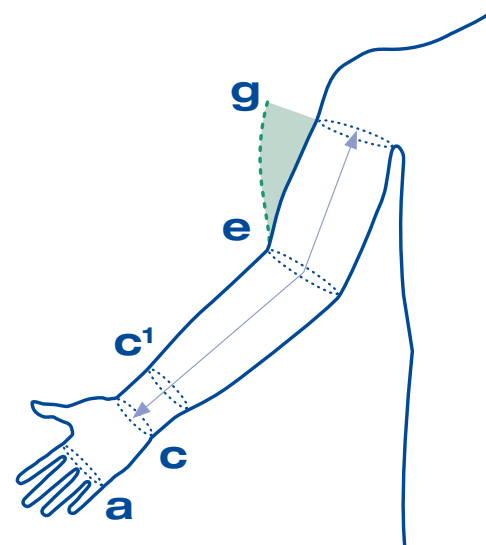
| Circumferences cm | XS | S | M | L |
|--------------------------------------|----------------|----------------|----------------|----------------|
| g Axilla | 33-36 28-32 | 36-40 32-36 | 40-44 36-40 | 44-48 40-44 |
| e Elbow Fold | 23-25 | 25-27 | 28-30 | 30-32 |
| c¹ 4cm Above Wrist | 15-17 | 17-19.5 | 19.5-22.5 | 22.5-26 |
| a Base of Fingers | 14-18 | 16-21 | 18-24 | 21-27 |

| Length cm | REGULAR | LONG |
|----------------------------|----------------|-------------|
| c-g Wrist to Axilla | 40-44 | 44-50 |



Options

| | | | | |
|-----------------|--|--|---|--|
| Grip Top |  3cm Strong Plain |  5cm Strong Plain |  5cm Fine Lace |  5cm Strong Lace |
|-----------------|--|--|---|--|

| | | |
|---------------|---|---|
| Colour |  Beige |  Black |
|---------------|---|---|

Shoulder Cap

Please state g-h measurement and left/right arm



With Velcro® Bra Attachment

Please state g-h measurement, g-t circumference and left/right arm



With Adjustable Strap

Doktus

Armsleeve

Material Composition: Polyamide, Elastane

Doktus is a circular knit fabric with thick, opaque body yarns.

Haddenham medical compression armsleeves utilise high-quality, long-lasting yarns that deliver RAL standard graduated compression to the upper limb.



Compression

| Class | mmHg | Stiffness | Well suited for | |
|---------|-------|-----------|---|--|
| | | | LYMPHOEDEMA | LIPOEDEMA |
| Class 2 | 23-32 | Medium | Moderate oedema which rarely reduces overnight with moderate pitting tissues and shape distortions. | Uneven skin with indentations in the fat tissue and lipomas. |
| Class 3 | 34-46 | High | Moderate oedema which rarely reduces overnight and is prone to rebound. | N/A |

Codes



Sleeve

+ GRIP TOP



Sleeve with Mitten

+ GRIP TOP

| | | | | |
|--------------|--------|--------|--------|--------|
| Class 2 | 215 | 201 | 217 | 202 |
| Class 3 | 216 | 301 | 219 | 302 |
| Unit of Sale | Single | Single | Single | Single |

Strength

Stiff Compression

Tackles stubborn rebound oedema

Sturdy body yarns


Distribute pressure and contain the oedema



Sizing

EXTRA WIDE shown in green

| Circumferences cm | XS | S | M | L |
|--------------------------------------|----------------|----------------|----------------|----------------|
| g Axilla | 33-36 28-32 | 36-40 32-36 | 40-44 36-40 | 44-48 40-44 |
| e Elbow Fold | 23-25 | 25-27 | 28-30 | 30-32 |
| c¹ 4cm Above Wrist | 15-17 | 17-19.5 | 19.5-22.5 | 22.5-26 |
| a Base of Fingers | 14-18 | 16-21 | 18-24 | 21-27 |
| Length cm | REGULAR | | LONG | |
| c-g Wrist to Axilla | 40-44 | | 44-50 | |

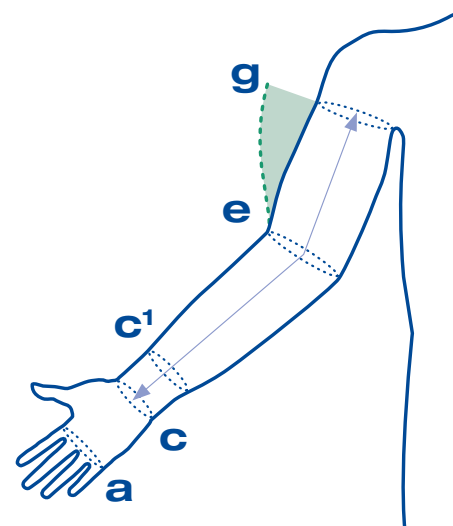

Options

Grip Top



3cm
Strong Plain



5cm
Strong Plain



5cm
Fine Lace



5cm
Strong Lace

Colour



Beige



Black



Navy

Shoulder Cap

Please state g-h measurement
and left/right arm



With Velcro® Bra Attachment

Please state g-h measurement,
g-t circumference and left/right arm



With Adjustable Strap

Microfine

Glove

Material Composition: Polyamide, Elastane

Microfine Gloves are an easy to measure, time-saving solution for oedema that extends to the fingers.

Haddenham Microfine is an ultra-thin and smooth fabric that trims to fit without fraying.

Finger Saving

No need to bandage fingers
Time saving and finger saving



Compression

| Class | mmHg | Well suited to |
|-------|-------|--|
| - | 20-36 | Positive Stemmer sign. Mild to moderate shape distortion and slight to moderate tissue pitting |



Sizing



| Circumferences cm | XXS | XS | S | M | L |
|--------------------------------|-------|-------|-------|-------|-------|
| a Base of Fingers | 14-16 | 16-18 | 18-20 | 20-22 | 22-24 |
| b Palm at Fold of Thumb | 15-17 | 17-19 | 19-21 | 21-23 | 24-26 |

Codes

All variants held **in stock**.

| | XXS | XS | S | M | L |
|--|--------|--------|--------|--------|--------|
|  Sand | 208S-0 | 208S-1 | 208S-2 | 208S-3 | 208S-4 |
|  Black | 208B-0 | 208B-1 | 208B-2 | 208B-3 | 208B-4 |
| Unit of Sale | Single | Single | Single | Single | Single |

Custom Made for Upper Limb

We aim to be flexible wherever possible by providing a made-to-measure service that offers unparalleled customisation and fit.

Bespoke

The widest choice

Select from our widest range of styles, fabric and options

Download the [Custom Made Hosiery](http://www.hadhealth.com.au/F0002a) form
www.hadhealth.com.au/F0002a ▶



Special Options

Padding



Partial Lining

UL-LIN-CM



Pocket

UL-POC-CM

Closures



Zips

UL-ZIP-CM



Velcro Fastenings

UL-VELF-CM



Velcro Straps

UL-VELS-CM

Finishing



Slant Top

UL-ST-CM

Shoulder Cap



Shoulder Cap & Velcro® Bra Attachment

UL-SCBA-CM




Shoulder Cap & Adjustable Strap

UL-SCAS-CM

Custom Made Codes


Pertex Light

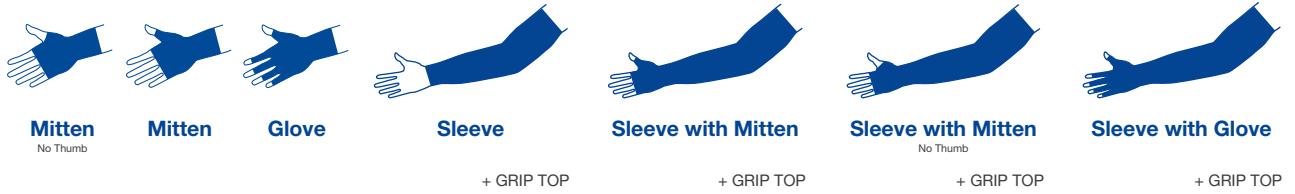

| | | | | | | | | | | | |
|--------------------|-------|-------|-------|-------|-------|-------|-------|-------|-------|-------|-------|
| RAL Class 1 | 150CM | 106CM | 105CM | 143CM | 145CM | 180CM | 181CM | 182CM | 183CM | 184CM | 185CM |
|--------------------|-------|-------|-------|-------|-------|-------|-------|-------|-------|-------|-------|

| | | | | | | | | | | | |
|--------------|--------|--------|--------|--------|--------|--------|--------|--------|--------|--------|--------|
| Unit of Sale | Single | Single | Single | Single | Single | Single | Single | Single | Single | Single | Single |
|--------------|--------|--------|--------|--------|--------|--------|--------|--------|--------|--------|--------|

Pertex

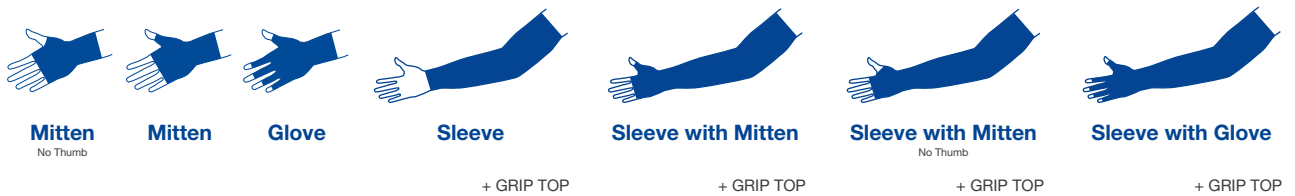


| | | | | | | | | | | | |
|--------------------|-------|-------|-------|-------|-------|-------|-------|-------|-------|-------|-------|
| RAL Class 2 | 250CM | 206CM | 205CM | 333CM | 203CM | 234CM | 204CM | 282CM | 283CM | 284CM | 285CM |
|--------------------|-------|-------|-------|-------|-------|-------|-------|-------|-------|-------|-------|

| | | | | | | | | | | | |
|--------------------|-------|-------|-------|-------|-------|-------|-------|-------|-------|-------|-------|
| RAL Class 3 | 350CM | 306CM | 305CM | 331CM | 311CM | 332CM | 312CM | 382CM | 383CM | 384CM | 385CM |
|--------------------|-------|-------|-------|-------|-------|-------|-------|-------|-------|-------|-------|

| | | | | | | | | | | | |
|--------------|--------|--------|--------|--------|--------|--------|--------|--------|--------|--------|--------|
| Unit of Sale | Single | Single | Single | Single | Single | Single | Single | Single | Single | Single | Single |
|--------------|--------|--------|--------|--------|--------|--------|--------|--------|--------|--------|--------|

Goldpunkt

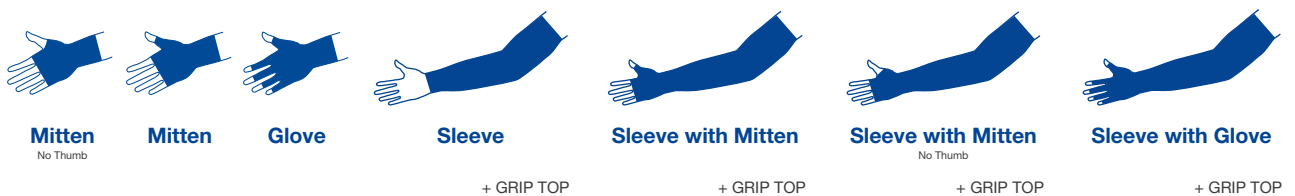

| | | | | | | | | | | | |
|--------------------|-------|-------|-------|-------|-------|-------|-------|-------|-------|-------|-------|
| RAL Class 2 | 251CM | 209CM | 207CM | 315CM | 213CM | 316CM | 214CM | 286CM | 287CM | 288CM | 289CM |
|--------------------|-------|-------|-------|-------|-------|-------|-------|-------|-------|-------|-------|

| | | | | | | | | | | | |
|--------------------|-------|-------|-------|-------|-------|-------|-------|-------|-------|-------|-------|
| RAL Class 3 | 351CM | 309CM | 307CM | 317CM | 313CM | 318CM | 314CM | 386CM | 387CM | 388CM | 389CM |
|--------------------|-------|-------|-------|-------|-------|-------|-------|-------|-------|-------|-------|

| | | | | | | | | | | | |
|--------------|--------|--------|--------|--------|--------|--------|--------|--------|--------|--------|--------|
| Unit of Sale | Single | Single | Single | Single | Single | Single | Single | Single | Single | Single | Single |
|--------------|--------|--------|--------|--------|--------|--------|--------|--------|--------|--------|--------|

Custom Made Codes

Venex



Mitten
No Thumb



Sleeve

+ GRIP TOP



Sleeve with Mitten
No Thumb

+ GRIP TOP

| | | | | | |
|--------------|--------|--------|--------|--------|--------|
| RAL Class 1 | 110CM | 103CM | 101CM | 100CM | 111CM |
| RAL Class 2 | 210CM | 223CM | 221CM | 200CM | 211CM |
| Unit of Sale | Single | Single | Single | Single | Single |

Star Cotton



Mitten
No Thumb



Sleeve

+ GRIP TOP



Sleeve with Mitten
No Thumb

+ GRIP TOP

| | | | | | |
|--------------|--------|--------|--------|--------|--------|
| RAL Class 1 | 170CM | 161CM | 163CM | 160CM | 171CM |
| RAL Class 2 | 270CM | 261CM | 263CM | 260CM | 271CM |
| Unit of Sale | Single | Single | Single | Single | Single |

Doktus



Mitten
No Thumb



Sleeve

+ GRIP TOP



Sleeve with Mitten
No Thumb

+ GRIP TOP

| | | | | | |
|--------------|--------|--------|--------|--------|--------|
| RAL Class 2 | 230CM | 215CM | 201CM | 220CM | 225CM |
| RAL Class 3 | 330CM | 216CM | 301CM | 320CM | 325CM |
| Unit of Sale | Single | Single | Single | Single | Single |



easywrap



easywrap®

An adjustable compression wrap system comprised of single-layer knitted fabric bands which are fastened in an overlapping pattern.



What makes it so patient-friendly?

Turn the page to discover all features [Page 70](#) ▶



Leg
[Page 74](#) ▶



Foot
[Page 76](#) ▶



Thigh
[Page 78](#) ▶



Knee
[Page 80](#) ▶




Liners
[Page 82](#) ▶



Extenders
[Page 83](#) ▶



Hand
[Page 84](#) ▶

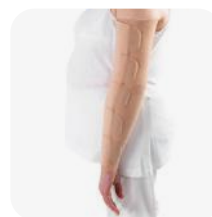

Arm
[Page 86](#) ▶



Additional Bands
[Page 88](#) ▶



Custom Made
[Page 89](#) ▶

Haddenham easywrap is a single layer, compression wrap system comprised of overlapping bands which mimic the therapeutic effect of short-stretch bandages.

A cost-effective alternative to bandaging during the decongestive phase of treatment, when tackling stubborn or rebound oedema, as well as for the treatment of venous leg ulcers.

Compression made easy






Reduce

High working pressures mean that the more active you are, the better your lymphatic system will work to reduce oedema.

Maintain

Low resting pressures and high static-stiffness maintain limb reduction when you are less active.

Two classes of compression

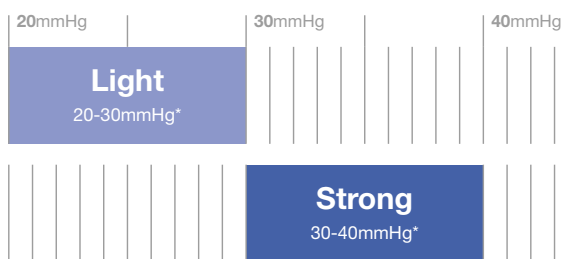

*Compression values delivered within $\pm 15\%$ + 1mmHg

Low profile

Can be worn comfortably under most everyday clothing and footwear.





Easy to apply

A distinctive “lock-out” or end-stretch makes them easy to apply at the correct tension.



Position



Stretch



Fasten

Liners

easywrap for lower limb comes with a pair of protective liners - one to wear and one to wash.

For more details on our liner policy see Liners Page 82 ►

Effective

“Patients who have previously been non concordant with treatment, whether that be due to the psychosocial aspects of bandages or the inability to apply compression garments have all experienced an increase in quality of life when using easywrap.”

- British Journal of Community Nursing





Mobility

Featuring a patented knee section which maintains freedom of movement and enhances everyday comfort.

“ This is very easy to put on, comfortable and really helpful for my lymphoedema.”

Patient review, lymphshop.com

Time-saving

A simple and cost-effective alternative to bandaging that promotes self-care and independence, and reduces waste.

Guaranteed
6 months of use.

Material composition
Polyester, Polyurethane

Availability
All variants held **in stock.**



easywrap

Leg

Material Composition: polyamide, elastane

The easywrap compression wrap system for lower limb comes in three parts, which work in isolation or combination.

easywrap Leg is comprised of short-stretch bands. Each band overlaps by 50% to ensure therapeutic, graduated compression and can be adjusted as swelling fluctuates.

“The most comfy wrap I have ever worn.”


Patient review, lymphshop.com



Compression

| Class | mmHg | Well suited for | LYMPHATIC | VENOUS | LIPOEDEMA |
|--------|-------|--|-----------|---|--|
| Light | 20-30 | Mild oedema, Fragile skin, palliative care | | Mixed aetiology ulcers. Telangiectasia, thread, spider or broken veins. Pigmentation or eczema, Lipodermatosclerosis or atrophie blanche. Corona phlebectatica. | Smooth or uneven skin with an increase of enlarged subcutaneous fat tissue or lipomas. |
| Strong | 30-40 | Moderate oedema with moderate shape distortion and deep pitting tissues. Post liposuction for lymphoedema. | | Pigmentation or eczema, Lipodermatosclerosis or atrophie blanche. Corona phlebectatica. Healed Venous Ulcer. | Large deformations of fat, lipolymphoedema, large fat nodules and lipomas. |

Sizing

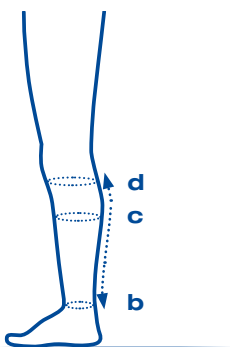

| Circumferences cm | XS | S | M | L | XL |
|--------------------------------|---------|-------|-------|-------|-------|
| c Widest point of calf | 38-45 | 44-52 | 50-60 | 55-65 | 60-70 |
| b Ankle | 23-27 | 28-32 | 33-38 | 39-44 | 45-52 |
| Contoured Lengths cm | REGULAR | | TALL | | |
| b-d Ankle to below knee | 30-34 | | 34-38 | | |

Easy to wear

Low profile

Can be worn under everyday clothing, even jeans

Velcro® Fasteners

For a strong hold and an effortless release

Adaptive

Highest static stiffness in class*

Stops oedema rebounding after reduction

High working pressure

for improved venous return



*Phillips, N. and Wright, T (2024) 'Adjustable Compression Wraps, Background, evidence and case studies on use in lymphatic and venous disease' Journal of Woundcare, 15 July.

Codes

Light 20-30mmHg

| COLOUR | LENGTH | XS | S | M | L | XL |
|--------|---------|---------|---------|---------|---------|---------|
| SAND | REGULAR | EL-LR1 | EL-LR2 | EL-LR3 | EL-LR4 | EL-LR5 |
| | TALL | EL-LT1 | EL-LT2 | EL-LT3 | EL-LT4 | EL-LT5 |
| BLACK | REGULAR | ELB-LR1 | ELB-LR2 | ELB-LR3 | ELB-LR4 | ELB-LR5 |
| | TALL | ELB-LT1 | ELB-LT2 | ELB-LT3 | ELB-LT4 | ELB-LT5 |

Strong 30-40mmHg

| COLOUR | LENGTH | XS | S | M | L | XL |
|--------|---------|---------|---------|---------|---------|---------|
| SAND | REGULAR | ES-LR1 | ES-LR2 | ES-LR3 | ES-LR4 | ES-LR5 |
| | TALL | ES-LT1 | ES-LT2 | ES-LT3 | ES-LT4 | ES-LT5 |
| BLACK | REGULAR | ESB-LR1 | ESB-LR2 | ESB-LR3 | ESB-LR4 | ESB-LR5 |
| | TALL | ESB-LT1 | ESB-LT2 | ESB-LT3 | ESB-LT4 | ESB-LT5 |

easywrap

Foot

Material Composition: polyamide, elastane

The easywrap compression wrap system for lower limb comes in three parts, which work in isolation or combination, and are designed to be worn over a liner.

The easywrap Foot consists of a set of short-stretch bands extending from a central spine at the back of the garment.

“ This is very easy to put on, comfortable and really helpful for my lymphoedema.”

Patient review, lymphshop.com




Compression

| Class | mmHg | Well suited for | LYMPHATIC | VENOUS | LIPOEDEMA |
|--------|-------|--|-----------|--|---|
| Light | 20-30 | Mild oedema of the foot, Fragile skin, palliative care. | | Mixed aetiology ulcers. Telangiectasia, thread, spider or broken veins. Recurrent/ Varicose veins. Pigmentation or eczema, Lipodermatosclerosis or atrophie blanche. Corona phlebectatica. | Feet are usually spared. |
| Strong | 30-40 | Moderate oedema of the foot with moderate shape distortion and deep pitting tissues. Post liposuction for lymphoedema. | | Recurrent/Active venous Leg Ulcer. Healed venous ulcer. | Lipolymphoedema, large fat nodules and lipomas. |

Sizing

| Circumferences cm | XS | S | M | L | XL | |
|---------------------------------------|----------------|-----------|-------|-----------|-------|-------|
| a¹ Mid-foot | 22-24 | 25-27 | 28-30 | 31-34 | 35-40 | |
| a-heel Base of toes to heel | REGULAR | 14.5-16.5 | 16-18 | 17.5-19.5 | 19-21 | 21-23 |
| | LONG | 16.5-18.5 | 18-20 | 19.5-21.5 | 21-23 | 23-25 |

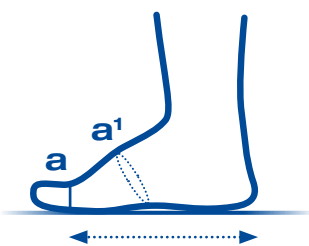

Easy to wear

Low profile

Fits into most shoes

Single posterior layer

For maximum comfort and smooth contouring

Versatile

The perfect partner

Pair with an easywrap Leg and/ or a Microfine Toecap for graduated compression from toes to knee.



Codes

Light 20-30mmHg

| COLOUR | LENGTH | XS | S | M | L | XL |
|--------|---------|---------|---------|---------|---------|---------|
| SAND | REGULAR | EL-FR1 | EL-FR2 | EL-FR3 | EL-FR4 | EL-FR5 |
| | TALL | EL-FL1 | EL-FL2 | EL-FL3 | EL-FL4 | EL-FL5 |
| BLACK | REGULAR | ELB-FR1 | ELB-FR2 | ELB-FR3 | ELB-FR4 | ELB-FR5 |
| | TALL | ELB-FL1 | ELB-FL2 | ELB-FL3 | ELB-FL4 | ELB-FL5 |

Strong 30-40mmHg

| COLOUR | LENGTH | XS | S | M | L | XL |
|--------|---------|---------|---------|---------|---------|---------|
| SAND | REGULAR | ES-FR1 | ES-FR2 | ES-FR3 | ES-FR4 | ES-FR5 |
| | TALL | ES-FL1 | ES-FL2 | ES-FL3 | ES-FL4 | ES-FL5 |
| BLACK | REGULAR | ESB-FR1 | ESB-FR2 | ESB-FR3 | ESB-FR4 | ESB-FR5 |
| | TALL | ESB-FL1 | ESB-FL2 | ESB-FL3 | ESB-FL4 | ESB-FL5 |

easywrap

Thigh with Knee

Material Composition: polyamide, elastane

The easywrap compression wrap system for lower limb comes in three parts, which work in isolation or combination, and are designed to be worn over a liner.


The easywrap Thigh with Knee is a two part solution for the upper leg.



Compression

| Class | mmHg | Well suited for | LYMPHATIC | VENOUS | LIPOEDEMA |
|--------|-------|--|-----------|---|--|
| Light | 20-30 | Mild oedema extending into the thigh, Fragile skin, palliative care. | | Telangiectasia, thread, spider or broken veins. | Smooth or uneven skin with an increase of enlarged subcutaneous fat tissue or lipomas. |
| Strong | 30-40 | Moderate oedema extending into the thigh, moderate shape distortion, deep pitting tissues. Post liposuction for lymphoedema. | | Recurrent/ Varicose veins. | Large deformations of fat especially on the thighs and knees. Lipo-lymphoedema. |

Sizing

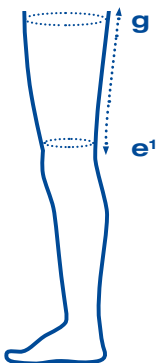

| Circumferences cm | XS | S | M | L | XL |
|--------------------------------------|-------|---------|-------|-------|--------|
| g Just below the groin | 60-70 | 65-75 | 70-80 | 75-90 | 85-100 |
| e¹ Bottom of the thigh | 47-52 | 53-59 | 60-66 | 67-73 | 74-80 |
| Contoured Lengths cm | SHORT | REGULAR | TALL | | |
| e¹-g Bottom of thigh to groin | 16-20 | 20-24 | 25-30 | | |

Comfortable

Patented zone behind knee

Padding and elliptical cut prevent pinching or cutting in, enhancing comfort and mobility



Wide bands

For easy donning



Codes

Light 20-30mmHg

| COLOUR | LENGTH | XS | S | M | L | XL |
|--|---------|---------|---------|---------|---------|---------|
|  SAND | SHORT | EL-TS1 | EL-TS2 | EL-TS3 | EL-TS4 | EL-TS5 |
| | REGULAR | EL-TR1 | EL-TR2 | EL-TR3 | EL-TR4 | EL-TR5 |
| | TALL | EL-TT1 | EL-TT2 | EL-TT3 | EL-TT4 | EL-TT5 |
|  BLACK | SHORT | ELB-TS1 | ELB-TS2 | ELB-TS3 | ELB-TS4 | ELB-TS5 |
| | REGULAR | ELB-TR1 | ELB-TR2 | ELB-TR3 | ELB-TR4 | ELB-TR5 |
| | TALL | ELB-TT1 | ELB-TT2 | ELB-TT3 | ELB-TT4 | ELB-TT5 |

Strong 30-40mmHg

| COLOUR | LENGTH | XS | S | M | L | XL |
|--|---------|---------|---------|---------|---------|---------|
|  SAND | SHORT | ES-TS1 | ES-TS2 | ES-TS3 | ES-TS4 | ES-TS5 |
| | REGULAR | ES-TR1 | ES-TR2 | ES-TR3 | ES-TR4 | ES-TR5 |
| | TALL | ES-TT1 | ES-TT2 | ES-TT3 | ES-TT4 | ES-TT5 |
|  BLACK | SHORT | ESB-TS1 | ESB-TS2 | ESB-TS3 | ESB-TS4 | ESB-TS5 |
| | REGULAR | ESB-TR1 | ESB-TR2 | ESB-TR3 | ESB-TR4 | ESB-TR5 |
| | TALL | ESB-TT1 | ESB-TT2 | ESB-TT3 | ESB-TT4 | ESB-TT5 |

easywrap

Knee

Material Composition: polyamide, elastane

The easywrap compression wrap system for lower limb comes in three parts, which work in isolation or combination, and are designed to be worn over a liner.


The easywrap Knee is a one part solution for the knee.



Compression

| Class | mmHg | Well suited for | LYMPHATIC | VENOUS | LIPOEDEMA |
|--------|-------|--|-----------|---|--|
| Light | 20-30 | Mild oedema extending into the thigh, Fragile skin, palliative care. | | Telangiectasia, thread, spider or broken veins. | Smooth or uneven skin with an increase of enlarged subcutaneous fat tissue or lipomas. |
| Strong | 30-40 | Moderate oedema extending into the thigh, moderate shape distortion, deep pitting tissues. Post liposuction for lymphoedema. | | Recurrent/ Varicose veins. | Large deformations of fat especially on the thighs and knees. Lipo-lymphoedema. |

Sizing

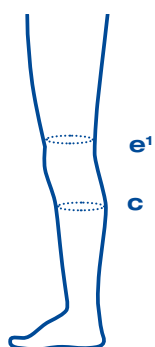

| Circumferences cm | XS | S | M | L | XL |
|-------------------------------|-------|-------|-------|-------|-------|
| e¹ Bottom of the thigh | 47-52 | 53-59 | 60-66 | 67-73 | 74-80 |
| c Widest calf | 38-45 | 44-52 | 50-60 | 55-65 | 60-70 |

Easy to wear

Wide bands

For easy donning

Comfortable



Patented zone behind knee

Padding and elliptical cut prevent pinching or cutting in, enhancing comfort and mobility





Codes

Light 20-30mmHg

| COLOUR | XS | S | M | L | XL |
|---|---------|---------|---------|---------|---------|
|  SAND | EL-KP1 | EL-KP2 | EL-KP3 | EL-KP4 | EL-KP5 |
|  BLACK | ELB-KP1 | ELB-KP2 | ELB-KP3 | ELB-KP4 | ELB-KP5 |

Strong 30-40mmHg

| COLOUR | XS | S | M | L | XL |
|---|---------|---------|---------|---------|---------|
|  SAND | ES-KP1 | ES-KP2 | ES-KP3 | ES-KP4 | ES-KP5 |
|  BLACK | ESB-KP1 | ESB-KP2 | ESB-KP3 | ESB-KP4 | ESB-KP5 |

easywrap

Liners

Material Composition: polyamide, elastane

Forms a barrier between the limb and easywrap which absorbs sweat and traps odours, reducing the need to wash the easywrap.

easywrap Standard liners are made of a non-compressive, stretchy, soft knit for maximum comfort and skin protection.

“Soft and comfortable against my sensitive skin. They wash well and dry quickly, good quality liners.”

Patient review, lymphshop.com



Codes

| COLOUR | LEG SIZE |  |  |
|--|---------------|---|---|
|  SAND | XS/S/M | E-L-AD | E-L-AG |
| | L/XL | E-L-ADW | E-L-AGW |
|  BLACK | XS/S/M | EB-L-AD | EB-L-AG |
| | L/XL | EB-L-ADW | EB-L-AGW |

*One pair of Below Knee Liners will be included free of charge when purchasing an easywrap Foot and Leg together. Likewise one pair of Thigh High liners are included free of charge when purchasing an easywrap Foot, Leg and Thigh together.

Extenders

Extends the circumference of a single band of easywrap by 10cm. When oedema in this area is reduced, the extender can then be easily removed.

Useful where patients measure outside the standard circumference size of a single measurement point, reducing the need for ordering an entire custom made garment.



Codes

| COLOUR | LENGTH | WIDTH | | |
|---|--------|----------|----------|----------|
| | | 8cm | 12cm | 16cm |
|  SAND | 10cm | EE-0810 | EE-1210 | EE-1610 |
|  BLACK | 10cm | EEB-0810 | EEB-1210 | EEB-1610 |

easywrap

Hand

Material Composition: polyamide, elastane

The easywrap compression wrap system for upper limb comes in two parts, which work in isolation or in combination.

easywrap Hand features two self-adjustable Velcro® tabs, which fasten around a silky smooth mitten. Can also be successfully paired with a *Microfine Glove* to add compression to fingers.



Compression

| Class | mmHg | Well suited for |
|-------|-------|---|
| Light | 20-30 | LYMPHATIC Mild to moderate shape distortion and slight to moderate tissue pitting. Post liposuction. |

Sizing



Circumferences cm

| | XS | S | M | L |
|----------------------------------|-----|-------|-------|-----|
| b Palm at fold of thumb | <19 | 19-21 | 21-23 | >24 |
| a Palm at base of fingers | <18 | 18-20 | 20-22 | >22 |

Easy to wear

Velcro® fasteners

Lengthened wrist section to ensure overlap with Easywrap Arm

4-way stretch soft fabric

for maximum comfort whilst maintaining range of motion.




Codes

Left 20-30mmHg

| COLOUR | XS | S | M | L |
|---|---------|---------|---------|---------|
|  SAND | EL-H1L | EL-H2L | EL-H3L | EL-H4L |
|  BLACK | ELB-H1L | ELB-H2L | ELB-H3L | ELB-H4L |

Right 20-30mmHg

| COLOUR | XS | S | M | L |
|---|---------|---------|---------|---------|
|  SAND | EL-H1R | EL-H2R | EL-H3R | EL-H4R |
|  BLACK | ELB-H1R | ELB-H2R | ELB-H3R | ELB-H4R |

easywrap

Arm

Material Composition: polyamide, elastane

The easywrap compression wrap system for upper limb comes in two parts, which work in isolation or combination.

easywrap Arm features self-adjustable Velcro® tabs, which fasten around a silky smooth sleeve.




Compression

Class **mmHg** **Well suited for**

Light 20-30 Mild to moderate shape distortion and slight to moderate tissue pitting. Post liposuction.

LIPOEDEMA
Smooth or uneven skin with an increase of enlarged subcutaneous fat tissue or lipomas.

Sizing

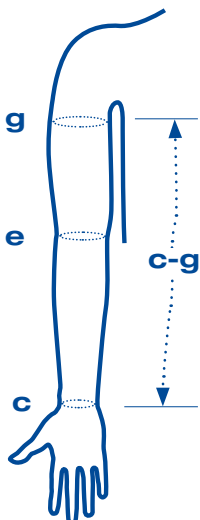

| Circumferences cm | S | M | L | XL |
|----------------------------|-------|---------|-------|-------|
| g Axilla | 22-31 | 29-39 | 32-45 | 36-50 |
| e Elbow Crease | 20-27 | 25-34 | 30-40 | 32-43 |
| c Wrist Crease | 14-18 | 16-21 | 19-25 | 19-25 |
| Contoured Lengths cm | SHORT | REGULAR | LONG | |
| c-g Axilla to wrist | 40-44 | 44-48 | 48-52 | |

Easy to wear

Velcro® fasteners

Can be applied and adjusted with one hand

4-way stretch soft fabric

for maximum comfort whilst maintaining range of motion.



Codes

Left 20-30mmHg

| COLOUR | LENGTH | S | M | L | XL |
|--------|---------|----------|----------|----------|----------|
| SAND | SHORT | EL-AS2L | EL-AS3L | EL-AS4L | EL-AS5L |
| | REGULAR | EL-AR2L | EL-AR3L | EL-AR4L | EL-AR5L |
| | LONG | EL-AL2L | EL-AL3L | EL-AL4L | EL-AL5L |
| BLACK | SHORT | ELB-AS2L | ELB-AS3L | ELB-AS4L | ELB-AS5L |
| | REGULAR | ELB-AR2L | ELB-AR3L | ELB-AR4L | ELB-AR5L |
| | LONG | ELB-AL2L | ELB-AL3L | ELB-AL4L | ELB-AL5L |

Right 20-30mmHg

| COLOUR | LENGTH | S | M | L | XL |
|--------|---------|----------|----------|----------|----------|
| SAND | SHORT | EL-AS2R | EL-AS3R | EL-AS4R | EL-AS5R |
| | REGULAR | EL-AR2R | EL-AR3R | EL-AR4R | EL-AR5R |
| | LONG | EL-AL2R | EL-AL3R | EL-AL4R | EL-AL5R |
| BLACK | SHORT | ELB-AS2R | ELB-AS3R | ELB-AS4R | ELB-AS5R |
| | REGULAR | ELB-AR2R | ELB-AR3R | ELB-AR4R | ELB-AR5R |
| | LONG | ELB-AL2R | ELB-AL3R | ELB-AL4R | ELB-AL5R |

Additional Bands

Material Composition: polyamide, elastane

A single band of easywrap lower limb material with Velcro® fasteners attached at either end.

They can be used to provide additional support and compression to either the upper or lower limb.



Compression



| Class | mmHg | Well suited for |
|--------|-------|---|
| Light | 20-30 | Mild to moderate shape distortion, moderate pitting. |
| Strong | 30-40 | Moderate shape distortion, massive localised lymphoedema, obesity related lymphoedema, moderate skin changes. |

Codes



Light 20-30mmHg

| | 8 x 40cm | 8 x 60cm |
|--|----------|----------|
|  SAND | EL-0840 | EL-0860 |
|  BLACK | ELB-0840 | ELB-0860 |
| Unit of Sale | Single | Single |



16 x 45cm 16 x 150cm

| | 16 x 45cm | 16 x 150cm |
|--|-----------|------------|
|  SAND | EL-1645 | EL-16150 |
|  BLACK | ELB-1645 | ELB-16150 |
| Unit of Sale | Single | Single |

Strong 30-40mmHg

| | 8 x 40cm | 8 x 60cm |
|--|----------|----------|
|  SAND | ES-0840 | ES-0860 |
|  BLACK | ESB-0840 | ESB-0860 |
| Unit of Sale | Single | Single |

16 x 45cm 16 x 150cm

| | 16 x 45cm | 16 x 150cm |
|--|-----------|------------|
|  SAND | ES-1645 | ES-16150 |
|  BLACK | ESB-1645 | ESB-16150 |
| Unit of Sale | Single | Single |

easywrap

Custom Made

Material Composition: polyamide, elastane

For patients who fall outside of the sizing chart ranges.

Easy-to-use, step-by-step measurement form that guides you through the process of ordering a custom made easywrap.

We aim to provide additional flexibility wherever possible. Please call or email us to discuss.

Codes

| | Foot | Leg | Thigh | Knee | Arm | Hand |
|-------------------------|--------|--------|--------|--------|--------|--------|
| Light 20-30mmHg | EL-FC | EL-LC | EL-TC | EL-KC | EL-AC | EL-HC |
| Strong 30-40mmHg | ES-FC | ES-LC | ES-TC | ES-KC | - | - |
| Unit of Sale | Single | Single | Single | Single | Single | Single |





Comfiwave



Comfiwave

A soft compressive fabric, with a distinctive wave pattern knit. The wave profile creates a pressure differential, that leaves grooves on a patient's skin that help soften the skin while promoting lymphatic drainage.



What makes it so patient-friendly?

Turn the page to discover all features [Page 92](#) ▶



Below Knee

[Page 96](#) ▶



Thigh

[Page 98](#) ▶



Toecap

[Page 100](#) ▶



Shorts

[Page 102](#) ▶



Abdominal Band

[Page 104](#) ▶



Sleeve

[Page 106](#) ▶



Sleeve + Mitten

[Page 108](#) ▶



Glove

[Page 110](#) ▶



Glove to Elbow

[Page 112](#) ▶



Glove to Axilla

[Page 114](#) ▶



Breast Band

[Page 116](#) ▶



Neck

[Page 118](#) ▶



Thigh + Hip

[Page 120](#) ▶



Kids

[Page 122](#) ▶

Comfiwave is a soft compressive fabric, with a distinctive wave pattern knit. The wave profile creates a pressure differential, that leaves lines/grooves on a patient's skin that help soften the tissues while promoting lymphatic drainage.

Comfiwave offers a unique additional option for patients to use in their self management regimen, which can be worn in a more relaxed setting at home and even overnight.

Compression to drift off in



Innovative

Comfiwave garments use proprietary 'low-resistance' yarns which allow them to be effortlessly donned and doffed.

Conforming

3D knitting technology creates a seamless knit which delivers a gentle yet targeted, therapeutic pressure of 15-21mmHg.



Breathable

High air permeability for coolness all year round, day or night.

Organic cotton

Soft, organic cotton against the skin ensures maximum comfort.





For night time

Continue compression therapy overnight in supreme comfort with Comfiwave's fabric providing gentle compression while you sleep.

Making waves

Comfiwave's wave patterning mimics the pressure differential massage effect that is ordinarily found in cut-foam, chip-bag style garments.

Effective

“Can be used for stand-alone treatment or as an adjunct to other treatment modalities, and for 24 hours a day if needed.”

- British Journal of Community Nursing

“Patients reported that Comfiwave was comfortable and easy to use.”

| Circumference measurements at baseline and 2 weeks after using Comfiwave | | | | | | |
|--|------|------|---------------|------------|--------------|--|
| D | F | G | Limb volume A | Unaffected | % difference | |
| 25 | 29 | 31.5 | 2329 | 1954 | +18% | |
| 24 | 27.5 | 31 | 2139 | 1953 | +8% | |

through her teenage years with bilateral lower leg oedema and recurrent episodes of cellulitis.

On initial assessment, there was evidence of bilateral lymphoedema below the knee, with both feet showing positive Stemmer's sign, mild hyperkeratosis and post-cellulitis skin discoloration on the left gaiter area. The left leg was 30% bigger than the right, with an ankle circumference of 51cm and calf of 65cm. There were distinct skin folds distorting the shape around her left ankle, as well as foot oedema, which was restricting footwear (Figure 4). The limb size also restricted clothing, and the patient was only able to wear very loose trousers or long skirts.

Initial advice of skin care, exercise, elevation and weight management were discussed, and 2 weeks of daily treatment were booked. Miss H was signed off work by her GP to enable her to attend the sessions. Daily treatment commenced in October 2018 and involved alternating days of MLD or intermittent compression using Haddenham

Figure 5. Miss H wearing Comfiwave on the left leg

Figure 6. Miss H's legs after using Comfiwave for 2 weeks

PRODUCT



ments
to the
y were
improve
ment of
that this
ent or as
24 hours
efficacy in
decreasing
edema and
ient, family

ent: a prospective
3-541. <https://doi.org/10.1016/j.ccn.2017.09.001>

ntaker J. Examination
in tissue oxygenation
429-435. <https://doi.org/10.1016/j.ccn.2017.09.001>

© IsoVida PtePaks in a



Simple sizing

Easy measuring with colour coded sizing system that eliminates the need for uncomfortable sewn-in labels. 3 red dots = Medium Regular.

“ I love this garment, I have been wearing it overnight for the last week and it's making a massive difference!

- Patient review, lymphshop.com



Kid-friendly

A one-of-a-kind addition to treatment options for children with lymphoedema.

Guaranteed

6 months of use

Material composition

Cotton, Polyester, Polyurethane

Availability

All variants held **in stock**

Comfiwave

Below Knee

Material Composition: Cotton, Polyester, Polyurethane

A soft compressive fabric, with a distinctive wave pattern knit. The wave profile creates a pressure differential, that leaves lines/grooves on a patient's skin that help soften the tissues while promoting lymphatic drainage.


Comfiwave offers a unique additional option for patients to use in their self management regimen, which can be worn in a more relaxed setting at home and even overnight.



Compression

| mmHg | Well suited for | | |
|-------|---|---|--|
| 15-21 | Mild to moderate oedema with or without pitting and/or fibrosis. Palliative care. | LYMPHOEDEMA | VENOUS DISEASE |
| | | Telangiectasia, thread, spider or broken veins. Recurrent/Varicose veins. Pigmentation or eczema, Lipodermatosclerosis or atrophie blanche. Corona phlebectatica. | LIPOEDEMA |
| | | | Smooth or uneven skin with an increase of enlarged subcutaneous fat tissue or lipomas. |

Sizing

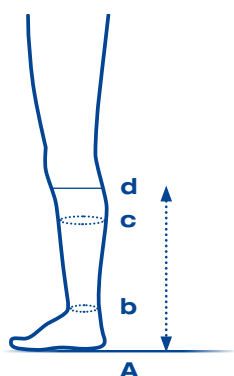

| Circumferences cm | S | M | L | XL |
|--------------------------------|---------|---------|---------|---------|
| c Widest Calf | 25 - 32 | 30 - 38 | 36 - 46 | 44 - 56 |
| b Ankle | 17 - 22 | 20 - 26 | 22 - 30 | 27 - 35 |
| Lengths cm | SHORT | REGULAR | LONG | |
| A-d Floor to Below Knee | 32 - 37 | 37 - 42 | 42 - 48 | |

Comfort

Stretchy lateral toe zone

For enhanced fit at ball of foot

Shaped ankle zone

Increases comfort and prevents creasing

Support

Enhanced instep compression

Added support around the top of the foot

Contoured heel section

Helps keep the garment in place

“This has been a real game changer for me. My leg has never been as good.”

Patient review, lymphshop.com



Codes

| | S | M | L | XL |
|---------|----------|----------|----------|----------|
| SHORT | CW1-AD2S | CW1-AD3S | CW1-AD4S | CW1-AD5S |
| REGULAR | CW1-AD2R | CW1-AD3R | CW1-AD4R | CW1-AD5R |
| LONG | CW1-AD2L | CW1-AD3L | CW1-AD4L | CW1-AD5L |

Unit of Sale

Single

Comfiwave

Thigh High

Material Composition: Cotton, Polyester, Polyurethane

A soft compressive fabric, with a distinctive wave pattern knit. The wave profile creates a pressure differential, that leaves lines/grooves on a patient's skin that help soften the tissues while promoting lymphatic drainage.


Comfiwave offers a unique additional option for patients to use in their self management regimen, which can be worn in a more relaxed setting at home and even overnight.



Compression

| mmHg | Well suited for | VENOUS DISEASE | LIPOEDEMA |
|-------|--|---|--|
| 15-21 | LYMPHOEDEMA Mild to moderate oedema with or without pitting and/or fibrosis. Palliative care. | Telangiectasia, thread, spider or broken veins. Recurrent/Varicose veins. Pigmentation or eczema, Lipodermatosclerosis or atrophie blanche. Corona phlebectatica. | Smooth or uneven skin with an increase of enlarged subcutaneous fat tissue or lipomas. |

Sizing

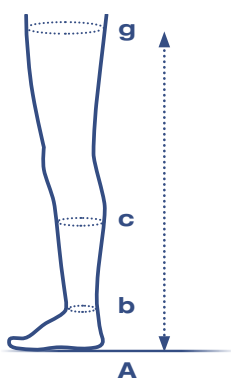

| Circumferences cm | S | M | L | XL | XXL |
|-------------------------------|---------|---------|---------|---------|---------|
| g Thigh Top | 40 - 48 | 46 - 56 | 54 - 65 | 62 - 75 | 72 - 92 |
| c Widest Calf | 25 - 32 | 30 - 38 | 36 - 46 | 44 - 56 | 52 - 64 |
| b Ankle | 17 - 22 | 20 - 26 | 22 - 30 | 27 - 35 | 27 - 35 |
| Lengths cm | SHORT | REGULAR | LONG | | |
| A-g Floor to Thigh Top | 64 - 70 | 70 - 77 | 77 - 84 | | |

Comfort

Stretchy lateral toe zone

For enhanced fit at ball of foot

Shaped ankle zone

Increases comfort and prevents creasing

Support

Enhanced instep compression

Added support around the top of the foot

Contoured heel section

Helps keep the garment in place

“Really comfortable to wear at night. Unlike other products, doesn't make my leg hot when temperature is high.”

Patient review, lymphshop.com



Codes

| | S | M | L | XL | XXL |
|---------|----------|----------|----------|----------|----------|
| SHORT | CW1-AG2S | CW1-AG3S | CW1-AG4S | CW1-AG5S | CW1-AG6S |
| REGULAR | CW1-AG2R | CW1-AG3R | CW1-AG4R | CW1-AG5R | CW1-AG6R |
| LONG | CW1-AG2L | CW1-AG3L | CW1-AG4L | CW1-AG5L | CW1-AG6L |

Unit of Sale

Single

Comfiwave

Toecap

Material Composition: Cotton, Polyester, Polyurethane

A soft compressive fabric, with a distinctive wave pattern knit. The wave profile creates a pressure differential, that leaves lines/grooves on a patient's skin that help soften the tissues while promoting lymphatic drainage.

Comfiwave offers a unique additional option for patients to use in their self management regimen, which can be worn in a more relaxed setting at home and even overnight.



Compression

| mmHg | Well suited for |
|-------|---|
| 15-21 | Toe swelling, base of toe swelling, positive Stemmer sign |

Sizing



Circumferences cm

a Ball of Foot

M

20 - 27

L

26 - 34

Comfort

3D knitting of the forefoot

Creates anatomical shaping for a better fit

Wider circumference at small toe

Creates an expanded 'pinky toe' opening to increase comfort

Support

Sturdy knit

Delivers a firm and supportive fit



“ This is an amazing garment. My toes go from fat bloated sausages to toe-like toes after I have been wearing it, and it fits really well.”

Patient review, lymphshop.com

Codes

| | | | | |
|----------------|----------|--------|----------|---|
| | M | | L | |
| REGULAR | CW1-ZA3 | ⋮ | CW1-ZA4 | ⋮ |
| Unit of Sale | | Single | | |

Comfiwave

Shorts

Material Composition: Cotton, Polyester, Polyurethane

A soft compressive fabric, with a distinctive wave pattern knit. The wave profile creates a pressure differential, that leaves lines/grooves on a patient's skin that help soften the tissues while promoting lymphatic drainage.

Comfiwave offers a unique additional option for patients to use in their self management regimen, which can be worn in a more relaxed setting at home and even overnight.



Compression


mmHg

Well suited for

15-21

Female genital oedema, male genital oedema, suprapubic swelling, hip/buttock swelling, obesity related lymphoedema

Sizing

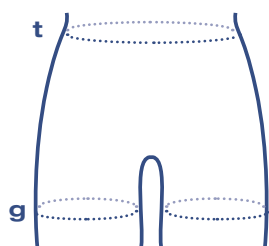

Circumferences cm

S

M

L

XL

XXL

| | S | M | L | XL | XXL |
|--------------------|---------|---------|----------|-----------|-----------|
| t WAIST | 50 - 68 | 66 - 86 | 84 - 104 | 102 - 124 | 120 - 146 |
| g THIGH TOP | 40 - 48 | 46 - 56 | 54 - 65 | 62 - 75 | 72 - 92 |

Comfort

3D Knitting

Conforming, seamless knit to target sensitive areas

Breathable cotton

Promotes airflow and reduces moisture and infection risk

Versatility

Pad and pull-up friendly

Compatible with incontinence products

Unisex design

Conforms for men or women



“ They felt amazing as soon as I put them on...so comfortable, soft and have the perfect amount of squeeze!”

Codes

| | | | | | | | | | | |
|----------------|----------|---|----------|---|----------|---|-----------|---|------------|---|
| | S | | M | | L | | XL | | XXL | |
| REGULAR | CW1-GT2 | ⋮ | CW1-GT3 | ⋮ | CW1-GT4 | ⋮ | CW1-GT5 | ⋮ | CW1-GT6 | ⋮ |
| Unit of Sale | | | | | Single | | | | | |

Comfiwave

Abdominal Band

Material Composition: Cotton, Polyester, Polyurethane

A soft compressive fabric, with a distinctive wave pattern knit. The wave profile creates a pressure differential, that leaves lines/grooves on a patient's skin that help soften the tissues while promoting lymphatic drainage.


Comfiwave offers a unique additional option for patients to use in their self management regimen, which can be worn in a more relaxed setting at home and even overnight.



Compression

| mmHg | Well suited for |
|-------|---|
| 15-21 | LYMPHOEDEMA Male or female suprapubic oedema. Obesity related lymphoedema. |
| | LIPOEDEMA Lipoedema of the pelvis, hips and buttocks. |

Sizing

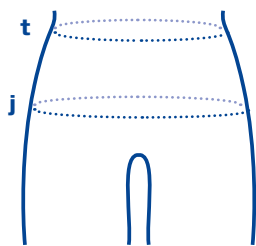

| Circumferences cm | M | L | XL |
|-------------------|----------|----------|-----------|
| t Waist | 60 - 75 | 70 - 85 | 90 - 120 |
| j Hips | 80 - 100 | 90 - 105 | 110 - 140 |

Comfort

3D mapped compression

Conforms to body shape where traditional compression garments can't.

Versatility

Use in combination

Can be used with the Comfiwave Thigh or Thigh+Hip garment.

Use post-operatively

Can be used over dressings for covering wounds, including fungating lesions.

“The first garment that offers help in this difficult to treat area. Very comfortable.”

Male lymphoedema patient, with secondary lymphoedema in the pubic area and upper thighs on both legs.



Codes

| | | | | | | |
|----------------|----------|---|----------|---|-----------|---|
| | M | | L | | XL | |
| REGULAR | CW1-HT3 | ■ | CW1-HT4 | ■ | CW1-HT5 | ■ |
| Unit of Sale | | | Single | | | |

Comfiwave

Sleeve

Material Composition: Cotton, Polyester, Polyurethane

A soft compressive fabric, with a distinctive wave pattern knit. The wave profile creates a pressure differential, that leaves lines/grooves on a patient's skin that help soften the tissues while promoting lymphatic drainage.


Comfiwave offers a unique additional option for patients to use in their self management regimen, which can be worn in a more relaxed setting at home and even overnight.



Compression

| mmHg | Well suited for |
|-------|--|
| 15-21 | <p>LYMPHOEDEMA</p> <p>Oedema which does not manifest in the hand, pitting and fibrosis to the forearm, palliative care.</p> <p>LIPOEDEMA</p> <p>Smooth or uneven skin with an increase of enlarged subcutaneous fat tissue or lipomas.</p> |

Sizing

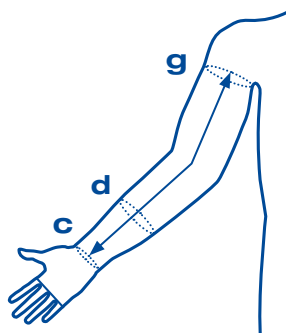

| Circumferences cm | | S | M | L |
|-------------------|-----------------------------|---------|---------|---------|
| g | Axilla | 25 - 30 | 30 - 40 | 40 - 50 |
| d | Mid-point of forearm | 23 - 26 | 26 - 30 | 30 - 38 |
| c | Wrist | 13 - 18 | 18 - 22 | 22 - 28 |
| Lengths cm | | SHORT | REGULAR | LONG |
| c-g | Wrist to Axilla | 36 - 42 | 40 - 46 | 44 - 50 |

Comfort

Elbow stretch zone

Pressure is evenly distributed to prevent tightness when the elbow bends.

Conforms to forearm and elbow

Prevents oedema from pooling in these difficult areas

Versatility

Can be used with Comfiwave Glove

Don and doff the glove as required without having to remove compression from your arm



Codes

| | S | M | L |
|---------|----------|----------|----------|
| SHORT | CW1-CG2S | CW1-CG3S | CW1-CG4S |
| REGULAR | CW1-CG2R | CW1-CG3R | CW1-CG4R |
| LONG | CW1-CG2L | CW1-CG3L | CW1-CG4L |

Unit of Sale

Single

Comfiwave

Sleeve with Mitten

Material Composition: Cotton, Polyester, Polyurethane

A soft compressive fabric, with a distinctive wave pattern knit. The wave profile creates a pressure differential, that leaves lines/grooves on a patient's skin that help soften the tissues while promoting lymphatic drainage.

Comfiwave offers a unique additional option for patients to use in their self management regimen, which can be worn in a more relaxed setting at home and even overnight.




Compression

mmHg Well suited for

15-21 Oedema extending from the dorsum of the hand throughout the arm, pitting and fibrosis with mild shape distortion, palliative care

Sizing

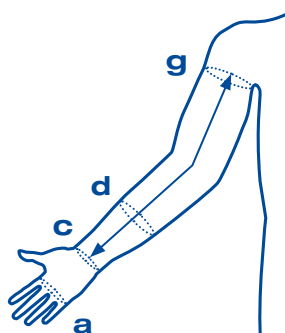

| Circumferences cm | S | M | L |
|---------------------------------|---------|---------|---------|
| g Axilla | 25 - 30 | 30 - 40 | 40 - 50 |
| d Mid-point of forearm | 23 - 26 | 26 - 30 | 30 - 38 |
| c Wrist | 13 - 18 | 18 - 22 | 22 - 28 |
| a Pal at base of fingers | 16 - 22 | 18 - 24 | 20 - 26 |
| Lengths cm | SHORT | REGULAR | |
| c-g Wrist to axilla | 36 - 42 | 40 - 46 | |

Comfort

Elbow stretch zone

Pressure is evenly distributed to prevent tightness when the elbow bends.

“Excellent, very comfortable to sleep in. Having had to do massage and bandaging in the past, this is simple and works!”

Patient review, lymphshop.com



Codes

| | S | M | L |
|---------|----------|----------|----------|
| SHORT | CW1-AH2S | CW1-AH3S | CW1-AH4S |
| REGULAR | CW1-AH2R | CW1-AH3R | CW1-AH4R |

Unit of Sale

Single

Comfiwave

Glove

Material Composition: Cotton, Polyester, Polyurethane

A soft compressive fabric, with a distinctive wave pattern knit. The wave profile creates a pressure differential, that leaves lines/grooves on a patient's skin that help soften the tissues while promoting lymphatic drainage.

Comfiwave offers a unique additional option for patients to use in their self management regimen, which can be worn in a more relaxed setting at home and even overnight.



Compression

| mmHg | Well suited for |
|-------|---|
| 15-21 | Positive Stemmer sign, mild shape distortion, pitting/fibrosis to the dorsum of the hand, palliative care |

Sizing



| Circumferences cm | S | M | L |
|------------------------|---------|---------|---------|
| b Base of thumb | 17 - 22 | 20 - 25 | 23 - 28 |
| c Wrist | 14 - 20 | 16 - 24 | 22 - 28 |

Comfort

Reduced elastic at fingertips

Increased comfort and circulation

Knitted gussets

Gussets in between the fingers improves comfort and reduces chafing

Versatility

Can be used with Comfiwave Sleeve

Don and doff the glove as required without having to remove compression from your arm



Codes

| | | | | | | |
|----------------|----------|--|----------|--|----------|--|
| | S | | M | | L | |
| REGULAR | CW1-ZC2 | | CW1-ZC3 | | CW1-ZC4 | |
| Unit of Sale | | | Single | | | |

Comfiwave

Glove to Elbow

Material Composition: Cotton, Polyester, Polyurethane

A soft compressive fabric, with a distinctive wave pattern knit. The wave profile creates a pressure differential, that leaves lines/grooves on a patient's skin that help soften the tissues while promoting lymphatic drainage.

Comfiwave offers a unique additional option for patients to use in their self management regimen, which can be worn in a more relaxed setting at home and even overnight.



Compression


mmHg

Well suited for

15-21

Oedema extending from the fingers (positive Stemmer sign) to just below the elbow, pitting and fibrosis to the forearm, mild shape distortion.

Sizing

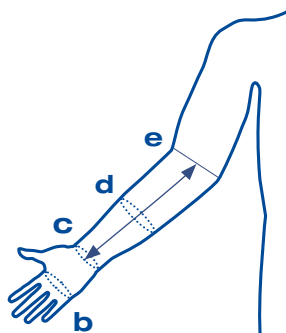

Circumferences cm

S

M

L

b Base of thumb webbing

17 - 22

20 - 25

23 - 28

c Wrist

14 - 20

16 - 24

22 - 28

d Mid-point of forearm

23 - 28

26 - 32

30 - 38

Lengths cm

REGULAR

c-e Wrist to elbow

20 - 24

Comfort

Reduced elastic at fingertips

Increased comfort and circulation

3D finger webbing

Extra cotton in the finger gussets
relieves pressure and prevents chafing



Codes

| | | | | | |
|----------------|----------|--|----------|--|----------|
| | S | | M | | L |
| REGULAR | CW1-ZE2 | | CW1-ZE3 | | CW1-ZE4 |
| Unit of Sale | | | Single | | |

Comfiwave

Glove to Axilla

Material Composition: Cotton, Polyester, Polyurethane

A soft compressive fabric, with a distinctive wave pattern knit. The wave profile creates a pressure differential, that leaves lines/grooves on a patient's skin that help soften the tissues while promoting lymphatic drainage.

Comfiwave offers a unique additional option for patients to use in their self management regimen, which can be worn in a more relaxed setting at home and even overnight.




Compression

mmHg Well suited for

15-21

Oedema extending from the fingers throughout the whole arm, pitting and fibrosis, palliative care, mild to moderate shape distortion

Sizing

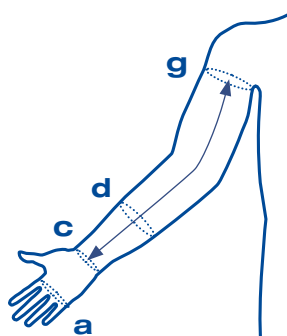

| Circumferences cm | | S | M | L |
|-------------------|-------------------------|---------|---------|---------|
| g | Axilla | 25 - 30 | 30 - 40 | 40 - 50 |
| d | Mid-point of forearm | 23 - 26 | 26 - 30 | 30 - 38 |
| c | Wrist | 13 - 18 | 18 - 22 | 22 - 28 |
| a | Palm at base of fingers | 16 - 22 | 18 - 24 | 20 - 26 |
| Lengths cm | | SHORT | REGULAR | |
| c-g | Wrist to Axilla | 36 - 42 | 40 - 46 | |

Comfort

Reduced elastic at fingertips

Increased comfort and circulation

3D Finger webbing

Extra cotton in the finger gussets
relieves pressure and prevents chafing

“I’m very impressed with how this works I wear it mainly at night and my fingers look amazing.”

Patient review, lymphshop.com



Codes

| | S | | M | | L | |
|---------|----------|--|----------|--|----------|--|
| SHORT | CW1-ZH2S | | CW1-ZH3S | | CW1-ZH4S | |
| REGULAR | CW1-ZH2R | | CW1-ZH3R | | CW1-ZH4R | |

Unit of Sale

Single

Comfiwave

Breast Band

Material Composition: Cotton, Polyester, Polyurethane

A soft compressive fabric, with a distinctive wave pattern knit. The wave profile creates a pressure differential, that leaves lines/grooves on a patient's skin that help soften the tissues while promoting lymphatic drainage.

Comfiwave offers a unique additional option for patients to use in their self management regimen, which can be worn in a more relaxed setting at home and even overnight.




Compression

mmHg Well suited for

15-21 Breast discomfort, mild to severe lymphoedema with or without pitting tissues and shape distortion. Post breast surgery.

Sizing

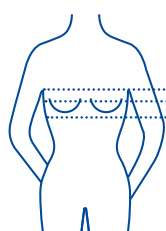

Circumferences cm

S

M

L

XL

Above breast

70 - 88

80 - 102

90 - 116

100 - 130

Over breast

70 - 92

80 - 108

90 - 126

100 - 144

Below breast

70 - 80

80 - 92

90 - 104

100 - 116

Comfort

Gentle bust coverage

A softer knit around the chest for a comfortably reduced compression level.

Support

Underarm support

Increased elastic thread count under the arms for stronger compression where you need it most.

Use post-operatively

Following breast augmentation or for those who have had breast conserving surgery such as lumpectomy.



“WOW!! This has significantly increased the effective management of my breast lymphoedema. I only wear it overnight and, every morning, my ‘lymph boob’ feels very, very close to just ‘normal’.”

Lymphoedema patient

Codes

| | | | | | | | | |
|----------------|----------|---|----------|--------|----------|---|-----------|---|
| | S | | M | | L | | XL | |
| REGULAR | CW1-TN2 | ⋮ | CW1-TN3 | ⋮ | CW1-TN4 | ⋮ | CW1-TN5 | ⋮ |
| Unit of Sale | | | | Single | | | | |

Comfiwave

Neck

Material Composition: Cotton, Polyester, Polyurethane

A soft compressive fabric, with a distinctive wave pattern knit. The wave profile creates a pressure differential, that leaves lines/grooves on a patient's skin that help soften the tissues while promoting lymphatic drainage.

Comfiwave offers a unique additional option for patients to use in their self management regimen, which can be worn in a more relaxed setting at home and even overnight.




Compression

mmHg Well suited for

15-21 Submental and submandibular lymphoedema with or with pitting and shape distortion. Post surgery.

Sizing

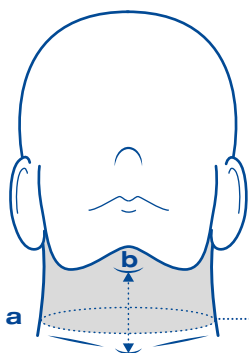

| Circumferences cm | S | M | L |
|-------------------|---------|---------|---------|
| a Base of neck | 32 - 38 | 37 - 43 | 42 - 47 |

| Lengths cm | SHORT | REGULAR |
|---------------------------------|-------|---------|
| a-b Base of neck to top of neck | 5-7 | 7-9 |

Comfort

Adaptive Knitting

Specially designed to accommodate drastic circumference changes from chin to neck.

Curved ear line

Features a subtle curve below the ears for a naturally contoured fit.

“The collar is keeping my lymphoedema completely under control without using the kinetic tape or self-massage. I can feel it working, move the skin, dispersing the fluid all of the time it’s on my neck.”

Lymphoedema patient

Codes

| | S | M | L |
|---------|----------|----------|----------|
| SHORT | CW1-HN2S | CW1-HN3S | CW1-HN4S |
| REGULAR | CW1-HN2R | CW1-HN3R | CW1-HN4R |

Unit of Sale

Single

Comfiwave

Thigh + Hip

Material Composition: Cotton, Polyester, Polyurethane

A soft compressive fabric, with a distinctive wave pattern knit. The wave profile creates a pressure differential, that leaves lines/grooves on a patient's skin that help soften the tissues while promoting lymphatic drainage.

Comfiwave offers a unique additional option for patients to use in their self management regimen, which can be worn in a more relaxed setting at home and even overnight.



Compression

mmHg

Well suited for

15-21


LYMPHOEDEMA

Mild to severe oedema of the thigh extending into the hip with or without pitting and shape distortion.

LIPOEDEMA

Smooth or uneven skin with an increase of enlarged subcutaneous fat tissue or lipomas.

Sizing

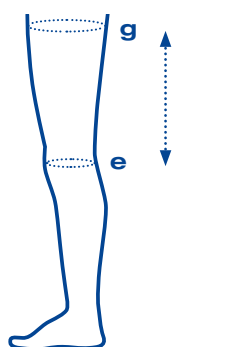

Circumferences cm

S

M

L

XL

g Thigh top

40 - 48

46 - 56

54 - 65

62 - 75

e Knee

27 - 35

33 - 39

41 - 50

48 - 60

Lengths cm

SHORT

REGULAR

e-g Knee to thigh top

20 - 26

25 - 31

Comfort

Hip flap

allows for wearing under normal underwear, where an abdominal band may not be needed.

Versatility

Use in combination

Can be combined with Comfiwave abdominal band and/or below knee for whole leg treatment



Codes

| | S | M | L | XL |
|---------|----------|----------|----------|----------|
| SHORT | CW1-ET2S | CW1-ET3S | CW1-ET4S | CW1-ET5S |
| REGULAR | CW1-ET2R | CW1-ET3R | CW1-ET4R | CW1-ET5R |

Unit of Sale

Single


Comfiwave

Kids Below Knee

Material Composition: Cotton, Polyester, Polyurethane

A one-of-a-kind addition to treatment options for children with lymphoedema. Optimised for infants and children with a lighter, softer feel, prioritising comfort during wear.


Comfiwave Kids offers a unique option to assist parents in the management of children with lymphoedema, which can be worn in a more relaxed setting at home and even overnight.

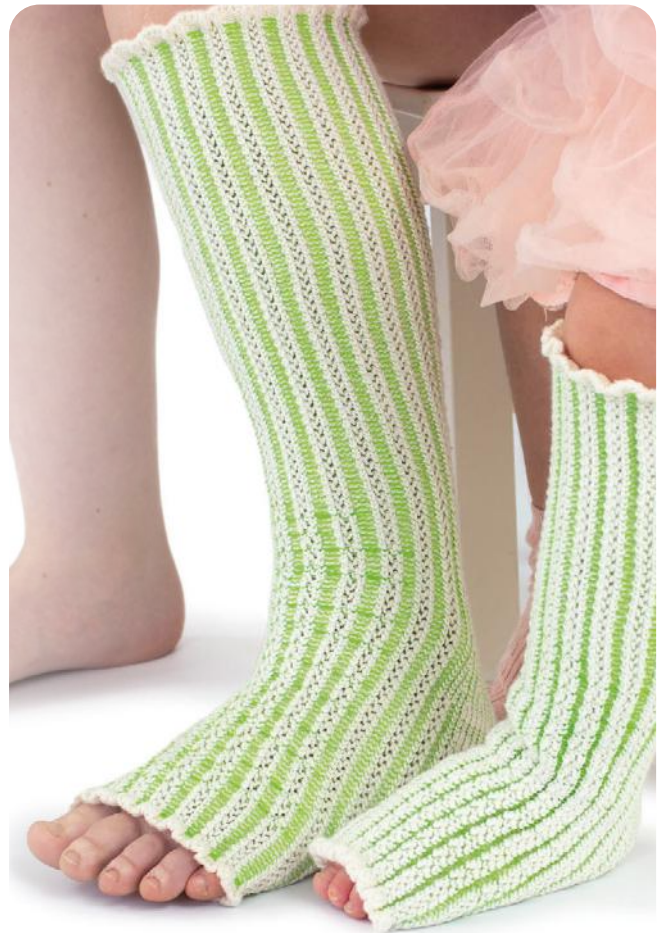

Compression

| | |
|-------|---|
| mmHg | Well suited for |
| 15-21 | Primary lymphoedema all stages in toddlers and children |

Sizing

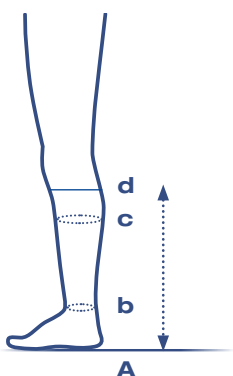

| Circumferences cm | PAEDIATRIC | | | |
|--------------------------------|------------|---------|---------|------------|
| c WIDEST CALF | 20 - 33 | | | |
| b ANKLE | 14 - 22 | | | |
| Lengths cm | SHORT | REGULAR | LONG | EXTRA LONG |
| A-d FLOOR TO BELOW KNEE | 17 - 22 | 22 - 27 | 27 - 32 | 32 - 37 |

Comfort

Lighter

The cotton ply has been reduced by 20%, resulting in an even lighter and more comfortable fabric for children.

Friendly

New Kids on the box

Comfiwave's packaging has had a makeover to appeal to children, with simple instructions to help them feel in control. Plus, shiny stickers in every box!



Codes

| | P | |
|-------------------|----------|------|
| SHORT | CW1-AD0S | ■ |
| REGULAR | CW1-AD0R | ■■ |
| LONG | CW1-AD0L | ■■■ |
| EXTRA LONG | CW1-AD0X | ■■■■ |

Unit of Sale

Single



etogarmen
Prairie Wear




etogarmments

Garments for head, neck, breast, genital, and torso oedema, as well as garments suitable for lipoedema and breast care.



Find out more

Turn the page to explore [Page 126](#) ▶

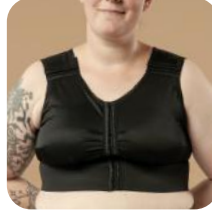

eto Bra
Page 128 ▶



eto Breast Band
Page 130 ▶



eto Head & Neck
Page 131 ▶



eto Vest
Page 132 ▶



eto Scrotal Support
Page 133 ▶



eto Shorts
Page 134 ▶



eto Grace
Page 136 ▶



eto Abdominal Pouch
Page 138 ▶

Prairie Wear

Supportive, secure and comfortable compression bras for post-surgical recovery, lymphedema management or active recovery.



Find out more

Explore the range
[Page 138](#) ▶



HuggerPRIMA
Page 140 ▶



HuggerVIDA
Page 142 ▶



HuggerLUMA
Page 144 ▶

etogarments

Eto garments encompass a broad range of supportive, breathable compression garments for treatment of oedema extending to the head, neck, breast, and torso, as well as garments suitable for lipoedema.



Comfort

Eto garments are made from a fabric that is silky smooth on the outside and lightly textured on the inside, for low friction and increased comfort.

External seams

Heat-sealed external seams reduce pressure marks on the skin.

“ There isn't much out there that can even come near it in terms of what you can do with it - pockets, flies and seams.”

Mark Pearson, St George's Hospital London



Highly configurable

Eto garments are available in extensive sizing options for treating even the most severe cases of swelling and shape distortion.

Guaranteed
6 months of use.

Availability
Made-to-order unless indicated

eto garments

Bras

Material Composition: Elastane

A high quality compression bra made from oeko-tex certified fabric, free from harmful chemicals and dyes.

Eto bras are configurable to the wearers specification - from altering the straps and closures, to specifying pockets for prostheses to paddings for extra comfort.

“ My patient absolutely loves this product and finds it incredibly comfortable and easy to wear.

Feedback from lymphoedema therapist




Compression

mmHg Well suited for

25-30 Post breast surgery. where there is pitting and some moderate skin changes to the breast.

Sizing

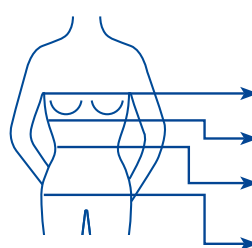

| Circumferences cm | XS | S | M | L | XL | XXL | XXXL |
|---------------------------|-------|-------|---------|---------|---------|---------|---------|
| Bra size | A | A / B | B / C | C / D | D / E | E / F | F / G |
| Above breast | 78-82 | 83-87 | 88-92 | 93-97 | 98-102 | 103-107 | 108-112 |
| Below Breast | 65-71 | 72-77 | 78-85 | 86-94 | 95-100 | 101-109 | 110-118 |
| Waist | 60-65 | 66-72 | 73-79 | 80-90 | 91-98 | 99-106 | 107-113 |
| Hips H603L only | 89-93 | 94-99 | 101-109 | 110-119 | 120-125 | 126-130 | 131-135 |

Configurable

Pockets

Add for a prosthesis or additional padding

Adjustable fastenings

Hook & eye fastenings to the front and straps allow support to be adjusted

Comfortable

Empire foam

Optional additional under-breast padding for extra comfort (pictured right)




Klima-comfort under arm gussets

Add for additional comfort under arm





Options

*Available for H601, H603 and H603L only.

| | | | |
|-------------------------|---|---|---|
| Colour |  Beige |  Black |  White |
| Front fastening* | Hook & Eye Default | Zip | |
| Cups* | Empire foam under breast | Pocket | |

| | | |
|-------------------------|-------------------------|-----------------------------------|
| Strap Width* | Narrow 5cm | Wide 8cm |
| Strap fastening* | Hook & Eye | Velcro |
| Openings* | Lower Neckline 4.5cm | Klima-comfort under arm gusset |

Codes

| |  |  |  |  |
|-----------------|---|---|--|---|
| 25-30mmHg | ETO-H601 | ETO-H603 | ETO-H603L | ETO-H607 |
| Neck Shape | V | Scoop | V | V |
| Strap width | Wide | Narrow | Narrow | Narrow |
| Strap fastening | Hook & Eye | Hook & Eye | Hook & Eye | None |
| Front fastening | Hook & Eye | Hook & Eye | Hook & Eye | None |
| Unit of Sale | Single | Single | Single | Single |

Custom made by quotation

eto garments

Breast Band

Material Composition: Elastane


High quality compression breast band made from oeko-tex certified fabric free from harmful chemicals and dyes.



Compression

| Class | mmHg | Well suited for |
|-------|-------|--|
| - | 25-30 | Post surgery, where oedema is extensive and extra compression is required to reduce the swelling |

Sizing

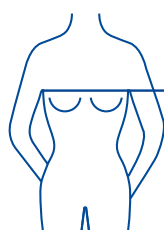

| Circumference cm | XS | S/M | L/XL | XXL |
|------------------|-------|--------|---------|---------|
| Above breast | 80-90 | 90-100 | 100-110 | 110-120 |

Options

Colour



Beige



Black



White

Codes



25-30mmHg

ETO-602

Unit of Sale

Single

eto garments

Head & Neck Straps

Material Composition: Elastane

A high quality compression garment for head and neck oedema, made from oeko-tex certified fabric free from harmful chemicals and dyes.

Compression

| Class | mmHg | Well suited for |
|-------|-------|--|
| - | 25-30 | Oedema of the neck, submental, submandibular region extending into the lower cheeks, moderate pitting with some shape distortion |

Sizing

| Circumferences cm | ETO 200, 201, 202, 203, 206 | | ETO209 | |
|----------------------------------|-----------------------------|---------|----------|---------|
| | STANDARD | XL | STANDARD | XL |
| 1 ABOVE EYEBROWS | < 66 | 66 - 71 | < 66 | 66 - 74 |
| 2 POINT OF CHIN TO CROWN OF HEAD | < 66 | 66 - 71 | < 66 | 66 - 74 |
| 3 NECK | < 37 | 37 - 40 | < 37 | 37 - 41 |

Options

Colour



Beige



White*

*Not available for ETO209 model

Codes



ETO-200



ETO-201



ETO-202



ETO-203



ETO-206



ETO-209

25-30mmHg

Unit of Sale

Single

Single

Single

Single

Single

Single

Custom made by quotation

Stock Options

All other variants made-to-order in 12-14 days

Codes

ETO-201
ETO-206
ETO-209

Size

Standard
XL

Colour

Beige
White*



ETO-209

eto garments

Vest

Material Composition: Elastane

High quality compression vest made from oeko-tex certified fabric free from harmful chemicals and dyes.


Compression vests that provide support and compression in the chest and abdominal/truncal area, with easy fastening and comfortable zip.



Compression

| Class | mmHg | Well suited for |
|-------|-------|--|
| - | 25-30 | Bilateral mastectomy or unilateral mastectomy in men, pitting and mild to moderate skin changes. |

Sizing

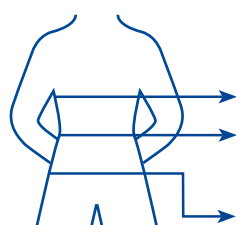

| Circumference cm | XS | S | M | L | XL | XXL |
|------------------|-------|-------|--------|---------|---------|---------|
| Chest | 79-83 | 84-89 | 90-99 | 100-110 | 111-120 | 121-125 |
| Waist | 75-79 | 80-85 | 86-95 | 96-106 | 107-115 | 116-122 |
| Hips | 85-89 | 90-95 | 96-105 | 106-116 | 117-123 | 124-130 |

Options

Colour



Beige



Black



White

Codes



25-30mmHg
ETO-402

Unit of Sale

Single



ETO-400

Single

eto garments

Scrotal Support

Material Composition: Elastane

High quality pouches made from oeko-tex certified fabric free from harmful chemicals and dyes.


Support pouch with an adjustable waistband and option for additional pad at perineum.



Compression

| Class | mmHg | Well suited for |
|-------|------|---------------------------------|
| - | 0 | Oedema of the scrotum and penis |

Sizing

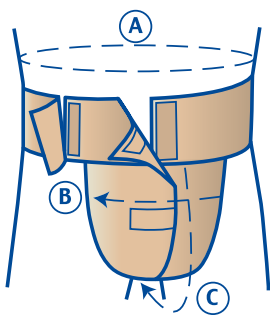

| | S/M | | | L/XL | | | XXL | |
|---|--------|-------|-------|---------|-------|-------|-------|-------|
| A | 90-105 | | | 106-119 | | | 120+ | |
| | 1 | 2 | 3 | 4 | 5 | 6 | 7 | 8 |
| B | 22-24 | 24-26 | 26-28 | 28-30 | 30-32 | 32-34 | 34-36 | 36-38 |
| C | 23-26 | 26-29 | 29-32 | 32-35 | 35-38 | 38-41 | 41-44 | 44-47 |

Please Note: Point 'B' is measured from right to left of groin over widest part. Point 'C' is measured from pubic bone to perineum. We require both measurements in order to manufacture this garment.

Options

| Colour | Beige | Black | White |
|--------|-------|-------|-------|
| | | | |

Codes

Scrotal Support

25-30mmHg

ETO-11P

Unit of Sale

Single

Custom made by quotation

eto garments

Female shorts


Material Composition: Elastane

High quality compression shorts made from oeko-tex certified fabric free from harmful chemicals and dyes.


A female genital compression garment with a cotton lined crotch and a pocket to add additional padding.

Compression

| Class | mmHg | Well suited for |
|-------|-------|--|
| - | 25-30 | Lymphoedema of the suprapubic region and labia |

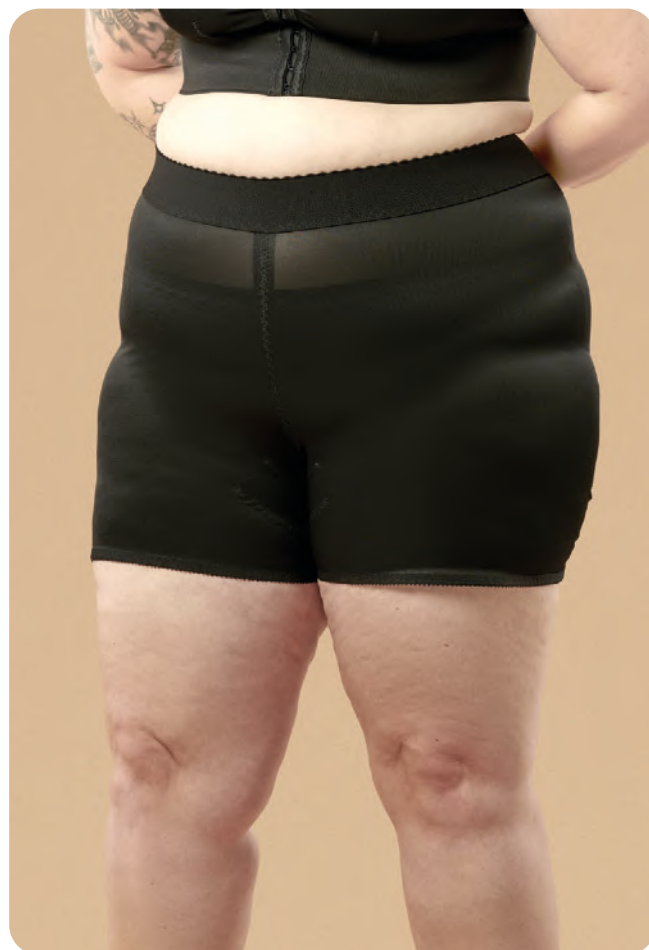

Sizing
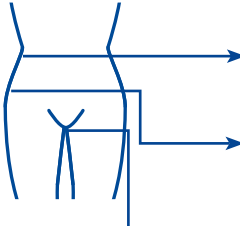
| | Circumferences cm | | | | | | | |
|--|-------------------|-------|---------|---------|---------|---------|---------|--|
| | XS | S | M | L | XL | XXL | XXXL | |
|  Waist | 60-65 | 66-72 | 73-79 | 80-90 | 91-98 | 99-106 | 107-114 | |
| Hip | 89-93 | 94-99 | 100-109 | 110-119 | 120-125 | 126-130 | 131-135 | |
| Left top | 49-51 | 52-54 | 55-58 | 59-61 | 62-64 | 65-68 | 69-72 | |

Options

Colour



Beige



Black



White

Codes



Sized

25-30mmHg

ETO-H106

Unit of Sale

Single

Custom made by quotation

eto garments

Male shorts

Material Composition: Elastane

High quality compression shorts made from oeko-tex certified fabric free from harmful chemicals and dyes.




A male genital compression garment with a cotton lined crotch with optional fly-opening.



Compression

| mmHg | Well suited for |
|-------|------------------|
| 25-30 | Scrotal swelling |

Sizing
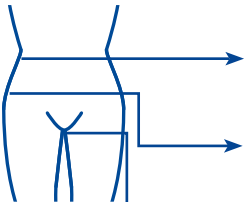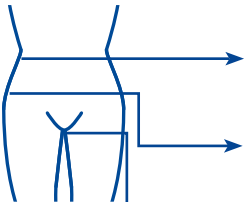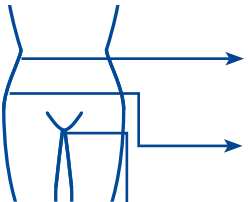
| | Circumferences cm | | | | | | | |
|---|-------------------|-------|---------|---------|---------|---------|---------|--|
| | XS | S | M | L | XL | XXL | XXXL | |
|  Waist | 60-65 | 66-72 | 73-79 | 80-90 | 91-98 | 99-106 | 107-114 | |
|  Hip | 89-93 | 94-99 | 100-109 | 110-119 | 120-125 | 126-130 | 131-135 | |
|  Left top | 49-51 | 52-54 | 55-58 | 59-61 | 62-64 | 65-68 | 69-72 | |

Options

| Colour |  |  |  |
|--------|---|---|---|
| | Beige | Black | White |

Codes



Sized

25-30mmHg

ETO-H99

Unit of Sale

Single



Sized

ETO-H96

Single

Custom made by quotation

eto garments

Grace Leggings

Material Composition: Elastane

High quality compression leggings made from oeko-tex certified fabric free from harmful chemicals and dyes.

Soft and stretchy to help donning and doffing. With a silky outer layer and lightly textured fabric against the skin to improve all-day comfort.

“ I like the nice feeling and the security it gives me. I wear the Eto Grace more when I am sore and achy, as it is so comfy.

Feedback from patient with lipoedema



Compression

| mmHg | Well suited for |
|-------|---|
| 25-30 | All types and stages of lipoedema, post liposuction |

Sizing

Download measurement form at www.hadhealth.com/f0060



Comfortable

Breathable

Warp knit with a light open weave to wick heat and moisture away

Advanced microfibres

Soft and stretchy to help donning and doffing.


Convenient

Lightweight

The fabric moves easily with you through your daily work/exercise

Machine washable

at 30 degrees

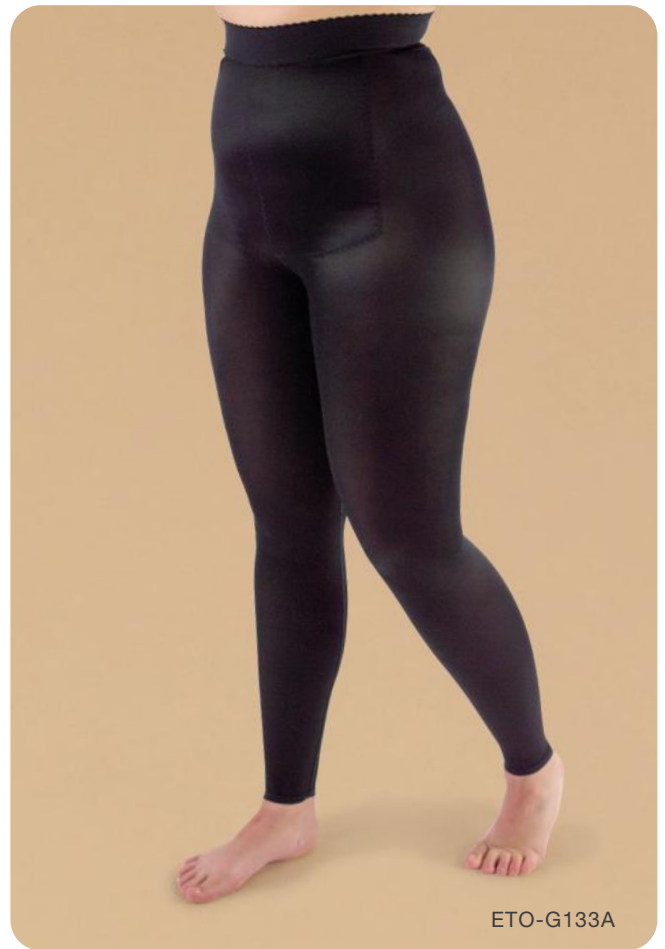

Codes



Tights



Footless Tights



Capri

25-30mmHg

ETO-G134

ETO-G133A

ETO-G133

Unit of Sale

Single

Single

Single

Options

Colour



Beige



Black



White

Front fastening

None
Default



Zip



Hook & Eye

Crotch

Closed
Default

Open

Prairie Wear

Haddenham Healthcare proudly offers Prairie Wear support bras, featuring 360-degree, wire-free mapped compression for comfort and support.

Ideal for post-surgery recovery, managing lymphedema, and everyday active wear, these bras combine functionality with comfort.




Simple sizing

Just one circumference measurement is needed to find a perfect fit.

Full coverage

360 degree targeted and mapped compression that hugs the body.

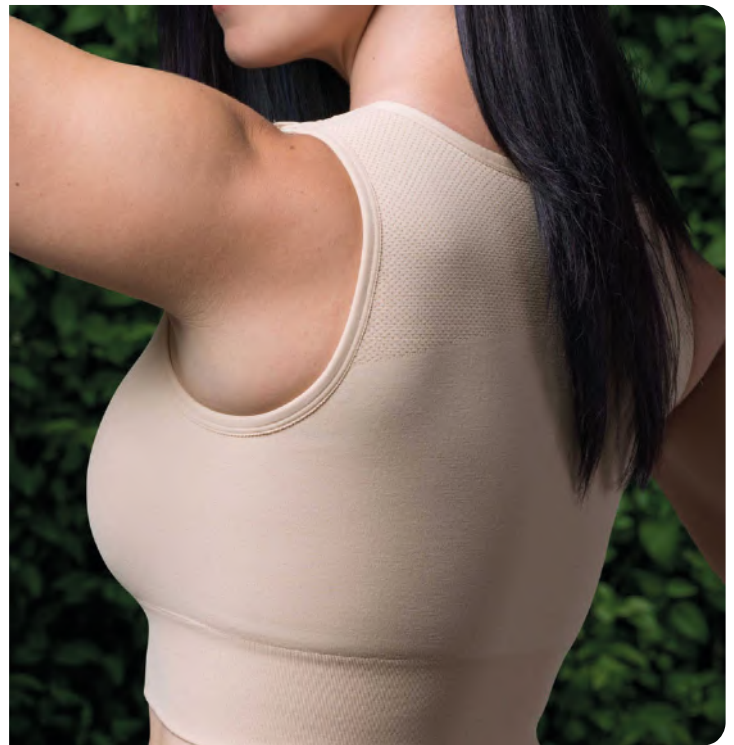

Accommodating

Extenders allow for phased support post-surgery and to adapt to fluctuation in oedema.

Also includes pockets for prosthetics, compression padding, ice and more.

Attractive

Prairie Bras are available in a pleasing mixture of bright and natural tones.



Availability

All variants held **in stock**

Prairie Wear

HuggerPrima

Material Composition: Nylon, Spandex

Comfortable support bras with 360-degree, wire-free mapped compression. For after surgery, for Lymphoedema, or as everyday active wear.

The HuggerPrima is a medium-to-high compression bra with high coverage.



Comfort

Extenders

For phased support as swelling reduces


Adjustable

Padded straps, and pockets for foams or prosthetics

Compression

| Class | mmHg | Well suited for |
|----------|------|--|
| Med-High | - | Post breast surgery, where there is pitting and some moderate skin changes to the breast |


Sizing

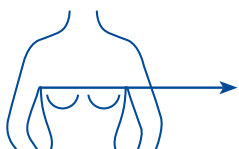

| Circumferences cm | XS | S | M | L | XL | 2XL | 3XL |
|-------------------|-------|-------|-------|--------|---------|---------|--|
| Above breast | 66-71 | 76-81 | 86-91 | 96-102 | 107-112 | 117-122 | 127-132 <small>2XL + L Extender</small> |



For **PRIMA**: *Upsize* if you're in-between sizes or you're a DD cup or higher.

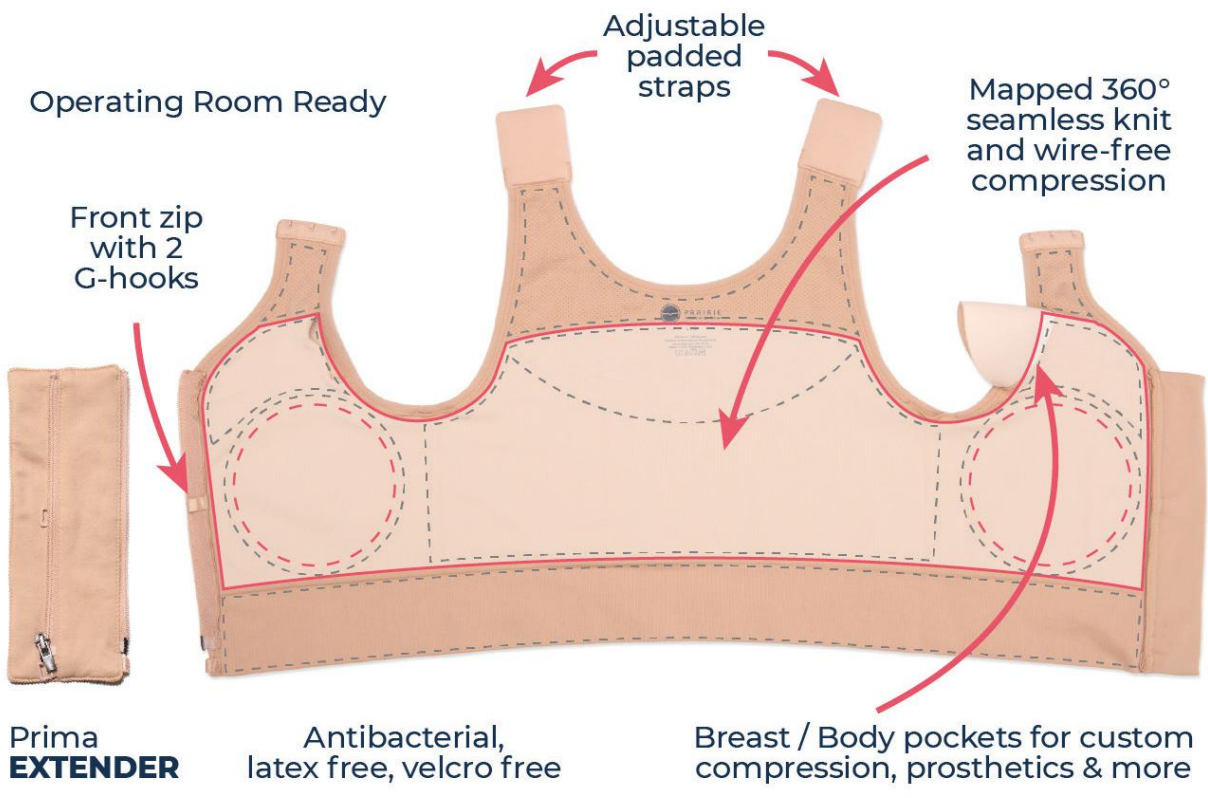

Options



Codes



HuggerPrima

HUG

Unit of Sale

Single



Extender

Bra sizes S-M

EXT-M

Single

Bra Sizes L-2XL

EXT-L

Single

Prairie Wear

HuggerVida

Material Composition: Nylon, Spandex

Comfortable support bras with 360-degree, wire-free mapped compression. For after surgery, for Lymphoedema, or as everyday active wear.

The HuggerVida is a low-to-medium compression bra, with low-to-medium coverage.



Comfort

Extenders

For phased support as swelling reduces


Adjustable

Padded straps, and pockets for foams or prosthesis

Compression

| Class | mmHg | Well suited for |
|---------|------|---|
| Low-med | - | As a step down from higher compression as maintenance post oedema reduction |


Sizing

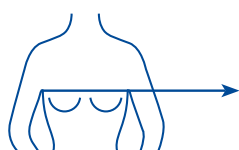

| Circumferences cm | XS | S | M | L | XL | 2XL | 3XL |
|-------------------|-------|-------|-------|--------|---------|---------|--|
| Above breast | 66-71 | 76-81 | 86-91 | 96-102 | 107-112 | 117-122 | 127-132 <small>2XL + L Extender</small> |



In-between sizes?
For **VIDA**, downsize.

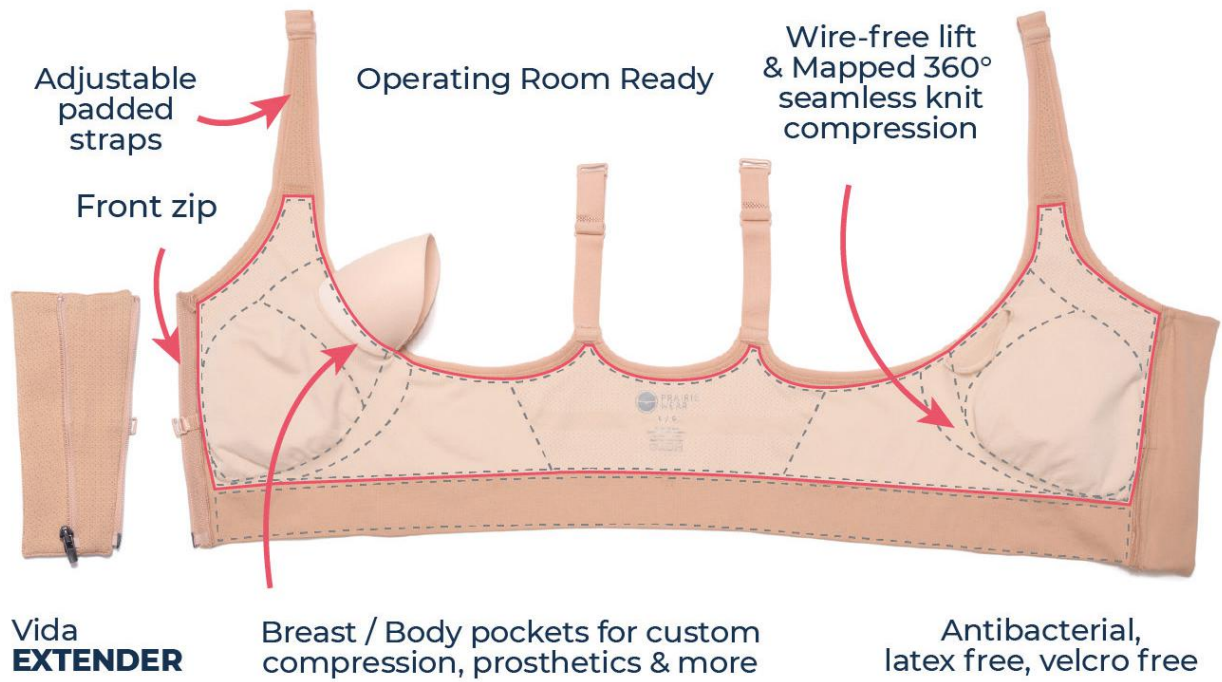

Options

Colour



Codes



HuggerVida

VIDA

Unit of Sale

Single



Extender

Bra sizes S-M

VXT-M

Single

Bra Sizes L-2XL

VXT-L

Single

Prairie Wear

HuggerLuma

Material Composition: Nylon, Spandex

Comfortable support bras with 360-degree, wire-free mapped compression. For after surgery, for Lymphoedema, or as everyday active wear.

The HuggerLuma is a medium-to-high compression sports bra with low coverage.



Comfort

Extenders

For phased support as swelling reduces


Adjustable

Padded straps and back band

Compression

| Class | mmHg | Well suited for |
|----------|------|---|
| Med-High | - | Post-surgery daily wear where oedema fluctuates and higher compression is needed. |

Sizing

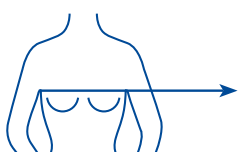

| Circumferences cm | XS | S | M | L | XL | 2XL |
|-------------------|-------|-------|-------|--------|---------|---------|
| Above breast | 66-71 | 76-81 | 86-91 | 96-102 | 107-112 | 117-122 |



For **LUMA**: *Downsize* if you're in-between sizes. *Upsize* if you're a DD cup or higher.



Options

COLOUR



Big Sky Blue



Fawn



No Moon Black

Codes



HuggerLuma

LUMA

Unit of Sale

Single



**Accessories
+ Bandages**



Delfin

PMC
41
Ref
38
PMC RATE
1.07

LYMPHSCANNER

Accessories

Compression bandages, kinesiology tape, foam paddings and skin adhesive for assisting existing compression treatment.

* Items held **in stock**.



Lymphscanner

Page 148 ▶



***Steve Donning Aids**

Page 150 ▶




Lymphpad Foam

Page 154 ▶



Spaghetti Foam

Page 155 ▶

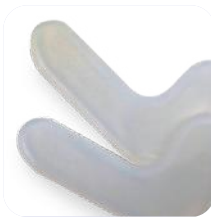

Malleolus Pads

Page 155 ▶



***Kinesiology Tape**

Page 156 ▶



Abdominal Binder

Page 157 ▶



***Klebi Skin Adhesive**

Page 158 ▶

Lymphscanner

PWC Measurement Unit

The Lymphscanner is an all in one Percentage Water Content (PWC) measurement unit that is composed of an integrated probe, a built-in contact pressure sensor and a display screen.



Codes

Lymphscanner Module

MDL2001

Service

MDL SERVICE



Precision


Tracks treatment effectiveness

Helps evaluate for response and intervention

Non-invasive measurements

Scanning is painless and can be repeated

The device is used for the assessment and detection of lymphoedema. It measures localized tissue water content and displays a PWC value.

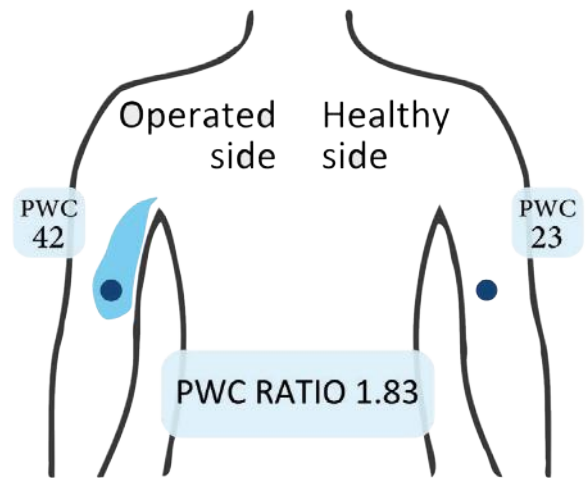

LymphScanner automatically calculates the interim ratio of PWC measurements

Measuring the percentage water content (PWC) of the tissue is useful in early detection of lymphoedema and in follow-up of lymphoedema treatments.

PWC values vary by measurement site and thus a measurement method capable of rapid skin scanning helps lymphologists and therapists record spatial information on the regional extent of the patient's lymphoedema.



Steve

Steve Complete

Material Composition: ABS, Nylon, Aluminium

The Steve range of donning aids offer solutions for both putting on and removing compression stockings without bending over.

The Steve Complete stocking aid is very lightweight and easy to use, making it ideal for use in cases of arthritis or rheumatism.



Effortless


Time saving

Reduces time spent donning and doffing garments.

Restores independence

Empowers patients to self manage their compression therapy.

Sizing

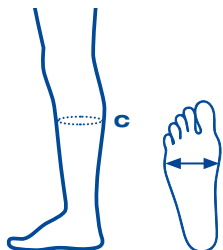

Measurements cm

XS

S

M

L

c **Widest Calf Circumference**

25-30

30-38

38-46

46-54

Foot Width Straight Distance

8.3

10.5

12.5

14.5

Codes

ST1119822014

ST119822014

ST219822014

ST319822014


Steve easyON

Material Composition: ABS

Easily put on all types of compression hosiery, with or without assistance.



Sizing

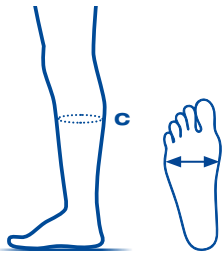

| Measurements cm | XS | S | M | L |
|---|-----------|----------|----------|----------|
| c Widest Calf Circumference | 25-30 | 30-38 | 38-46 | 46-54 |
| Foot Width Straight Distance | 8.3 | 10.5 | 12.5 | 14.5 |

Codes

AT1119822014 AT119822014 AT219822014 AT319822014


Steve easyOFF Ring

Material Composition: Nylon

Makes removing compression hosiery.



Sizing

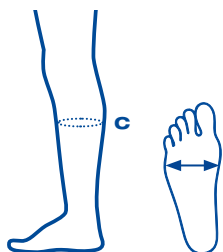

| Measurements cm | XS | S | M | L |
|---|-----------|----------|----------|----------|
| c Widest Calf Circumference | 25-30 | 30-38 | 38-46 | 46-54 |
| Foot Width Straight Distance | 8.3 | 10.5 | 12.5 | 14.5 |

Codes

R1119822014 R119822014 R219822014 R319822014

Please note: the Steve easyON and the Steve easyOFF Ring are also compatible for donning/doffing armsleeves. When selecting either of these products for use with upper limb garments, measure arm circumference at point g (axilla) and compare to the widest calf values listed above.

Steve Travel Bag

Material Composition: Nylon

Bag for transporting Steve donning/doffing aids.

Codes

1 Bag

TB119822015



Steve Glide Dolphin - Open Toe

Material Composition: Nylon with Silicone Coating

Easy and economical way of getting your compression hosiery on and off.

Sizing

Measurements cm

S

L

Shoe Size

4-9

10-13

Codes

SGO119822014

SGO219822014



Steve Glide Dolphin - Closed Toe

Material Composition: Nylon with Silicone Coating

Easy and economical way of getting your compression hosiery on and off.

Codes

1 Size

SGC119822014



Steve Gloves

Material Composition: Polyester, Nitrile

Maximum grip for your compression hosiery. Latex free.



Sizing

| | S | M | L | XL |
|----------------------|---------|---------|---------|-----------|
| Straight distance cm | | | | |
| Width of hand | 7.5 - 8 | 8 - 8.5 | 9 - 9.5 | 10 - 10.5 |

Codes

| | | | |
|-------------|-------------|-------------|-------------|
| GN119822014 | GN219822014 | GN319822014 | GN419822014 |
|-------------|-------------|-------------|-------------|

Steve Latex Gloves

Material Composition: Polyester, **Latex**

Maximum grip for your compression hosiery. Contains latex.



Sizing

| | S | M | L | XL |
|----------------------|---------|---------|---------|-----------|
| Straight distance cm | | | | |
| Width of hand | 7.5 - 8 | 8 - 8.5 | 9 - 9.5 | 10 - 10.5 |

Codes

| | | | |
|-------------|-------------|-------------|-------------|
| GL119822014 | GL219822014 | GL319822014 | GL419822014 |
|-------------|-------------|-------------|-------------|

Lymphpad

Foam Padding

Material Composition: Styrene Butadiene, Polyamide, Polyester



Haddenham Lymphpad –when used in conjunction with compression therapy– can help to break down fibrotic tissue by creating changes in tissue pressure and enhancing the massaging effect of treatment.

Lymphpad comes in either a dot or a wave form pattern, backed by a woven fabric.

Supportive

Increase lymphatic drainage

Supplements existing compression treatments

Break down fibrotic tissue

Softening skin for improved mobility and comfort

Codes

Dot Pattern

20.5cm x 29.5cm

LPD

Wave Pattern

20.5cm x 29.5cm


LPW

Spaghetti Foam

Large economical sheet that can be cut to size.

1.3m x 1m

87000

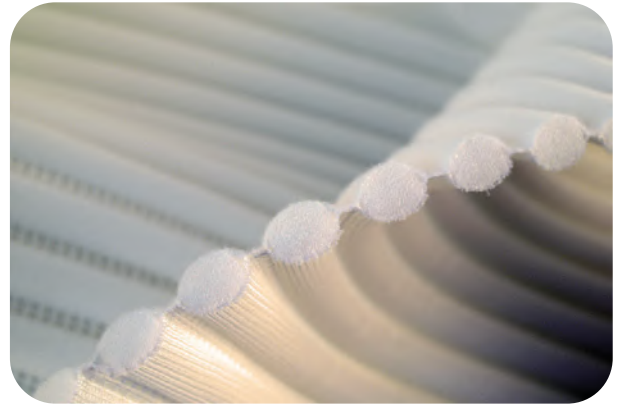

LTA Blue Round Ball

Helps carry out self lymphatic drainage.

1 Ball

440



Malleolus Pads

Soft silicone pad for use at either side of the ankle.

Left Side

425

Right Side

426



K-Active

Kinesiology Tape

K-Active kinesiology tape can be used to assist with lymphatic drainage by gently lifting the skin, helping congested fluid to move deeper into the lymphatic system.



K-Active is available in three different strength categories: Classic, Elite and Gentle. Elite is suitable for moist or oily skin, as well as those active during the day. Gentle is recommended for children, elderly patients and those with sensitive skin.

“The tape is very good. After applying the tape it will last in place for 3-4 days easily, allowing for showers etc. It’s a great help with the lymphoedema I have in my foot.”

Patient review, lymphshop.com

Codes All variants held **in stock**.

| | Classic | Elite | Gentle |
|---|----------|----------|----------|
| | 5cm x 5m | 5cm x 5m | 5cm x 5m |
|  Beige | KA-050 | KAE-050 | KAG-050 |
|  Black | KA-050BL | - | - |
|  Blue | KA-050B | - | - |
|  Pink | KA-050R | - | - |
|  Green | KA-050GR | - | - |

Abdominal Binder

Material Composition: Polyamide, Viscose, Elastane

The Haddenham abdominal binder supports the healing process of postoperative wounds, providing high lateral stability.


Supportive

High lateral stability

Supports the spine using long longitudinal elastic compression.



Sizing

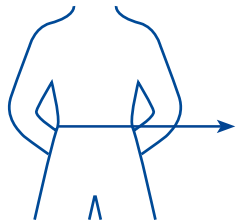

Circumference cm

S

M

L

XL

XXL

Waist

70-90

90-110

109-130

125-150

140-170

Options

Colour



White

Codes



39270
Unisex

Klebi Skin Adhesive

Ingredients: Isopropyl Alcohol, Glycerin, Water

Water soluble liquid adhesive used for maintaining the position of hosiery, orthopaedic aids, bandages and other textile articles.

410



Terms and conditions

| | |
|---------------------|---|
| Settlement | Nett 30 days. Accounts payable monthly. Interest on amounts outstanding over 30 days at the current bank rate + 3% on a daily basis. |
| Delivery | Standard charge of \$19.00 ex GST per delivery. Additional charge for courier service. Delivery times are approximate. Not responsible for courier's failure due to adverse weather or exceptional circumstances. |
| Prices | Subject to change without notice. Exclude GST. |
| Returns | There is a 90 day time limit in which to return all products. The following are required for return authorisation: Products code(s), Reason for return, applicable invoice number(s) or order reference(s). Returns will not be accepted without a Returns Authorisation Number. Customer liability returns for Credit Note may be subject to a \$12.00 handling charge per item. Returned goods must be saleable and have a minimum of 12 months shelf life remaining. Custom made items are non-returnable unless faulty. |
| Non-delivery | Non-receipt of goods must be notified in writing within 21 days of the date on the despatch note/invoice. |
| Warranty | Products are guaranteed against defects in material and manufacture. Once a garment is put into use, a 6-month warranty applies, covering material defects only. The warranty does not cover normal wear and tear, misuse, improper care, or use beyond the expiration date. Haddenham Healthcare reserves the right to replace or repair at its discretion. Specific warranty terms may vary per product. Consult product's 'Instructions for Use' for details. |
| Shelf life | Products have a shelf life of 3 years under correct storage conditions (unless otherwise specified). |
| Liability | Responsibility for selection and application of products rests with professional staff treating the patient. |
| Privacy | Privacy policy can be viewed at www.hadhealth.com.au/privacy-policy |
| Colours | Product colours depicted as accurately as printing process allows. Sample swatches available on request. |

Contraindications

Absolute contraindications for lower limb

1. Advanced peripheral arterial occlusive disease, 2. Decompensated Heart Disease, 3. Septic phlebitis & Phlegmasia cerulea dolens, 4. Diabetic patients with lack or reduced sensation in lower limb or foot, 5. Supportive dermatoses, intolerance of fabric, 6. Sensory disturbances of the limb, advanced peripheral neuropathy

Relative contraindications for lower limb

1. Primary chronic arthritis, 2. Advanced cancer to the root of the limb

Absolute contraindications for upper limb

1. Super vena cava obstruction, 2. Axillary vein thrombosis, 3. Supportive dermatoses, intolerance to fabric

Relative contraindications for upper limb

1. Brachial plexus neuropathy, 2. Rheumatoid arthritis, 3. Diabetes (peripheral neuropathy), 4. Advanced cancer to the root of the limb, 5. Cellulitis

Additional contraindications apply to particular garments. Always consult product instructions prior to use. Haddenham Healthcare Ltd assumes no liability for contraindicated use of garments.

Contact Us

Call: 03 9544 5515

Email: sales@hadhealth.com.au

Prices effective from

September 2025

Issue 8