APCAD S

MILLING BLOCKS

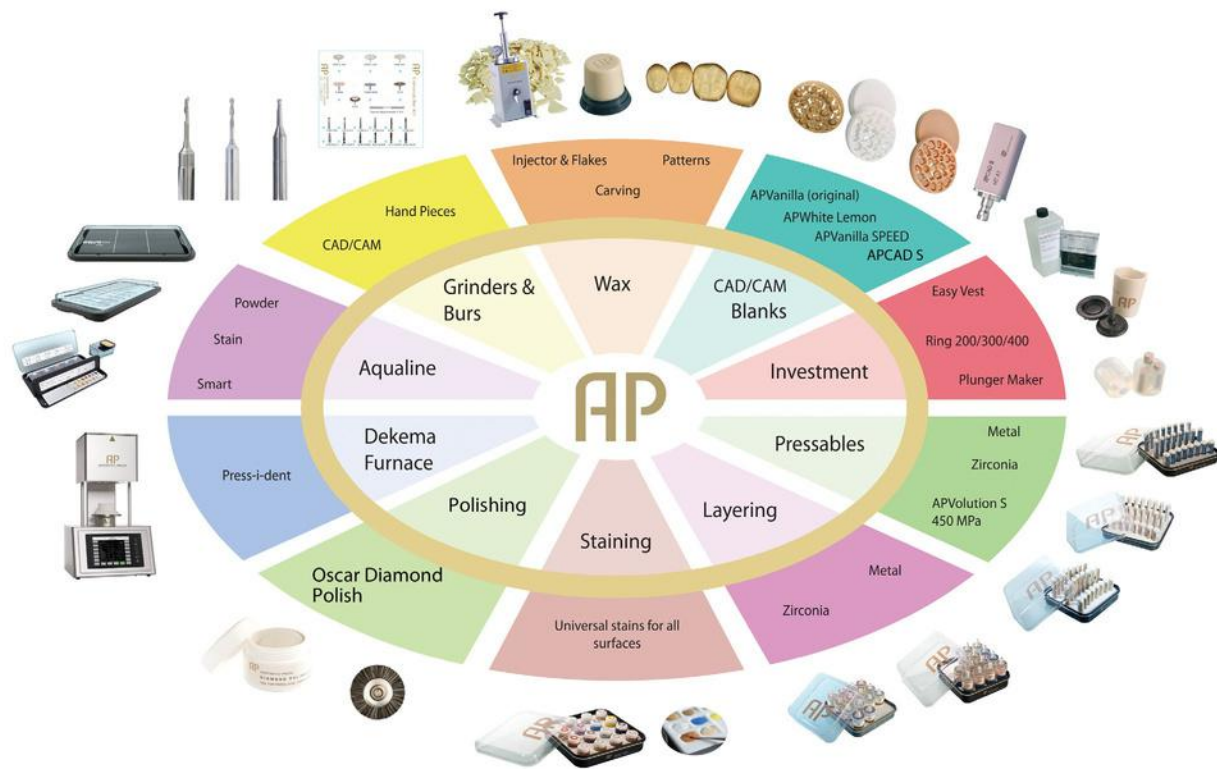
Lithium Disilicate 400±60 MPa

APVolution S

Ingots and Color Key Shade Guides

A Complete Workflow!

This is catalog only an extraction of our APVolution S product line. Please log on to our webshop or download our catalog on our website www.apdental.net for our full Aesthetic-Press system of products.

APCAD Blue Blocks- Lithium Disilicate milling blocks. The high strength performance and amazing aesthetics of this material make it our no.1 choice for restorative dentistry.

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The APVolution Color Key Shade Guide makes this product unique:

- Choose a shade with the material the technician is working with
- Choose a shade with a realistic indicator thickness
- Choose the right kind of indicator in conjunction with the stump shade that comes closest to the patients shade
- modify the indicator while taking the shade with your stains

Shade guide available in 1mm and 0.5 mm thickness for crowns, veneers and minimal invasive cases

APCAD S MILLING BLOCKS

Low Translucent

Size/Größe	Product Code/ Artikelnummer
APCAD S LT A1 18-15-13	SLTA1-18-15-13
APCAD S LT A1 40-15-15	SLTA1-40-15-15
APCAD S LT A2 18-15-13	SLTA2-18-15-13
APCAD S LT A2 40-15-15	SLTA2-40-15-15
APCAD S LT A3 18-15-13	SLTA3-18-15-13
APCAD S LT A3 40-15-15	SLTA3-40-15-15
APCAD S LT A3.5 18-15-13	SLTA3.5-18-15-13
APCAD S LT B1 18-15-13	SLTB1-18-15-13
APCAD S LT B1 40-15-15	SLTB1-40-15-15
APCAD S LT B2 18-15-13	SLTB2-18-15-13
APCAD S LT C1 18-15-13	SLTC1-18-15-13
APCAD S LT C2 18-15-13	SLTC2-18-15-13
APCAD S LT D2 18-15-13	SLTD2-18-15-13
APCAD S LT BL1 18-15-13	SLTBL1-18-15-13
APCAD S LT BL2 18-15-13	SLTBL2-18-15-13
APCAD S LT BL3 18-15-13	SLTBL3-18-15-13
APCAD S LT BL4 18-15-13	SLTBL4-18-15-13

FIRING CHART

FIRING CHART / BRENNTABELLE

CRYSTALLIZATION PROGRAM - APCAD S MILLING BLOCKS

	Idle Temp Bereitschaft	Dry Time Vortrocken	Rising °C/min Steigrate °C/min	End Temp Endtemperature	Holding Time Haltezeit	Vac Start	Vac End
Phase 1	400	6 min	90° C	820° C	0:10	550	820
Phase 2	-	-	30° C	840° C	7:00	550	840

AP Evolution S

400±60 MPa Lithium Disilicate

Color Key Shade Guide

The only shade guide
on the market with
12 stump shades

Indicator + Stump = Shade

Realistic indicator thickness of 0.9-1.1mm

LT, MT, HT, HT+ & Bleach Shades

Find the right opacity and value before you start the case

www.apdental.net

LT - Low Translucent

The LT ingots show a high level of chroma and opacity, which helps to cover dark stump shades or certain discolorations.

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Indication: Full crowns & bridges. In order to get a natural looking crown, the LT ingots can be used for the press/cut back and layering technique. The additional transparent layer will give the crown a life-like appearance, yet the core material will ensure the chroma needed to match the shade.

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MT - Medium Translucent

The MT ingots show a medium level of chroma and opacity, which helps to cover light stump shade discolorations.

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Indication: Full crowns and Veneers. In order to get a natural looking crown, the MT ingots can be used for both the press/cut back /layering technique and press & stain technique. The technician can define the amount of opacity needed through a defined depth cut - internal staining and micro-layering.

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HT - High Translucent

The HT ingots show a high level of translucency.

Indication: In order to get a natural looking crown, the HT ingots can be used for the press and stain technique. The overall appearance will be a natural translucent looking crown or veneer or even for non prep cases.

The HT+ ingots are designed for thin veneers or inlay only restorations offering a maximum of translucency.

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Bleach Ingots

The eleven bleach shades are available in different opacities as LT, MT and HT ingots offer a wide spectrum to choose from.

Indication: In order to get a natural looking crown, the Bleach ingots can be used in a variety of scenarios for either press & stain, or cut back & layering technique.

The technician can even increase the value if needed by adding a thin layer of transpa white layering powder mixed with a white modifier.

The APVolution S&Z Powder selection offers a wide range of powders compatible with the APVolution S ingots.

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Temperatures can vary between different furnaces up to 40°C.

Observe the visual result and contact customer support in case of questions.

FIRING CHART - BUILD UP POWDER / BRENNTABELLE - SCHICHTKERAMIK							S & Z Powder / Pulver
Build Up Powder	Idle Temp	Rising °C/min	End Temp	Preheat	Holding Time	Vac on	
Schichtkeramik	Bereitschaft	Steigrate °C/min	Endtemp.	Vortrocknen	Haltezeit	Vak an	
Dentin / Incisals	400°C	45°C	780°C	4 min	1 min	450°C	
Glaze Bake / Glanzbrand	400°C	45°C	750°C	4 min	1 min	-	
Correction Powder / Korrektur Pulver	400°C	45°C	720°C	4 min	1 min	450°C	

SHADE GUIDE

APColor Key Shade Guide

**Product Code/
Artikelnummer**

Product Name/ Artikel

Size/Größe

APVOLUTION S LT	APCKLT
APVOLUTION S MT	APCKMT
APVOLUTION S HT & HT +	APCKHT
APVOLUTION S BLEACH	APCKBL
APVOLUTION S STUMP	APCKST

APVolution S Lithium Disilicate 400±60 MPa Color Key Shade Guide

Unlocking the mystery of shade

communication By Jörg Müller, MDT, CEO of
Aesthetic-Press

INTRODUCTION

Taking the right shade for a patient is the everyday challenge for dentists and for technicians. To find the right matching shade, many variables have a big impact on the result. Some of them are the light source, the time of day, the angle at which the light shines onto the tooth, the surface texture and reflection of the tooth, the density of the material and the underlying stump shade or framework for the final restoration.

The standard shade guides offered by the industry up to now, have certain unrealistic features technicians have difficulties to match.

First, the thickness of the shade guide is sometimes 3 to 4 x thicker than the actual crown or veneer restoration. Since color is a result of various layers, shade matching depends on the skillset and artistic work of the technician to recreate the effects, shade, value and chroma. In simple words, the technician has to work his magic to recreate a color in a crown which oftentimes is only 0.6 mm - 1.2 mm thick vs. the shade guide showing a thickness of up to 4 mm.

THE CURRENT PROBLEM

Shade guides are often:

- too thick
- porous surface
- made out of a material the patient case is not going to be made out of
- impossible evaluation of opacity level -
- impossible evaluation of stump shade effect to the desired final shade.

With the current porcelain systems on the market, the technician is not able to see or evaluate how the color of the porcelain restoration will be influenced by the underlying stump shade before starting the case. The stump shade is the most critical aspect in any full porcelain restoration, either a veneer or a full porcelain crown. The stump shade influences the final color and often times has a negative effect on the shade of the crown.

This can only be avoided by choosing a certain material opacity or value to block a dark or grey looking stump. Therefore, most of the shade guides of full porcelain systems on the market offer color indicators which show a porous and unfinished surface. This leads to an unsatisfying evaluation of the true shade and most importantly does not allow different case scenarios with the variety of different stump shades.

SOLUTION

...see the shade before you start the case!!

The design of the AP Color Key Shade guides shows a thickness of 0.9 mm at the incisal edge, about 1.00 mm in the center of the indicator and around 1.3 mm towards the cervical or neck of the tooth. These thicknesses are realistic for a technician to match a shade, as they reflect the average thickness of a crown.

WHY SHOULD DENTISTS USE THE AP COLOR KEY SHADE GUIDE?

The current shades guides on the market only offer a 2 dimensional approach to color and show only chroma and value. In terms of opacity, they are only 1 dimensional.

Shade guides show a color, for example a VITA A2 only in 1 level of opacity, whether the patient is 18 or 80 years old.

The underlying shade can be an A2, yet the depth and the translucency are completely different, an effect the generic shade guides are not able to match.

The APColor Key Shade Guide offers a 4 dimensional approach.

- Color in A, B, C and D Chroma,
- Value in 1, 2, 3 and 4
- Opacity, eg LT, MT, HT and HT+ in combination with the stump shade
- Stump shade in 1-8

- Choose a shade with the material the technician is working with.

- Choose a shade with a realistic thickness of indicator.

- Choose the right kind of indicator in conjunction with the stump shade that comes closest to the patients shade.

ADVANTAGES

- Higher success rate-Less re-do rate
Select matching ingots before the case begins

- Traditional shade guides mostly only offer shade selection in one opacity, missing a big part of the color issue.

Choose matching opacity/translucency:
MT, LT, HT, HT +

- Shade guide manufactured with the porcelain available for the case - ensures a more realistic color selection

- Technicians can select ingots with their eye rather than depending on an app

- Much improved communication between dentist and patient, chairside, true color selection possible.

APColor Key Shade Guide Indicators

Process in pictures

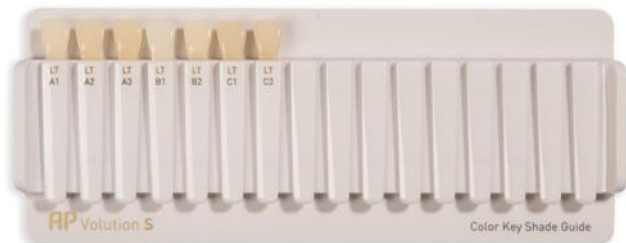
SHADE GUIDE SELECTION

The APVolution S Color Key Shade Guide offers 4 different translucencies/opacities. The technician can choose which ingot is most suitable to match the patient's shade according to their stump shade.

1. Low Translucency or LT

This material shows more opaque porcelain which can cover dark stump shades or certain discolorations.

Indication: In order to get a natural looking crown, the LT ingots can be used for the press/cut back and layering technique. The additional transparent layer will give the crown a life-like appearance, yet the core material will ensure the chroma needed to match the shade.

2. Medium Translucency or MT

This material shows a medium opacity level to cover stump shades with mild discolorations.

Indication: In order to get a natural looking crown, the MT ingots can be used for the press/cut back and layer technique, or press and stain technique. The technique applied will depend on the technician's visual evaluation.

3. High Translucency or HT and HT+

This material shows a high translucency level to cover stump shades with no discoloration.

Indication: In order to get a natural looking crown, the HT ingots can be used for the press and stain technique. The overall appearance will be a natural translucent looking crown.

The HT+ ingots are designed for thin veneers or inlay only restorations offering the maximum translucency.

4. Bleach Shades

The eleven bleach shades are available in different opacities such as LT, MT and HT ingots and offer a wide spectrum to choose from.

Indication: In order to get a natural looking crown, the Bleach ingots can be used in a variety of scenarios for either press & stain, or cut back & layer technique.

The technician can even increase the value if needed by adding a thin layer of transpa white layering powder mixed with a white modifier. The APVolution S&Z Powder selection offers a wide range of powders compatible with the APVolution S ingots.

5. Stump Shades

The stump shades are made in a shape to match the concavity of the APColor Key Shade Guide indicators.

To get the best possible light transmission, a thin film of Vaseline can be applied onto the Stump Shade indicator to ensure the best possible light transmission into the porcelain indicator.

...UNLOCK THE
MYSTERY OF SHADE
COMMUNICATION

INNOVATION:

Innovation: The concavity of the manually finished and polished porcelain indicator in combination with the stump shades offer an excellent way to select the right porcelain shade and opacity before starting the case.

The perfectly finished surface texture of the indicator enhances the reflection and visual result comparable to a high end restoration.

This increases the efficiency and success rate of any full porcelain restoration.

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