SD

Dental Instruments

Cheerful Lab.

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Normally, the carbon in metals can cause rust and brittleness.
The metal used in SD Instruments contains a low amount of carbon
and has a new element added to maintain high hardness.

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The metal has undergone vacuum melting furnace heat treatment,
which removes impurities from the material, refines the structure,
and the material improves toughness.

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The size, angle, and edge surface are precisely adjusted. There is no
discrepancy between the specifications and the actual product.

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
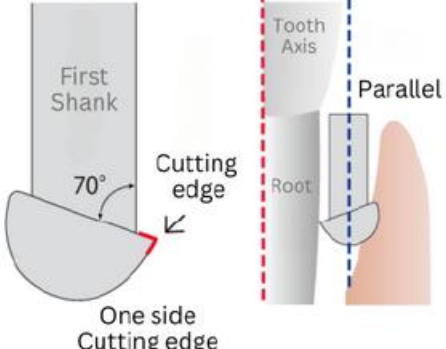
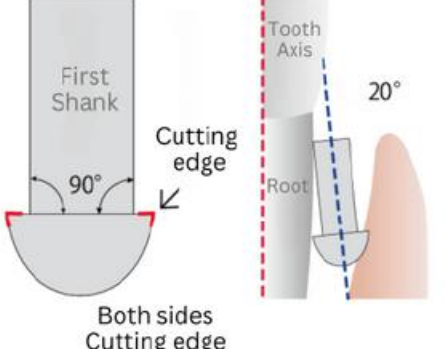
Many instruments are designed by professionals based on their vast experiences.
This is why SD Instruments are easier to use and more practical.
They are continuously updated with new versions.

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
	GRACEY CURETTE	UNIVERSAL CURETTE
Use Position	Specific Position (Tool Number = Position)	All Positions
Blade Angle	70° to the first shank	90° to the first shank
Operation Angle	Parallel to tooth axis	20° to tooth axis
	 First Shank 70° Cutting edge One side Cutting edge Tooth Axis Parallel Root	 First Shank 90° Cutting edge Both sides Cutting edge Tooth Axis 20° Root

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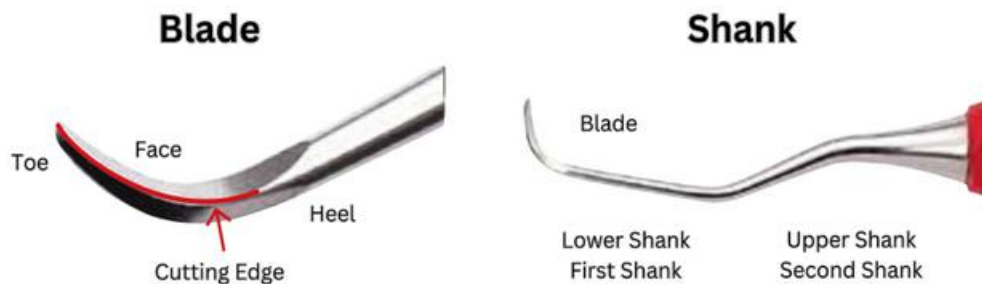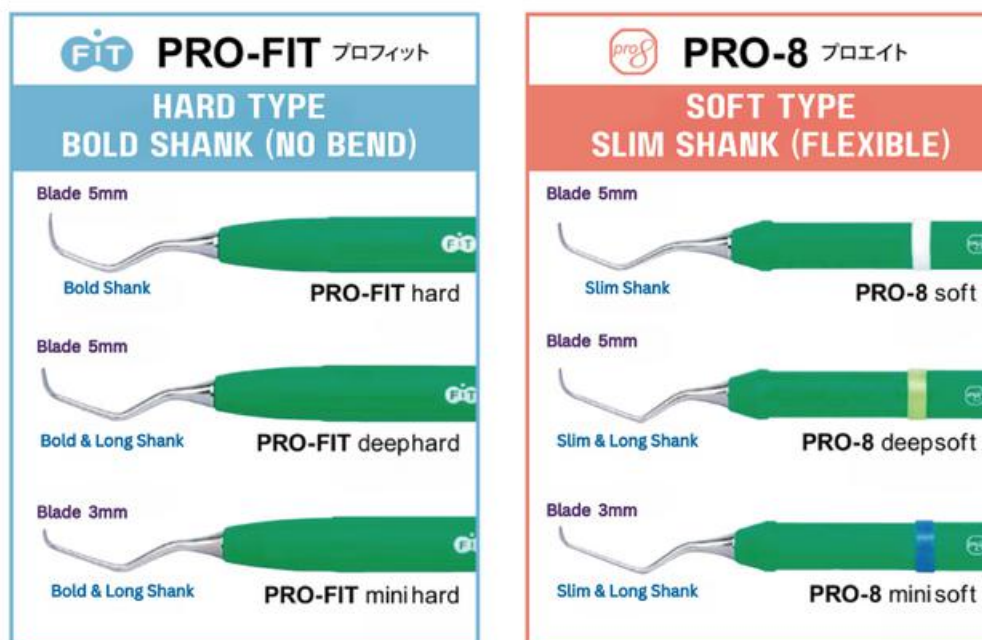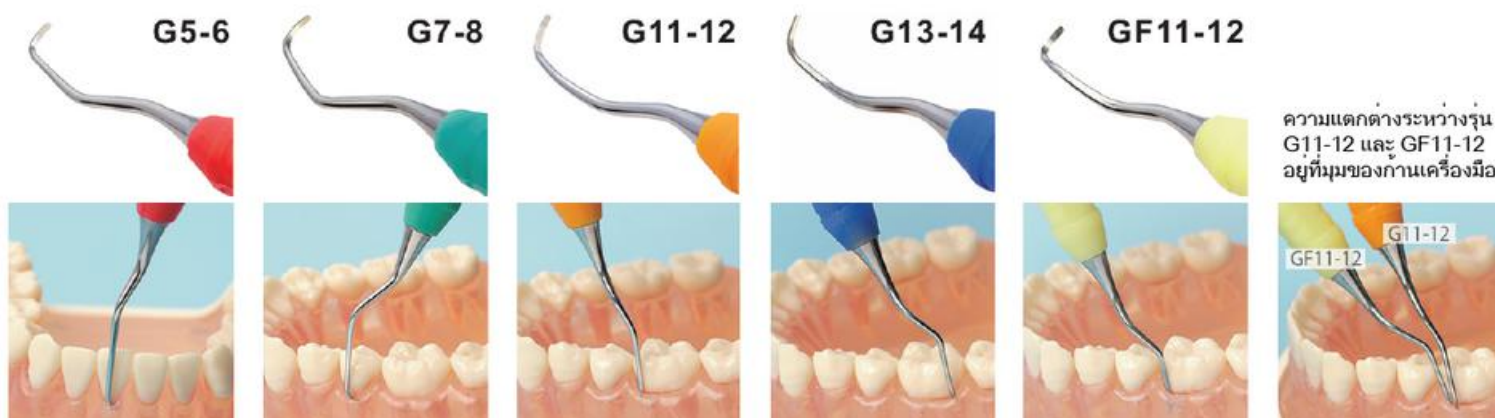
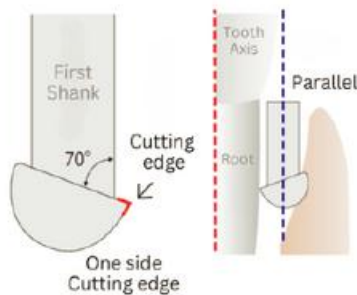
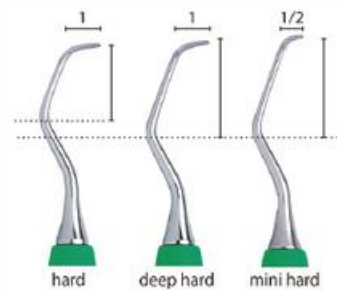
Used for removing large amounts of firmly attached calculus. Ideal for gross scaling.

PRO-FIT hard G5-6	G 5-6
PRO-FIT hard G7-8	G 7-8
PRO-FIT hard G11-12	G 11-12
PRO-FIT hard G13-14	G 13-14
PRO-FIT hard GF 11-12	GF 11-12

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Used for SRP in deep pockets. The shank does not flex, it is ideal for firm SRP.

The shank is longer than the hard type.

PRO-FIT deep hard G5-6	G 5-6
PRO-FIT deep hard G7-8	G 7-8
PRO-FIT deep hard G11-12	G 11-12
PRO-FIT deep hard G13-14	G 13-14
PRO-FIT deep hard GF 11-12	GF 11-12

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Used for SRP in narrow pockets. Ideal for furcation areas and the lingual side of anterior teeth.

The shank is longer than the hard type, and the blade length is about half of it.

PRO-FIT mini hard G5-6	G 5-6
PRO-FIT mini hard G7-8	G 7-8
PRO-FIT mini hard G11-12	G 11-12
PRO-FIT mini hard G13-14	G 13-14
PRO-FIT mini hard GF 11-12	GF 11-12

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PRO-8 is a soft type with a thin and flexible shank.

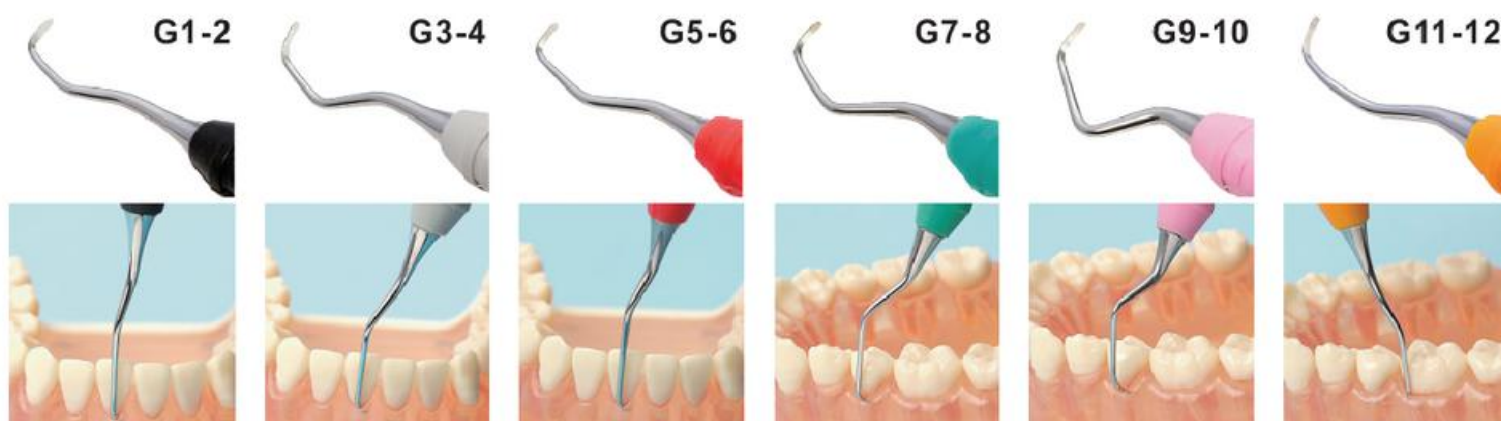
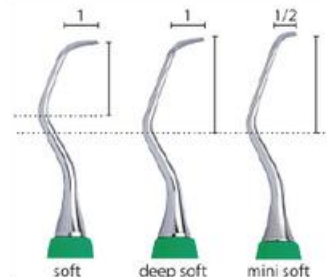
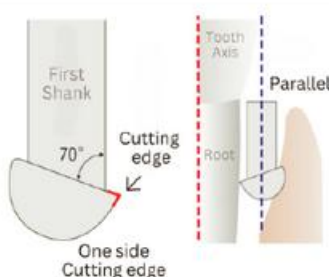
The simple, lightweight octagonal handle has a comfortable thickness that fits well in the hand, allowing delicate tactile feelings to be transmitted to the fingers.

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Model (e.g., G1-2) is identified by handle color.
Type (soft / deep soft / mini soft)
is identified by ring color.

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
Used for SRP in narrow pockets. The small blade minimizes damage to periodontal tissue.
It is ideal for Re-SRP.

PRO-8 G5-6	G 5-6
PRO-8 G7-8	G 7-8
PRO-8 G11-12	G 11-12
PRO-8 G13-14	G 13-14
PRO-8 GF11-12	GF11-12

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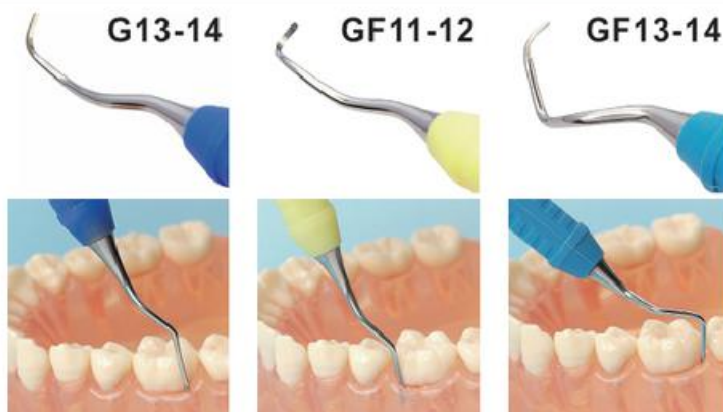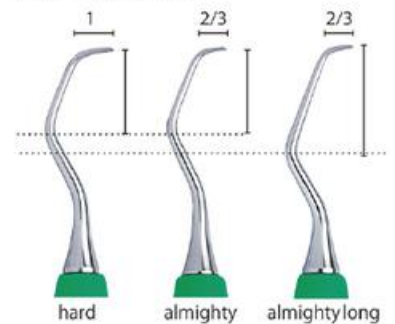
A round and bold handle that provides a stable grip.

Designed with consideration for the morphology of Asian teeth, making it adaptable to all types of cases.
The blade thickness is thin, and the blade length is just compact for versatile use.

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It can be used for all types of cases, from initial treatment to maintenance, with a wide range of applications.

PRO-FIT almighty G5-6	G 5-6
PRO-FIT almighty G7-8	G 7-8
PRO-FIT almighty G11-12	G 11-12
PRO-FIT almighty G13-14	G 13-14
PRO-FIT almighty GF11-12	GF 11-12

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The almighty long has longer shank, it is suitable for deep pockets and molar areas.

PRO-FIT almighty long G5-6	G 5-6
PRO-FIT almighty long G7-8	G 7-8
PRO-FIT almighty long G11-12	G 11-12
PRO-FIT almighty long G13-14	G 13-14
PRO-FIT almighty long GF11-12	GF 11-12

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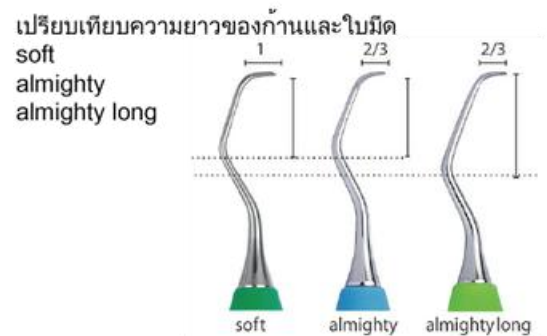
A simple, octagonal handle that fits comfortably in the hand.

Designed with consideration for the morphology of Asian teeth, making it adaptable to all types of cases.
The blade thickness is thin, and the blade length is just right for versatile use.

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Used for all types of cases, from initial treatment to maintenance, with a wide range of applications..

PRO-8 G5-6	G 5-6
PRO-8 G7-8	G 7-8
PRO-8 G11-12	G 11-12
PRO-8 G13-14	G 13-14
PRO-8 GF11-12	GF 11-12

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The almighty long has longer shank. It is suitable for deep pockets and molar areas.

PRO-8 G5-6	G 5-6
PRO-8 G7-8	G 7-8
PRO-8 G11-12	G 11-12
PRO-8 G13-14	G 13-14
PRO-8 GF11-12	GF 11-12

MINT curette scaler

Designed by
DH. Masumi Nagatani (Japan)

Developed a unique curettage technique aimed at improving inflammation without causing gum recession. One of Japan's leading dental hygienists, instructing 700 active dental hygienists annually.

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Mint is developed by expert dental hygienists through extensive experiences

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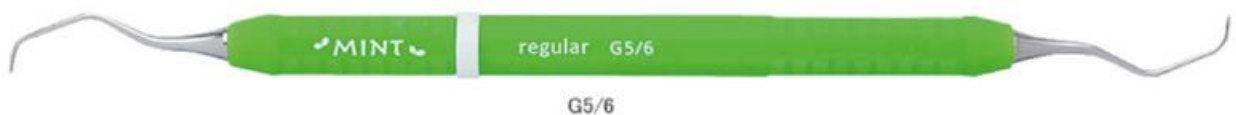
Designed for moderate to severe periodontal disease,
providing access to line angles, small curves on the
buccal/lingual surfaces, and grooves on the proximal surfaces.

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Adapted for the line angles of anterior teeth and the small
curves on the labial and lingual surfaces.

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Adapted for the distal line angles of molars and deep pockets
on the lingual surfaces of molars.

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A multi-curette that can be used on the entire arch,
an excellent item for adapting to root curves.

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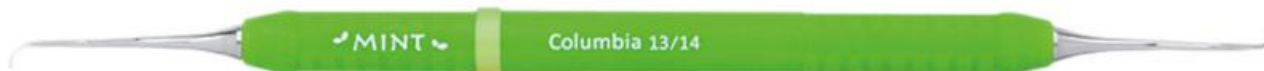
The ultimate item designed for approaching the distal furcation.

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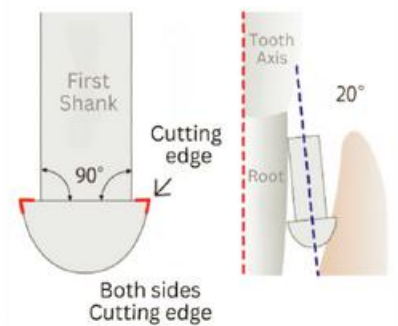
The MK UCS Scaler was developed to ensure "operational continuity and versatility".

This double-ended blade scaler can achieve them by the ergonomic design.

By mastering basic techniques and gaining clinical experience, you can perform scaling on all tooth surfaces with just this one instrument.

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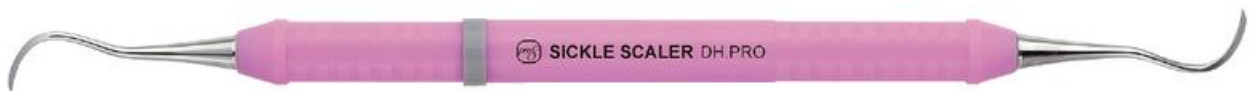
Developed based on clinical field recommendations. The blade has been made thinner for smoother insertion into contact areas. With just one scaler, you can efficiently handle all tooth surfaces.

Designed by DH Kazuko Tsuchiya (Japan)

JACQUETTE SCALER double-ended sickle scaler

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For anterior super marginal area.

One blade is sickle type, and the another is scoop type. It is easy to remove calculus, stain, plaque effectively.

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With the blade set at a 240° angle, it remains effective even during rotational or pulling motions. This allows for minimal removal of softened dentin, preserving as much tooth structure as possible. The vibrations and noise from the turbine or engine ensure a gentle treatment that won't cause discomfort for the patient.

240° Blade

Round 1S (straight) on flat surfaces such as cavity walls and cavity floors.

Ø 1.0mm

1S

Ø 1.2mm

2S

Ø 1.5mm

3S

Ø 2.0mm

4S

Round 1C (curve) in undercut areas and proximal surfaces

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The scaler crushes the calculus to make it easier to use with the curette. The blade is thin and small, allowing it to fit into narrow pockets. Please be sure to use curettes for finishing SRP.

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
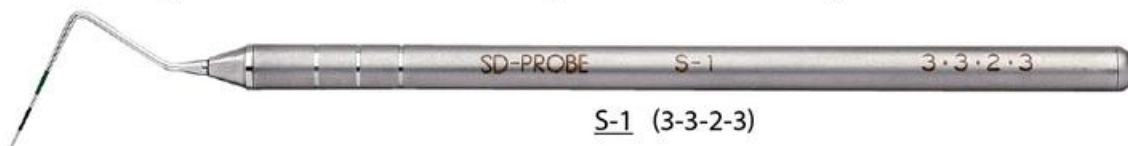
Used for diagnosis such as PPD, BOP, and assessing gum morphology.
The markings are color-coded in black and green for better visibility.

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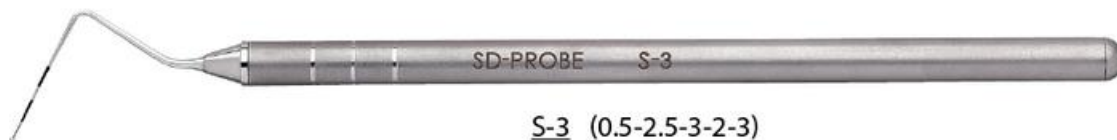
The tip has an ultra-small 0.5mm ball, allowing for gentle operation at the bottom of the pocket.
It is a convenient probe that allows for measurement while detecting the root surface.

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The 1mm interval markings are useful for more precise measurements.

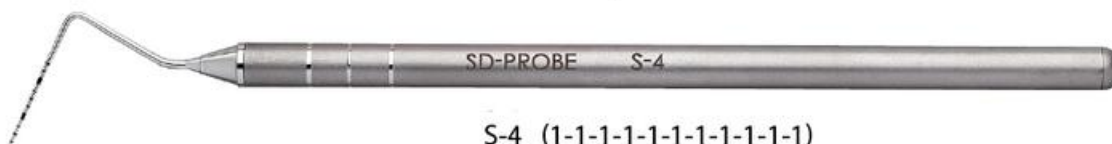This is an essential item for periodontal treatments and periodontal surgery, where high accuracy is required.
The 5mm and 10mm marks are color-coded for easy verification.

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A lightweight probe for root surface detection. The lightweight handle transmits the condition of the root surface sharply to the fingers. Once used, it becomes an indispensable tool for root planing and root surface detection.

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Used for diagnosing furcation defects. It allows for the assessment of the morphology of the furcation, destruction of the periodontal tissue, and the extent of bone loss.

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Used for tactile detection of the root surface and the detection of subgingival calculus.

The narrow shank allows the condition of the root surface to be easily transmitted to the fingers.

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Used for detecting subgingival calculus and tactile examination of the root surface.

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This instrument is designed to facilitate gingival retractions more easily and without causing discomfort to the patient. The thickness of the tool tip has been minimized, and it is easier to rotate and insert the retraction cord.

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Microsurgical Instruments for 3D Root Canal Treatment

A wide variety of instruments to accommodate different root canal anatomies. Available in 20 types.

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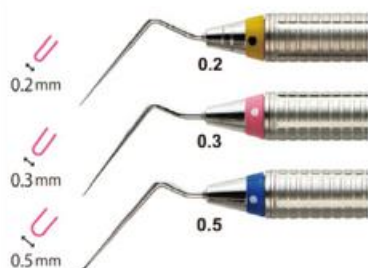
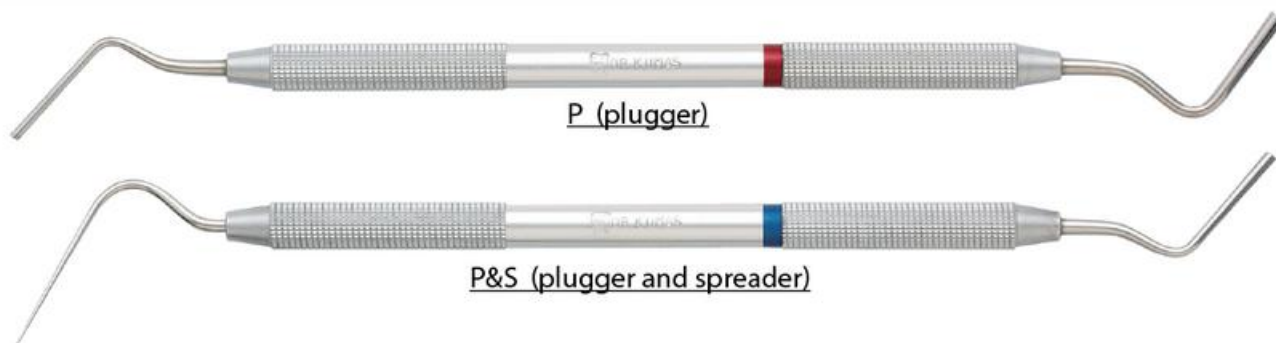
Used for lateral condensation filling. The gutta-percha point is pressed against the root canal wall to create space, and additional points are added to continue filling the root canal.

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The tip is tapered, making it easy to cut gutta-percha points. Even when the tip is heated, the handle stays cool for comfortable use. The Heat Cutter P&S can be used for both lateral condensation and vertical condensation filling.

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This instrument is indispensable for the removal of carious lesions, even in areas that are difficult to access.
 The curve of the shank ensures the reliable removal of carious lesions at the enamel-dentin junction.

Ø 0.7mm

Excavator 0.7 S (straight)

Ø 1.0mm

Excavator 1.0 S (straight)

Ø 0.7mm

Excavator 0.7 C (curve)

Ø 1.0mm

Excavator 1.0 C (curve)

Designed by Prof. Masashi Miyazaki (Nihon University)

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The exquisitely designed curve of the shank enables a cutting feel and accessibility that were previously unattainable with other instruments, significantly improving treatment efficiency

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This instrument provides access to all areas, whether anterior or molar, making it easy to shape resin. It excels in versatility, allowing use on proximal surfaces, margins, or occlusal surfaces (such as cusp ridge formation), and it can be controlled like a resin brush.

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The normal M-M Resin Creator has been upgraded with DLC coating, enhancing the surface hardness of the working area, making it more resistant to scratches. It also excels in smoothness, ensuring better resin release and enabling smooth resin shaping.

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This instrument is excellent for stain application, which was previously difficult for characterization. It is also ideal for spot filling with flow resin.

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This instrument has effectiveness of function and design in anterior tooth fillings. It particularly enhances the control of resin paste on the proximal surfaces

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This instrument improves the operability of various filling procedures in the molar area. It allows for the natural enhancement of cusp ridge formation and makes it easier to restore the anatomical shape of the occlusal surface.

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This instrument allows for the adjustment of the resin filling margin without damaging the tooth structure. With just this one tool, all issues related to margin adjustment can be resolved at once.

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SUNDENTAL CO., LTD. (Product Owner)
CHEERFUL LAB CO., LTD. (Distributor)