



**1** Discard 1ml of calcium chloride solution

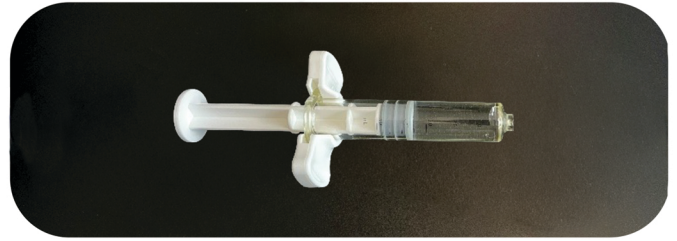

**2** Join the two syringes together



**3** Push the liquid into the powder, never powder first

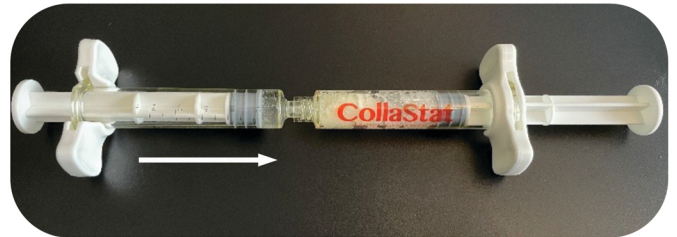

**4** Perform a total of 25 passes at a fast pace  
*To create a higher viscosity, increase the number of passes*



**5** Leave the solution in the syringe labelled Collastat, apply applicator & allow to settle for 30 seconds prior to use



**Scan the QR code to see a video of this process and access further information**

Rev 6 09/12/25



01782 637009  
sales@deltasurgical.co.uk



**Delta Surgical Ltd.**

Unit 10, Evolution,  
Lymedale Business Park,  
Newcastle-under-Lyme,  
Staffordshire, ST5 9QF