


**1** Remove syringe end caps and join syringes together



**2** Push the liquid into the powder, never powder first

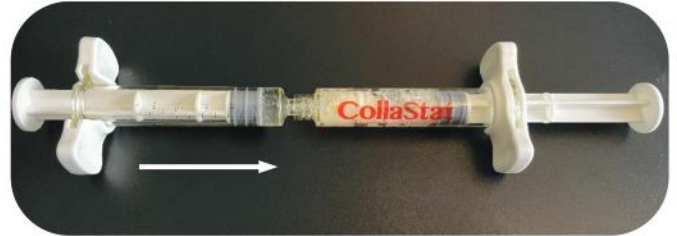

**3** Perform a total of 25 passes at a fast pace  
*To create a higher viscosity, increase the number of passes*



**4** Undo syringes and test a drop. If a thicker consistency is needed rejoin syringes and perform a few more passes.



**5** Leave solution in syringe labelled Collastat and apply applicator



Rev 7 30/03/26



01782 637009

sales@deltasurgical.co.uk



**Delta Surgical Ltd.**

Unit 10, Evolution,  
Lyndale Business Park,  
Newcastle-under-Lyme,  
Staffordshire, ST5 9QF

Shane Scott  
Surgical Sales - Territory Manager - South

Mobile +44 (0) 7395 792 803

Office +44 (0) 1782 637 009

Fax +44 (0) 1782 620 882

Email shane@deltasurgical.co.uk