



---THE BUNK POLICE---

# SIMON'S

TEST-KIT-GUIDE

## INSTRUCTIONS

1. PUT YOUR SAMPLE IN THE TEST TUBE
2. ADD 1-2 DROPS OF LIQUID FROM THE BOTTLE
3. COMPARE COLOR WITH REACTIONS IN THIS BOOKLET
4. THOROUGHLY RINSE THE TEST TUBE WITH WATER
5. DRY THE TEST TUBE BEFORE RE-USE

## SAMPLE SIZE

**PILL:** SCRAPE OFF A SMALL AMOUNT - AROUND 1/20TH OF A PILL

**POWDER:** USE 5 MILLIGRAMS - ABOUT THE SIZE OF A DROP OF WATER


**PAPER:** CUT OFF A CORNER - APPROXIMATELY 1/10TH OF A BLOTTER

**LIQUID:** DROP A SMALL AMOUNT ON TO A PIECE OF UNTREATED PAPER AND ALLOW IT TO DRY BEFORE TESTING


**WARNING!** THE TESTING LIQUID IS A STRONG ACID  
DO NOT GET IT ON YOUR SKIN OR IN YOUR EYES - DO NOT INGEST  
USE IN A WELL VENTILATED AREA AND WEAR GLOVES & EYE PROTECTION  
IF POSSIBLE. IF THE BOTTLE BECOMES BLOCKED, DO NOT ATTEMPT TO  
CLEAR BY SQUEEZING. INSTEAD CLEAR THE BLOCKAGE BY SHAKING  
VIGOROUSLY OR INSERTING A PAPERCLIP / SIMILAR OBJECT.

## KEEP OUT OF HEAT/SUNLIGHT

DO NOT REMOVE ABSORBENT POWDER FROM CONTAINER

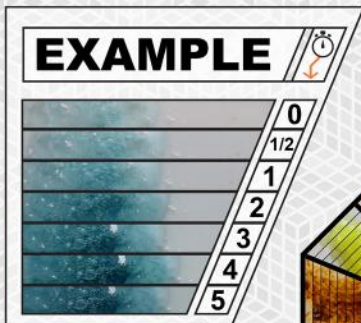

TIME (SECONDS)

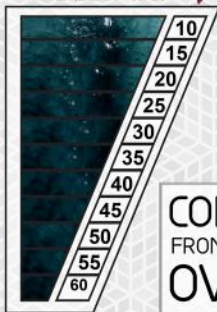

COMPARE YOUR REACTION  
FROM THE OUTSIDE TO THE INSIDE OF THE CHART  
OVER SIXTY SECONDS

## USING THE ADVANCED SECTION

AS YOU FLIP THROUGH THE FOLLOWING SECTION, YOU'LL SEE WHAT WE CALL "**PRESENCE CHARTS**" ON THE *LEFTHAND* PAGES AND ROUND "**COLOR SPECTRUMS**" ON THE *RIGHTHAND* PAGES.

**COLOR SPECTRUMS** - THESE ROUND "WHEELS" EACH CONTAIN THE COLOR REACTIONS FROM 16 DIFFERENT (LAB VERIFIED) SUBSTANCES. THE REACTIONS START AT THE OUTSIDE OF THE SPECTRUM (ZERO SECONDS) AND MOVE OVER TIME TO THE CENTER OF THE SPECTRUM (60 SECONDS.) FOR MORE INFO ABOUT READING COLOR SPECTRUMS PLEASE SEE THE **EXAMPLE ON PAGE ONE** AS WELL AS THE **TIPS & TRICKS** SECTION TOWARDS THE BACK OF THIS BOOKLET.

**PRESENCE CHARTS** - THESE CHARTS CAN TELL YOU WHICH OF THE RCs / CUTTING AGENTS ARE LIKELY TO BE PRESENT IN THE SUBSTANCE YOU'RE CURRENTLY TESTING. SIMPLY PUT, PRESENCE CHARTS TELL YOU WHICH REACTIONS YOU CAN EASILY RULE OUT. FOR EXAMPLE, YOU WOULD NEVER FIND DMT IN AN MDMA SAMPLE, SO WHY WASTE TIME LOOKING FOR IT?










(EXAMPLE CHART AND SYMBOL KEY CAN BE FOUND ON THE FOLLOWING PAGE)

AT THE TOP RIGHT CORNER OF THE CHART ① YOU'LL FIND A ROW OF SYMBOLS - EACH REPRESENTING A COMMONLY TESTED SUBSTANCE. TO USE THE CHART, FIRST FIND THE SYMBOL THAT REPRESENTS THE SUBSTANCE YOU'RE TESTING. FOLLOW THE LINE THAT RUNS DOWN FROM THAT SYMBOL.

RED "X" MARKS ON THIS LINE INDICATE THAT THE RC / CUTTING AGENT LISTED TO THE LEFT OF THE MARK ② HAS A HIGH LIKELYHOOD OF BEING PRESENT IN THE SUBSTANCE YOU'RE TESTING. SUBSTANCES WITH GREY "Ø" SYMBOLS ARE NOT OF CONCERN WHILE TESTING THIS PARTICULAR SUBSTANCE AND CAN BE DISREGARDED.

## PRESENCE CHART - EXAMPLE

↓ 1

↓ SUBSTANCE NAME ↓									
4-FA	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input checked="" type="radio"/>	<input checked="" type="radio"/>	<input checked="" type="radio"/>	<input checked="" type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>
5-MEO-DMT	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
25i-NBOME	<input type="radio"/>	<input checked="" type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
CODEINE	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input checked="" type="radio"/>
METHYLONE	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>

↑ 2

## SUBSTANCE SYMBOL KEY



DMT



MESCALINE



LSD



MUSHROOMS



MDMA



KETAMINE



AMPHETAMINES



COCAINE



OPIATES



NOT AN ADULTERANT

**SIMON'S**

MDMA


MDA

MESCALINE

CLEAR  
REACTION  
NO COLOR  
CHANGE

DMT

END

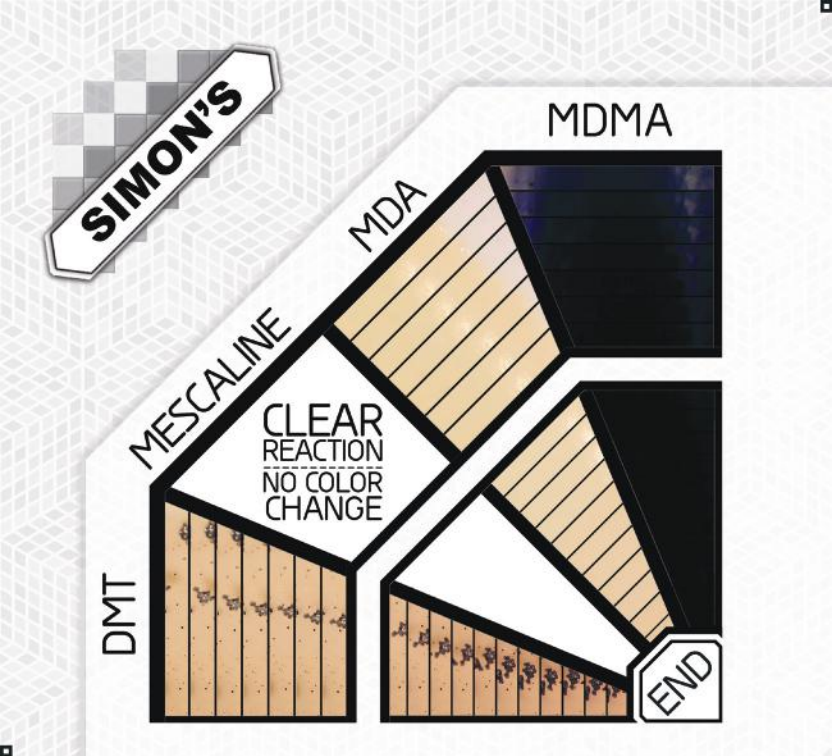

KETAMINE

AMPHETAMINE

**SIMON'S**

COCAINE

HEROIN

END



↓ SUBSTANCE NAME ↓



2C-BZ



MEXEDRONE



4-HO-DET



4-AcO-DALT



LSD Drop



LSD Blotter



LIDOCAINE



KETAMINE



TETRACAINE



MXE



PROCAINE



25N-NBOMe



a-PVP



2C-iP




4-FA



3-MeO-PCP




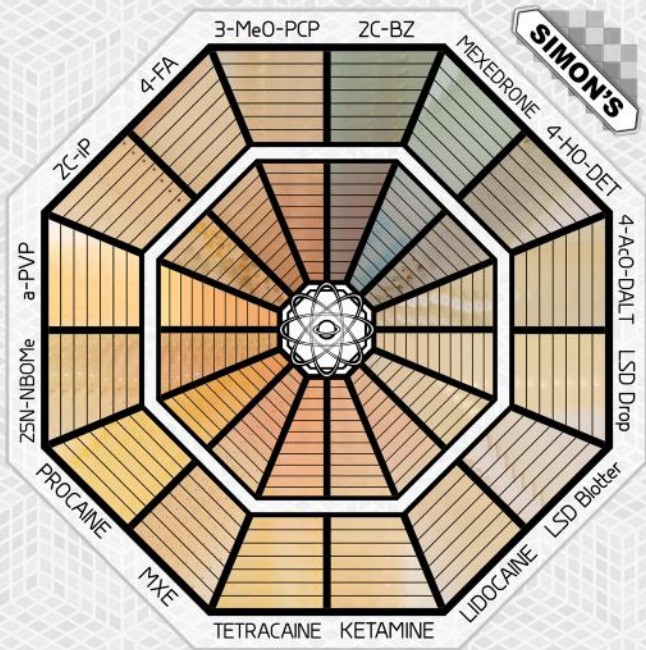

↓ SUBSTANCE NAME ↓



MDPV	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
25iP-NBOMe	<input type="radio"/>	<input checked="" type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
CAFFEINE	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input checked="" type="radio"/>	<input checked="" type="radio"/>	<input checked="" type="radio"/>	<input checked="" type="radio"/>
MXP	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
PENTYLONE	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
25I-NBOMe	<input type="radio"/>	<input checked="" type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
OXYCODONE	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
BENZOCAINE	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>
DPT	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
MDA	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
25D-NBOMe	<input type="radio"/>	<input checked="" type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
AMT	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
4-FA	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input checked="" type="radio"/>	<input checked="" type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>
25B-NBOMe	<input type="radio"/>	<input checked="" type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
-	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
2C-B	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>

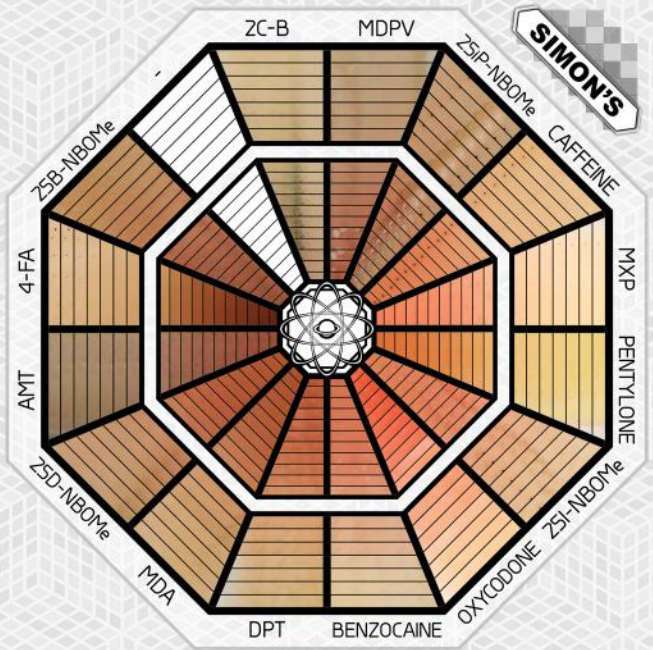

↓ SUBSTANCE NAME ↓



5-MeO-MiPT



2C-C



DOB



2C-B-FLY



a-PVT



DOiPR



6-APDB



2-MMC



ETHYLONE



3-CMC



MEPHENMETRAZINE



MEPHEDRONE



PHENETRAZINE



ETHCATHINONE



5-DBFPV



METHEDRONE






↓ SUBSTANCE NAME ↓




BUTYLONE	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
METHYLONE	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
BUPHEDRONE	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
PMEA	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
PMMA	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
2-FMA	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input checked="" type="radio"/>	<input checked="" type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>
METHAMPHETAMINE	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input checked="" type="radio"/>	<input checked="" type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>
MPA	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input checked="" type="radio"/>	<input checked="" type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>
4-FMA	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input checked="" type="radio"/>	<input checked="" type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>
PENTEDRONE	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>
4-MEC	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>
2C-T-7	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
METHYLPHENIDATE	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input checked="" type="radio"/>	<input checked="" type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>
3-FPM	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
MDMA	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
5-MAPB	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input checked="" type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>

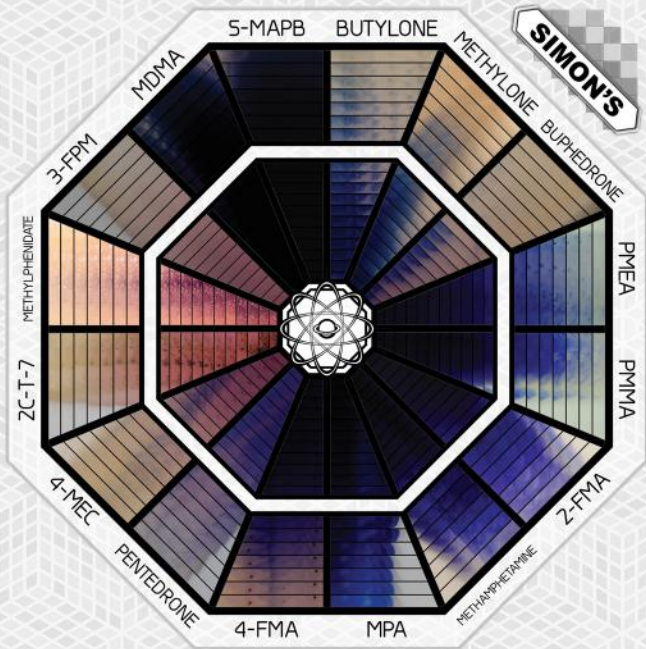

↓ SUBSTANCE NAME ↓



a-PPP					X		X	X	
3-FMC									
TMA-6		X							
HEROIN									
a-PHP					X		X	X	
Z5E-NBOMe		X	X						
5-APDB					X				
PMA					X				
5-EAPB					X				
ZC-I		X			X				
ADDERALL					X		X	X	
3C-E		X							
6-APB SUCCANATE					X				
3,4-CTMP									
6-APB					X				
ESCALINE HCl		X							

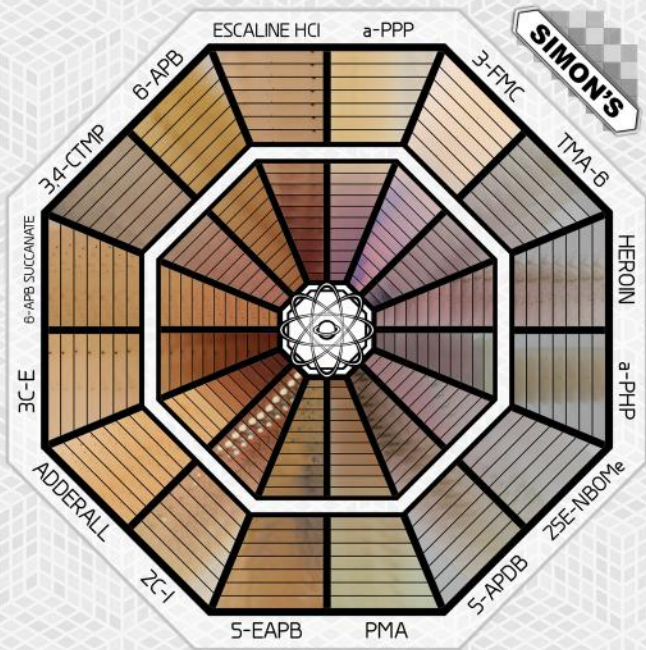

↓ SUBSTANCE NAME ↓



3C-P



MDAI



2C-P



DOC



2C-D



2C-E



HDMP-28



3-MeO-PCE



-



-



-



25C-NBOMe



DMT



5-MeO-AMT




5-APB



5-MeO-DMT



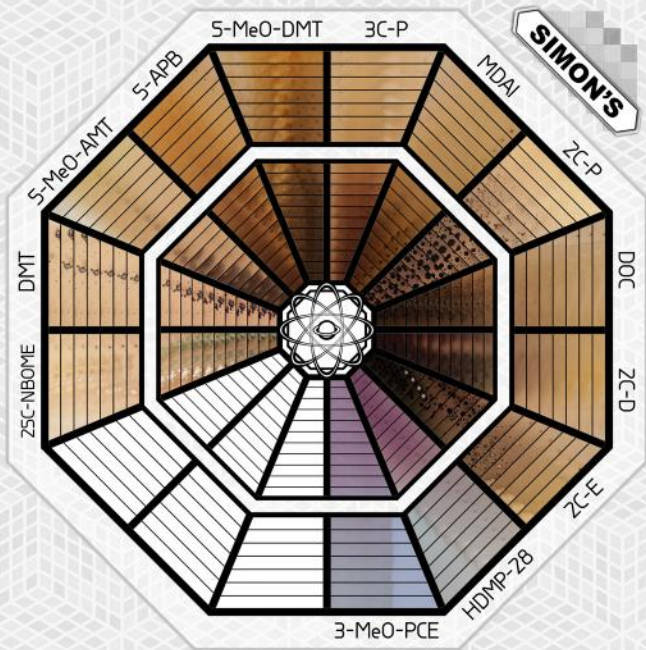

USE THESE KITS TO TEST  
AMPHETAMINE

MARQUIS

COLOR  
MATCHES  
SPECTRUM

MECKE

ANY  
OTHER  
COLOR

NOT  
AMPHETAMINE

COLOR  
MATCHES  
SPECTRUM

SIMON'S

ANY  
OTHER  
COLOR


NOT  
AMPHETAMINE

ZERO  
REACTION

AMPHETAMINE

BRIGHT  
BLUE

METHAMPHETAMINE

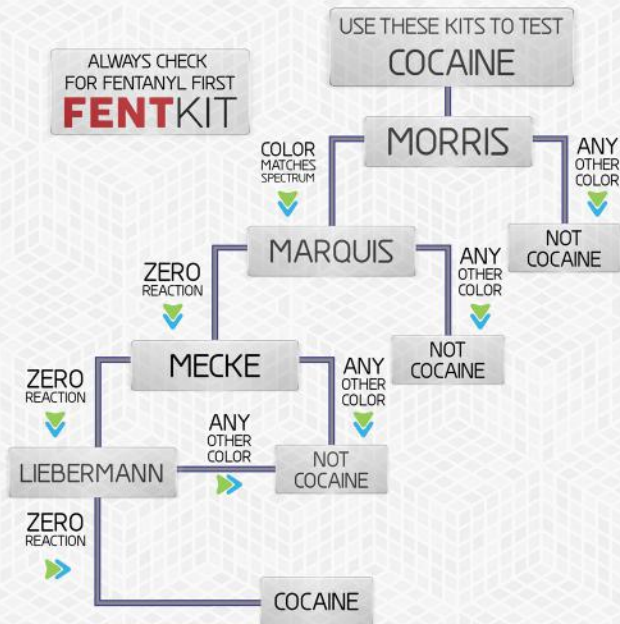

USE THESE KITS TO TEST  
DMT

HOFMANN

COLOR  
MATCHES  
SPECTRUM



ANY  
OTHER  
COLOR



MECKE

NOT  
DMT

COLOR  
MATCHES  
SPECTRUM




ANY  
OTHER  
COLOR




DMT

NOT  
DMT

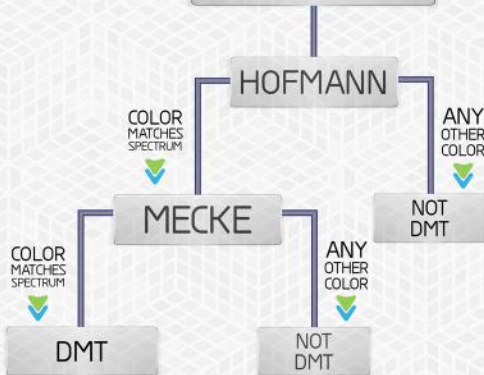

ALWAYS CHECK  
FOR FENTANYL FIRST  
**FENT**KIT

USE THESE KITS TO TEST  
**HEROIN & PAIN PILLS**

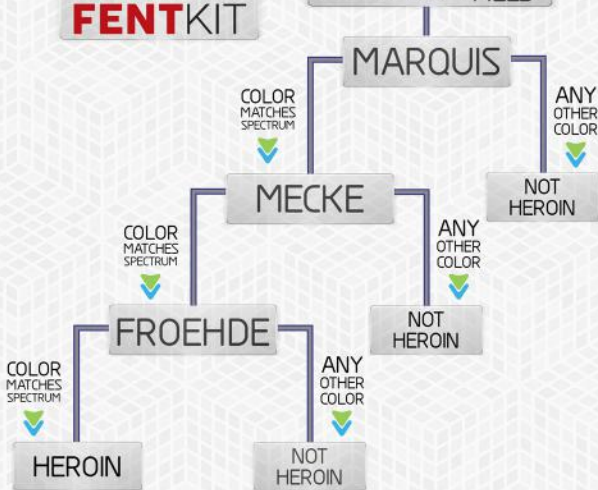

USE THESE KITS TO TEST  
KETAMINE

MORRIS

COLOR  
MATCHES  
SPECTRUM



ANY  
OTHER  
COLOR



NOT  
KETAMINE

MECKE

ZERO  
REACTION



ANY  
OTHER  
COLOR



NOT  
KETAMINE

LIEBERMANN

COLOR  
MATCHES  
SPECTRUM



ANY  
OTHER  
COLOR



ANY  
OTHER  
COLOR




NOT  
KETAMINE

FROEHDE

ZERO  
REACTION



KETAMINE

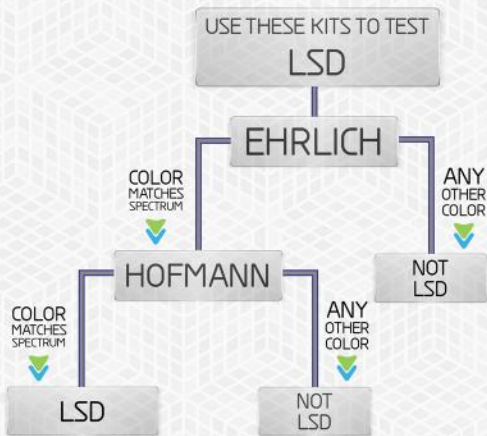

USE THESE KITS TO TEST  
MDMA

MARQUIS

COLOR  
MATCHES  
SPECTRUM

MECKE

ANY  
OTHER  
COLOR

NOT MDMA

COLOR  
MATCHES  
SPECTRUM

SIMON'S

ANY  
OTHER  
COLOR


NOT MDMA


BRIGHT  
BLUE

MDMA

ZERO  
REACTION

MDA

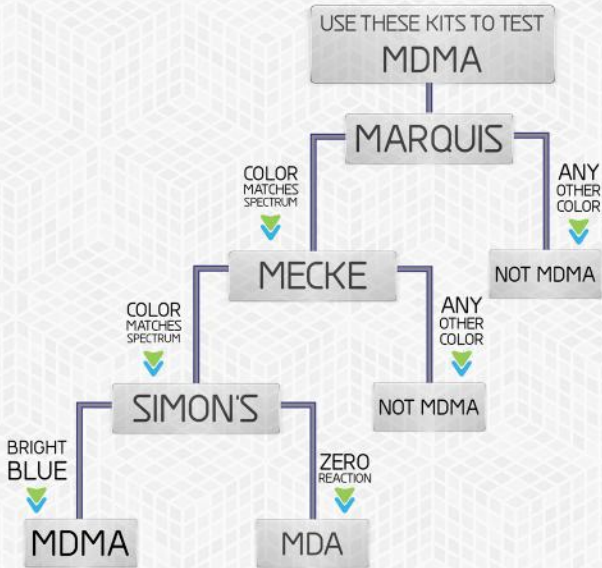

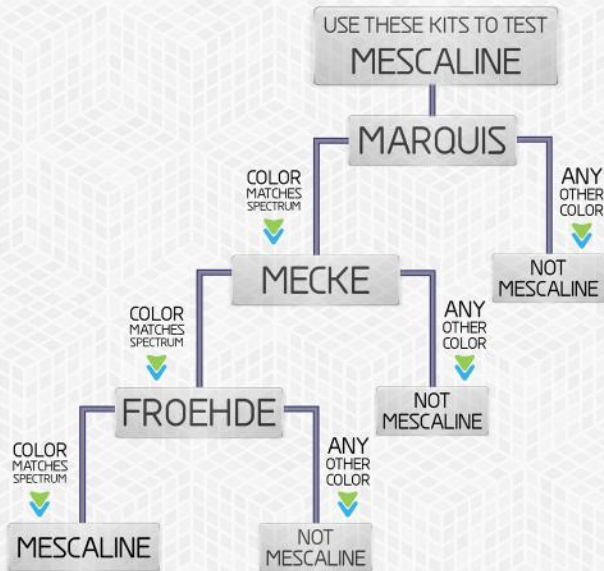

USE THESE KITS TO TEST  
METHAMPHETAMINE

MARQUIS

COLOR  
MATCHES  
SPECTRUM



ANY  
OTHER  
COLOR



MECKE

COLOR  
MATCHES  
SPECTRUM



ANY  
OTHER  
COLOR



NOT  
METHAMPHETAMINE

SIMON'S

NOT  
METHAMPHETAMINE

BRIGHT  
BLUE



METHAMPHETAMINE

ZERO  
REACTION



AMPHETAMINE

## TIPS & TRICKS

1. USE MORE THAN ONE TYPE OF KIT FOR INCREASED ACCURACY
2. MAKE SURE TO CONDUCT TESTS IN A WELL LIT AREA
3. CRUSH YOUR SAMPLE INTO A POWDER BEFORE TESTING
4. THE FIRST 1 - 3 SECONDS OF THE REACTION ARE THE MOST IMPORTANT - PAY CLOSE ATTENTION DURING THIS TIME OR RECORD THE REACTION ON YOUR PHONE SO YOU CAN REPLAY AND ANALYZE IT AS MANY TIMES AS YOU LIKE.
5. THERE ARE MANY SUBSTANCES THAT DO NOT CHANGE COLOR OR REACT IN ANY WAY. **SOME OF OUR KITS TAKE ADVANTAGE OF THIS, ALLOWING YOU TO SEE THE REACTIONS OF UNWANTED SUBSTANCES (ADULTERANTS) THAT MIGHT BE IN YOUR SAMPLE ALONG WITH THE DESIRED SUBSTANCE.**
6. OUR REACTION CHARTS ARE MADE USING TIME-LAPSE PHOTOS OF VERIFIED PURE SAMPLES AS THEY REACT. BECAUSE YOU'RE LOOKING AT REAL PHOTOS OF A REACTION, YOU CAN USE OTHER INFORMATION BEYOND JUST COLOR AND TIMING TO IDENTIFY YOUR SUBSTANCE. DID THE POWDER DISSOLVE QUICKLY OR SLOWLY? WERE THERE LARGE BUBBLES OR SMALL RAPID ONES? ETC..

**THE VIDEOS USED TO MAKE OUR CHARTS ARE AVAILABLE ON OUR VIMEO / ANDROID APP / AND BUNKPOLICE.COM**

# SAFETY & STORAGE

**WARNING!** THE TESTING LIQUID IS A STRONG ACID

AVOID SKIN AND EYE CONTACT - DO NOT INGEST

CONDUCT TESTS IN A WELL VENTILATED AREA

USE GLOVES IF POSSIBLE & WASH HANDS AFTER USE

IF THE BOTTLE BECOMES BLOCKED, DO NOT ATTEMPT TO CLEAR IT BY SQUEEZING. INSTEAD CLEAR THE BLOCKAGE BY SHAKING THE BOTTLE VIGOROUSLY OR INSERTING A PAPERCLIP OR SIMILAR OBJECT IF NECESSARY.

KEEP REFRIGERATED - FROZEN IF POSSIBLE  
(IF USED OUTDOORS KEEP SHADED OR IN A COOLER)

SHELF LIFE DEPENDS ON STORAGE CONDITIONS  
IF KEPT REFRIGERATED YOUR KIT CAN LAST SIX MONTHS TO A YEAR  
(WE DO NOT RECOMMEND USING KITS OVER ONE YEAR OLD)





...QUESTION EVERYTHING...  
@BUNKPOLICE / BUNKPOLICE.COM