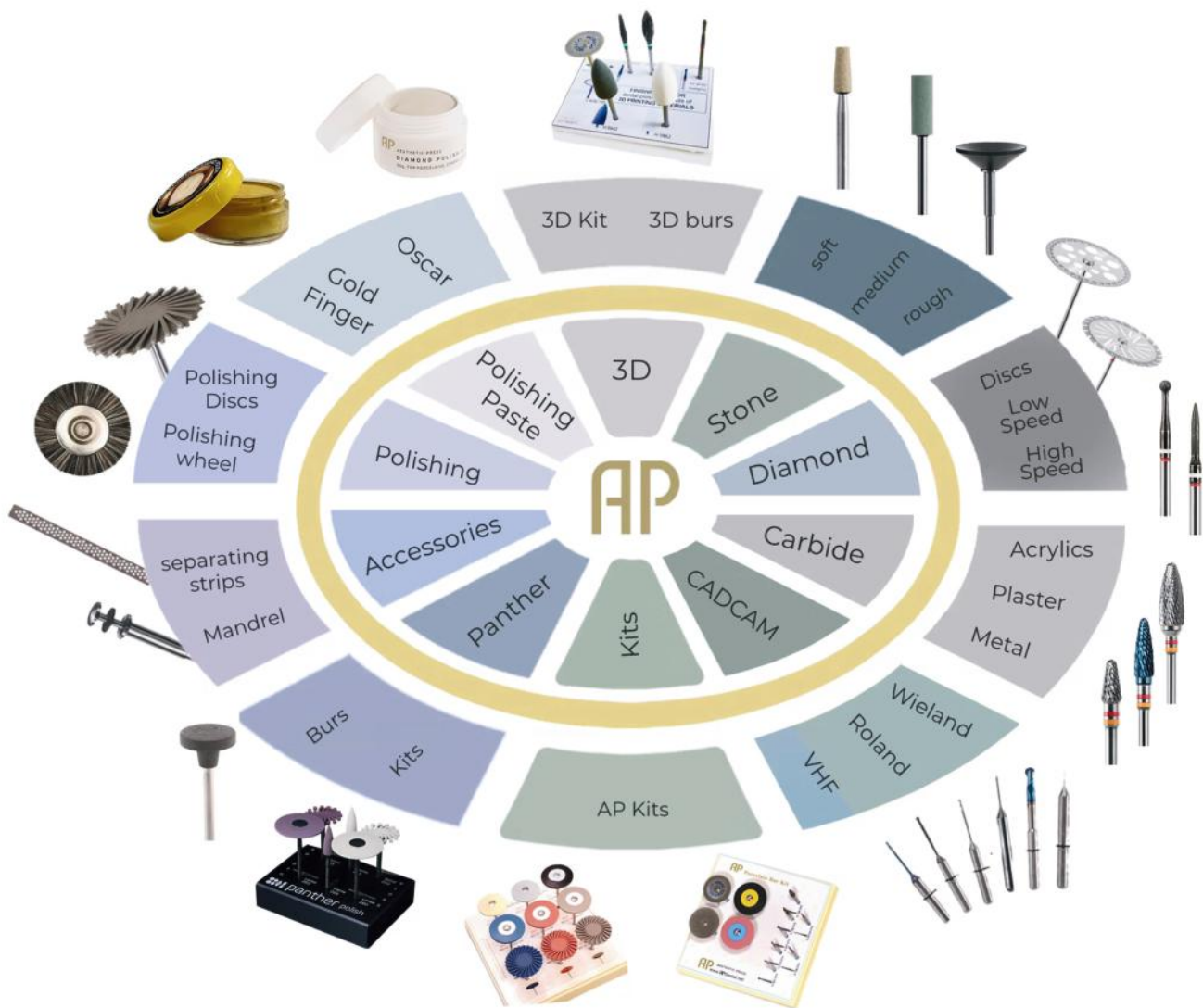
AP Rotary Systems

Complete your workflow

Rotating instruments for Dental Laboratory

As Aesthetic Press Dental's founder, I launched this game-changer in 2006 in San Francisco, fueled by 17 years running labs in San Francisco and Düsseldorf. Our Porcelain Pressable Systems and materials for metal, zirconia, and Lithium Disilicate and our famous APVanilla Wax blank—perfected through relentless R&D and thousands of restorations—deliver jaw-dropping efficiency and simplicity, outshining competitors. We strip complexity, sparking innovative, user-friendly solutions.

With decades of hands-on expertise, we craft cutting-edge tools for technicians and dentists, transforming workflows and aesthetics with rock-solid results. As an MDT and innovator, I personally back every product with unmatched tech support and in-house seminars. Visit **www.apdental.net** to ignite your mastery and redefine dental excellence with us!

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AP AESTHETIC PRESS
EFFICIENCY THROUGH SIMPLICITY

Jörg Müller
MDT, CEO and Founder

4 - 21	TUNGSTEN CARBIDE INSTRUMENTS HARTMETALL-INSTRUMENTE
21 - 26	CAD CAM/ TOOLS FOR MILLING CADCAM WERKZEUGE FÜR DIE FRÄSTECHNIK
27	WAX WACHS
28 - 36	DIAMOND COATED INSTRUMENTS DIAMANT-INSTRUMENTE
28 - 32	ZIRCON OXIDE ABRASIVES ZIRKONOXID-SCHLEIFKÖRPER
36	3D PRINT INSTRUMENTS 3D DRUCK - INSTRUMENTE
37 - 40	DISCS SCHEIBEN
47	POLISHING PASTE POLIERPASTEN
41	SINTERED DIAMONDS SINTERDIAMANT-INSTRUMENTE
42 - 43 & 48 - 51	STONE BURS STEINBOHRER
50 - 59	POLISHING · ABRASIVES · MANDRELS POLIEREN · STEINE · MANDRELLE
55	SEPARATING DISCS TRENNSCHEIBEN
60 - 62	PANTHER PANTHER
63	AP BUR KITS AP SCHLEIFKÖRPER KITS

is the further development of the successful NEM-toothing Denture

- Especially efficient, effortless and controllable working
- New crosscut: fine chip breaker as closed ellipses
- Group tooththing with up to 9 groups – coarse and fine tooththings alternate per turn
- Unmatched cutting performance with smooth surfaces at the same time
- Easy penetration behaviour into the material – for a safe working method
- Very fine and grainy chips – do not pierce the skin

Application: Metal cutting, especially viscose non-precious alloys, like cnc-milled or laser sintered structures

Universal cutter for all metals

- Two different cuts (fine and coarse on one tool)
- Fast cutting even with the hardest metals
- Right-wound transversal cut for very short chips, this avoids unpleasant needle shaped chips

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Ø mm / L

Characteristic: extremely high durability, easy cleaning and longer service life, reduced heat development

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
NEM COMBINATION TOOTHING

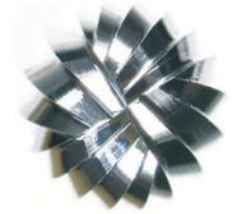fine and coarse, pre-, and finest grinding of non-precious alloys

- Two different cuts (coarse and fine on one tool)
- Two chip-breaker grooves for fine- and coarse-grained abrasion
- Blades penetrate easily and quickly even into the hardest non-precious metals and CrCo alloys

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DOUBLE CUTS

supercoarse for plaster and acrylics

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DOUBLE CUT coarse for plaster and acrylics

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AP CROSS CUT

KGF (ROUGH CROSS CUT) & KNF

- Cutting edge with roughened chamfer
- Controlled working by reduced roughing effect
- Improved surface characteristics for less rework
- Smooth running

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DOUBLE CUT MEDIUM

for CrCo, precious alloys and solid acrylics

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
DIAMOND CUT EXTRA FINE for ceramics and all alloys

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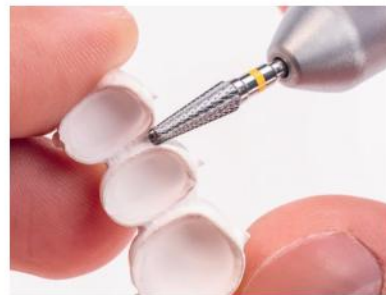● Fein · Fine

● Extrafein · Extra fine

STAGGERED TOOTHING for left-handed

- Inverted blade geometry
- Without changing hands, what avoids the risk of injury
- Facilitates working for left-handed users
- Distinguishing feature: additional white colour ring

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SGF TWISTED CUTS SUPERCOARSE

with guide phase and s-shaped central cutting edge for plaster

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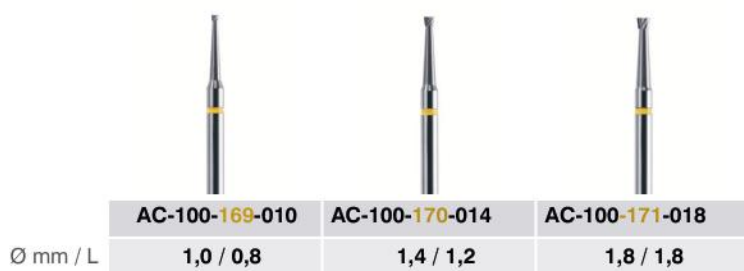
TWISTED CUT FINE

for fissures, all precious alloys and ceramics

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LGQ LAMELLAS TOOTHING CROSS CUT

Lamellas toothings coarse with cross-cut, quick removal of synthetic materials, smooth surfaces, very high abrasion performance highly suitable for 3D-printing synthetics

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AP

TWISTED CUTS FINE CROSS CUT
for gold-, Pd-alloy, non-precious alloys
and acrylics

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- All diameters have 8 blades for short grainy chips
- Easy and quick penetration even into the hardest alloys

Recommendations:

Optimum Speed: 20.000 – 25.000 min⁻¹ with light contact pressure.

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Ø mm / L

CARBIDE BURS HARTMETALL BOHRER

through a customized process, a particularly hard material is melted under heat into the metal which means advantages such as:

- the bur is custom-built with hardened surface
- a slower speed of rotation is possible
- the bur has longer durability
- the surface that is being polished is smoother
- reduced deterioration to the bur

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TWIST CUT TO THE RIGHT AND CROSS CUT for standard applications

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0,9 / 4,0

1,0 / 4,0

Twisted cut, to the right with fine cross cut

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AC-100-270-010

PIN DRILL

tapered, compatible with Giroform, Zeiser et.

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Schaft
Shank

WIELAND ZENOTEC

burs are precision milling tools for Zenotec Select CAD/CAM systems.

Available in uncoated, CVD Diamond, and nanocomposite coating variants, they ensure accurate, efficient milling of dental restorations like crowns and bridges from materials such as zirconium oxide and ceramics, delivering smooth, high-quality results.

Wieland Zenotec Mini AC Blue

Wieland Zenotec Select ZRO-P AC Crystal

Wieland Zenotec Select Black Diamond Coated

Wieland Zenotec Select Universal Blue

Wieland Zenotec Select Magic Black Premium Coated

Wieland Zenotec Select Universal

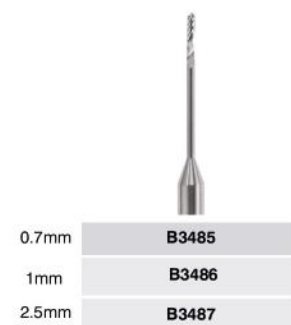
Wieland Zenotech Universal Mini

Wieland Zenotec Mini ZRO Premium AC Crystal

Wieland Zenotec Mini PMMA - Wax

ROLAND

CAD/CAM milling burs, designed for DWX series mills (DWX-4, DWX-50, DWX-51, DWX-52), come in various coatings for durability and mill materials like zirconia, PMMA, and glass ceramics. They ensure high-quality restorations with excellent precision.

Roland Black Diamond Bur

diamond CVD coated

Roland Rough Anatomy

PMMA

Roland Rough Anatomy

Universal

Roland Deep Anatomy

PMMA

Roland deep anatomy

Universal

for plastics, composite and wax
für Kunststoffe, Composite, Wachs.

VHF CAD/CAM burs from VHF Camfacture AG in Germany

are designed for precise milling of zirconia, PMMA, and wax. With durable coatings like diamond or TiSiN, they ensure long-lasting performance in VHF machines like the K5 and S2.

VHF Universal Carbide

VHF BLUE

VHF Black Diamond coated

1. STEP ●

1. SCHRITT

- Form milling, roughing and finest milling produce a very smooth surface
- Very long service life

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AC-100-282-012

AC-100-284-025

- Very long and thin
- For special demands
- Use only low contact pressure

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AC-100-290-012

TUNGSTEN CARBIDE cone cutter 1° KONUSFRÄSER 1°

						
Shank						
Schaft 2,35 mm	AC-100-291-016	AC-100-293-029	AC-100-295-016	AC-100-297-029	AC-100-299-016	AC-100-301-029
Schaft 3,0 mm	AC-100-292-016	AC-100-294-029	AC-100-296-016	AC-100-298-029	AC-100-300-016	AC-100-302-029

AP

RQF CUTTER – FINE SPIRAL TOOTHING with fine cross cut Pre- and fine milling of non-precious alloys and gold alloys with one cutter

- By means of speed and contact pressure, the intended characteristic can be realized
- Fast cutting and smooth running
- Preferable to use milling model
Depending on the material / alloy mostly in the range 20.000–25.000 min⁻¹

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XNE / FNE - TOOTHING

for hardly machinable non-precious alloys,
measurable larger removal performance, smooth, plain surface, unequalled endurance

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

PRECISION CUTTER – round, handpiece, left-hand twist, right cutting

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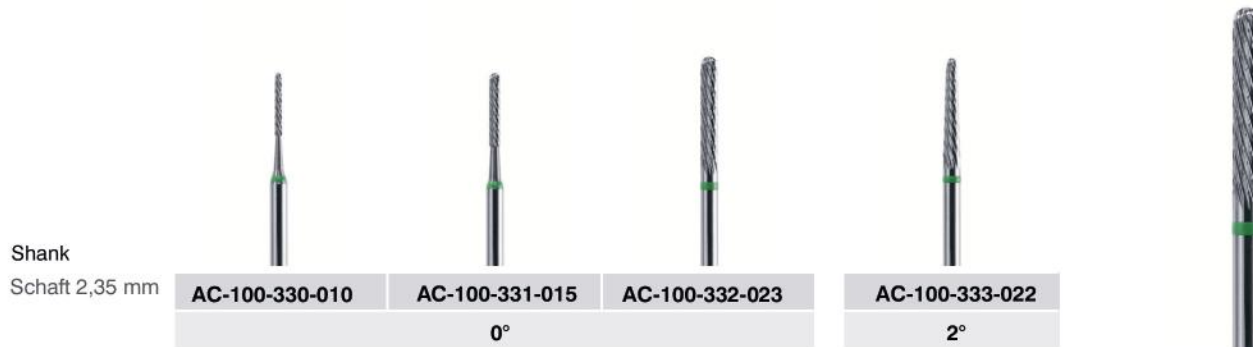MICRO-MILLING – round

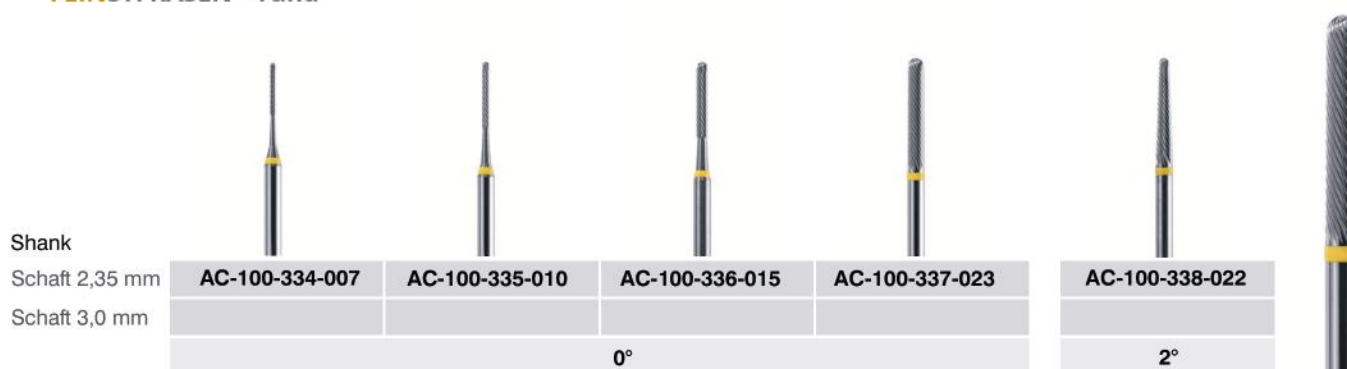FEINSTFRÄSEN – rund

ROUGHING CUTTER – high cutting performance, e.g. abutment

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PARALLEL - and TAPER -TC-Cutter

for titanium and non-precious alloys. Pre-cutter, left-hand twist, right cutting

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Schaft 2,35 mm

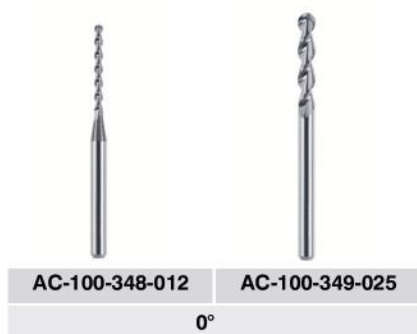
WAXCUTTER WACHSFRÄSER

**Reducing bush,
for the use of wax cutters
in the 3,0 mm collet chuck**

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Schaft 3,0 mm

AP VANILLA WAX BLANK

This original AP vanilla-scented wax blank is perfect for milling, containing no resin components. Its unique properties allow easy adjustment of milled crowns, making carving or adding material as simple as working with full contour wax.

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35 g

AP CARVING WAX

the perfect choice for precise wax-ups thanks to its outstanding sculpting and scraping properties. This wax is plastic-free and ash-free. Its high stability and low melting point allow for precise waxing. It also has a pleasant vanilla or lemon scent.

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ZIRCON OXIDE ABRASIVES with a special fine diamond bonding especially tuned for sintered zircon oxide ceramics, to avoid micro cracks and overheating.
FG Shank, to be used only with spray cooling.

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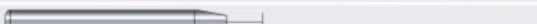
ISO Ø = 1/100 mm · ISO 040 = 4,0 mm

The total length of the instruments may vary according to length of working part and neck length

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- Good removal rate
- Extra stable, insensitive to temperature high performance diamond grit
- High grinding performance, significantly longer service life due to the special diamond grit
- Use only with sufficient water cooling, do not use them "dry"

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2K **Fine/Medium** Fein/Mittel

COMBINED GRINDER (FINE/MEDIUM)

for all high performance ceramics like ZrO+lithium disilicate

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- Very good, constant grinding performance
- Long service lifes
- Use only with sufficient water cooling, do not use them "dry"

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160.000 min⁻¹

Spray: 50 ml/min.

with an innovative diamond coating
for lithium disilicate and all CAD/CAM ceramics
like zirconium oxide

- High-performance diamond grain with proven Black- binding matrix
- Unsurpassed service life
- Excellent removal and cutting performance
- Pleasantly quiet grinding characteristics

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DIAMOND separating strips

are made of rust-free stainless steel and are smooth, coated with diamond grit. They have a far longer life due to their diamond coating and can be autoclaved and sterilized. Set with 10 pieces

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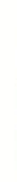
AP DIAMOND BURS

ROUND DIAMOND

for concave areas and slight adjustments of inlay / onlays and crowns.

The bur comes in a package of 5.

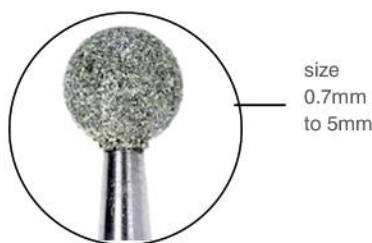
Sizes 2,9 mm and higher come as single burs.

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size
0.7mm
to 5mm

GALVANIC DIAMOND - Rounded Cylinder

Precise chamfer or step preparation enables an optimal fit of the body's own material and foreign material to enable high durability of the crown.

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size
1,2mm
to 4mm

AP DIAMOND GRINDER

for a precise and creative finishing with outstanding grinding performance. Especially for forming anatomy, shaping, and contouring.

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HO-200-021-027

HO-200-022-040

Diamond Grinders L, S and Needle are part of the AP Universal Bur Kit (page 63)

Relining, tray material, 3D printing material

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3D FINISHING BUR KIT

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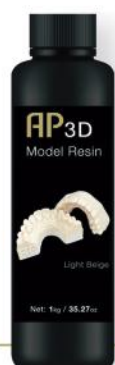
Effortlessly polish, contour, and finish 3D dental prints (resin and filament) with this expertly crafted kit. Designed for precision and efficiency, it's the ultimate solution for perfecting printed restorations.

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Light Beige

AP DENTAL

offers a variety of 3D Printing materials

544 Super coarse Supergrob

508 534 Coarse Grob

524 524 526 Medium Mittel

514 516 518 Fine Fein

504 506 Extra fine Extrafein

484 Ultra fine Ultrafein

Diaflex original Diamond Disc double-sided, 0.3 mm.

This diamond disc is multi-coated and ideal for all saw cutting

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Diaflex Diamond Disc

ideal for Plaster Sectioning for Model Prep, double-sided, 0.3 mm. This diamond disc is multi-coated and ideal for all saw cutting.

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Superdiaflex®

is a multi-coated diamond 0,17 mm thin disc and ideal for all saw cutting while working in the laboratory - whether separating, contouring or even grinding ceramics.

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Diaflex Transvident

is double-sided. With the Diaflex® disc, Horico invented the world's first flexible and rust-proof diamond disc with a thickness of 0.3 mm.

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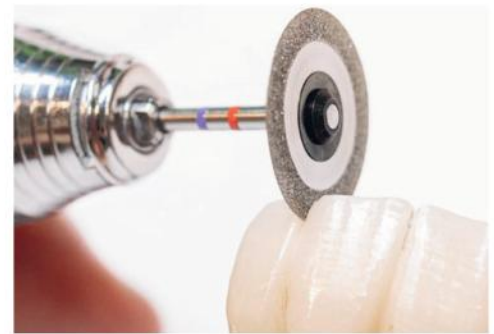
Also appropriate to separate press ceramic of the pressing cone.
Work with low contact pressure optimum speed:
cs. 10.000 – 12.000 min⁻¹

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526 AC-300-012-190
19,0 / 0,25

FG shank for laboratory turbine
mini disc, flexible, double side coated
rim, use only with water cooling

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Net disc, run transparent, double-sided coating for ceramics and composite, fine cut, for a better visibility during separation

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Spoke Disc, double side coated for ceramics and composite

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DISC FOR PLASTER

GIPSSCHEIBEN

Disc for plaster with saws, double side coated
Gips-Scheibe, gezahnt, voll belegt

Disc for plaster perforated with saws, double side coated rim
Gips-Scheibe, perforiert gezahnt, randbelegt

514			
	AC-300-019-300	AC-300-020-300	AC-300-021-450
Ø mm / L	30,0 / 0,25	30,0 / 0,38	45,0 / 0,38

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FLEXI FILM DIAMOND DISCS FLEXIFILM DIAMANTSCHLEIBEN

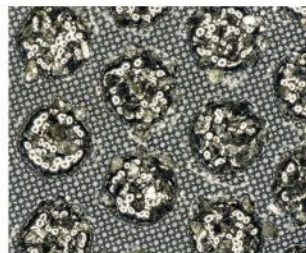
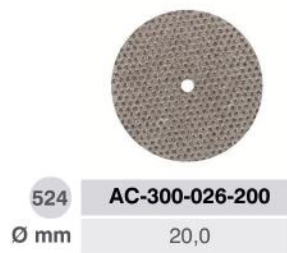
- Tear proof plastic film as carrier material
- Flexible and robust at the same time
- Galvanically bonded diamond grain for a long service life
- Depending on the contact pressure a quick removal or a fine surface treatment are possible
- Especially for ceramic processing without cooling option
- Defined and even surface pattern blocks the clogging of the diamond coating

Tip: For the treatment and smoothing of the emergence profile of individual ZrO abutments

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524 524 526 Medium Mittel

484 Ultra fine Ultrafein

Sintered Diamonds and Sintered Discs, shank Ø 2,35 mm

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BLACK ONYX

Precision meets performance
especially for zirconium oxide and lithium disilicate,
immaculate surfaces, super-soft grinding
performance, highest removal

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Ø mm / L

SMART GRINDER AC

Medium grit, highly suitable for zirconium oxide, lithium
disilicate, veneer ceramics, very smooth removal

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Ø mm / L

SMART GRINDER MF

Medium grit, highly suitable for zirconium oxide, lithium disilicate,
veneer ceramics, medium removal

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Ø mm / L

SMART GRINDER MC

Coarse grit, highly suitable for zirconium oxide, lithium disilicate, suitable for veneer ceramics and NEM/Ti, medium to high removal

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SMART GRINDER SC

Very coarse grit, highly suitable for zirconium oxide, NEM/Ti, suitable for lithium disilicate, limited suitability for veneer ceramics, high removal

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SMART GRINDER MAX

Very coarse, removal optimised grit, highly suitable for zirconium oxide, NEM/Ti, limited suitability for lithium disilicate and veneer ceramics, very high removal

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CERAMIC POLISHING highshine KERAMIKPOLITUR HOCHGLANZ

Interspersed with diamonds in high concentration. Without plastic center

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Ø mm / L

AC-600-001-260	AC-600-002-260	AC-600-003-260	AC-600-004-260
26,0 / 2,0	26,0 / 2,0	26,0 / 2,0	26,0 / 2,0
coarse grob	medium mittel	fine fein	extrafine extrafein

Ø mm / L

AC-600-005-220	AC-600-006-220
22,0 / 1,0	22,0 / 1,0
fine fein	extrafine extrafein

AC-600-007-045	AC-600-008-045	AC-600-009-045
4,5 / 13,0	4,5 / 13,0	4,5 / 13,0
medium mittel	fine fein	extrafine extrafein

Ø mm / L

AC-600-010-110	AC-600-011-110	AC-600-012-110	AC-600-013-170	AC-600-014-170	AC-600-015-170
11,0 / 2,0	11,0 / 2,0	11,0 / 2,0	17,0 / 3,0	17,0 / 3,0	17,0 / 3,0
medium mittel	fine fein	extrafine extrafein	medium mittel	fine fein	extrafine extrafein

Step 1 Schritt 1

Step 2 Schritt 2

Step 3 Schritt 3

Segmented disque-shaped ceramic polisher

with 100 % diamond grain

- For ZrO, lithium silicate and veneering ceramic
- Flexibly adapts to almost any surface structure
- Instructions for use:
maximum speed 10.000 min⁻¹, optimum speed 8.000 – 10.000 min⁻¹

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DIAMOND POLISHERS / Abrasives

DIAMANTPOLIERER

Two-step polishing system
for polishing of lithium
disilicate ceramic, silicate-
ceramic, leucite-ceramic.

Recommendations:

Max. speed

10.000 min⁻¹, work with a low
contact pressure

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Cutting Wheel Diapol

is a high performance cutting wheel showing an outstanding performance cutting and smoothening any zirconia or lithium disilicate surface, and ideal for finishing margins

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speed 15,000

Diapol H 9906

Diamond coded silicon wheel, to smoothen and pre polish all porcelain surfaces. This disc is ideal to polish smaller areas such as contact points. Technicians can condition the labial surface of anteriors with this wheel- for perfect natural appearances.

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Z-Shape No 1

grinding wheel, Course +

is a multipurpose diamond grinding wheels for exceptional grinding performance. Ideal for finishing margins on zirconia frameworks as well as shaping or reducing porcelains.

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*This Wheel is part of
the AP Porcelain Bur Kit*

9113

speed 15,000

Z-Shape No 3

rubber wheel, matt finish

The ideal rubber wheel to smoothen and surface seal any zirconia and lithium disilicate crowns. The perfect step before using a diamond polishing wheel.

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Z-Shape No 2

grinding wheel, course

is a high performance cutting and grinding wheel - a bit softer than No 1 wheel, yet this wheel shows an outstanding performance cutting and smoothening any zirconia or lithium disilicate surface, ideal for finishing margins.

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Z-Shape No 4

High Shine

The final step to polish any zirconia or lithium disilicate surface. The technician can fine-tune the luster and surface shine of their crowns or bridges. Facial, buccal/lingual surfaces or marginal ridges will get the last final finish to complete your restoration for a beautiful, homogenous result.

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RESH **AP** E

Oscar Diamond Polishing Paste Oscar Diamant Polier Paste

The "Oscar" diamond polishing paste is the ideal solution for every case. Oscar impresses with its outstanding polishing performance, which gives all surfaces a homogeneous shine. Due to the high diamond content, Oscar polishing paste is suitable for plastic, metal, ceramic and zirconium surfaces. Even on rough surfaces, "Oscar" produces excellent results surprisingly quickly. Highly praised by many of our customers

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AP GOLDFINGER

**20g Diamond Polishing Paste Gold
for Ceramic, Composite, Metal and Zirconia**

AESTHETIC-PRESS
www.apdental.net

REF: 007

GOLDFINGER 007

polish and finish for all ceramics,
NEM/titanium alloys and composites

- Extremely economical to use due to good adhesion
- High surface quality with the utilization of goat hair brushes

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Corund light brown – ceramic bond, soft grinding characteristics, for finish grinding of precious metals, non-precious metals and modelling compound

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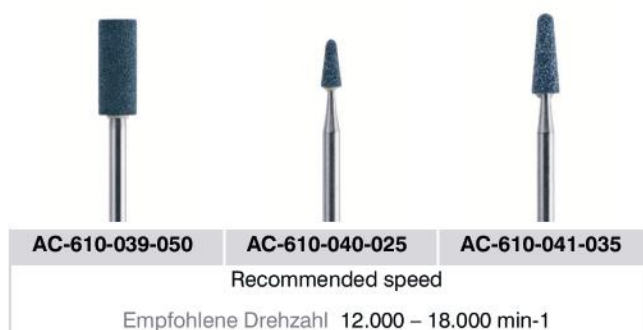
Corund blue – ceramic bond, soft grinding characteristics, for finish grinding of precious metals and reduced metal alloys. Special abrasive for frameworks

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EM Silicon carbide, green

ceramic bond, grit size medium for ceramics and precious metal alloys

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Recommended speed

Empfohlene Drehzahl 15.000 – 20.000 min⁻¹

Arkansas, white, for fine grinding of composites, precious alloys

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POLISHERS unmounted

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universal polisher, medium, white, for all metals, composite and plastics

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Pre-polisher, fine grain, hard bond, brown,
for precious metals

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22,0 / 3,0

6,0 / 22,0

Universal polisher, fine, blue,
for precious metals

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22,0 / 3,0

7,0 / 20,0

Universal polisher, fine bond, mint,
for non-precious alloys

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22,0 / 3,0

7,0 / 22,0

Chrome Master, medium for CrCo alloys

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Mandrel for occlusal surface
Okklusal-Polierminen Mandrell

POLISHER PINS · MANDRELS SCHLEIFRÄDER · MANDRELLE

Unisharp disc, extremely sharp SiC grain, universally applicable on all materials.

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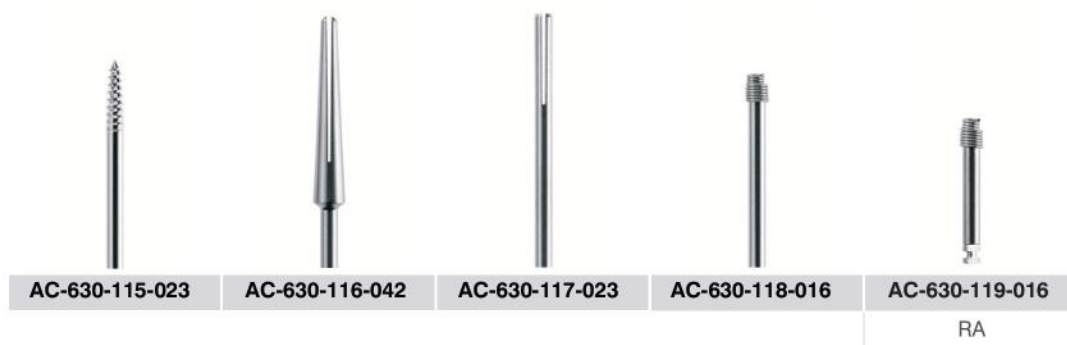
Felt polisher, Polishing tool made of compressed felt

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25,0 / 4,0

Ø mm / L

Ideal for rounding off and smoothing the edge
areas of thermoforming splints

- Soft elastomer fiber polisher for splint
polishing
- With low heat generation and long service
life thanks to impregnated special fiber

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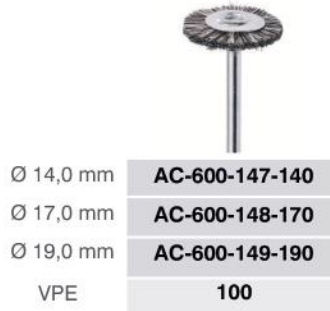
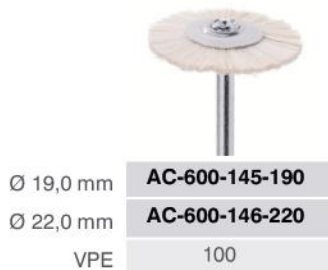
Cotton, High shine for all materials

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Ø 25,0 mm
VPE
AC-600-159-190
10

Linen, high shine polishing for all materials

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Ø 22,0 mm
VPE
AC-600-160-220
10

Leather high shine for all materials

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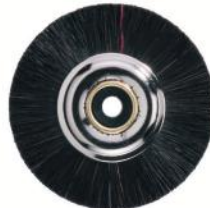
Polishing brush, 2-row, super hard,
bristles black, with plastic core

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Polishing brush, extra hard
2-row, with plastic core

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Unparalleled precision with Panther Burs,
masterfully crafted in Germany. Engineered for cutting-edge dental
materials like zirconia, lithium disilicate, and PMMA,
these burs redefine excellence

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Panther Polish Kit

for exquisite polishing!

This Panther kit with three geometries each of the Smooth and Gloss
instruments from Panther is suitable for polishing zirconium oxide, ceramic
and lithium disilicate

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PAKIT
4 bur set

Panther Abutment Surface Kit

With this slim kit you can achieve an optimized surface for better
tissue attachment in modern implant prosthetics.

The first stage involves processing with Panther Abutment Surface
lense 260 rough or lense 150 rough.

The second and final stage involves processing particularly the submucosal
areas with Panther Edition lense 260 smooth or lense 150 smooth

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PGKIT
4 bur set

Panther green bur and polishing kit ideal for zirconium
oxide, resulting in perfect accentuation of unsintered
zirconium oxide. Controlled removal and precise contouring!
Optimum roughness for best zirconium oxide brilliance

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Panther Stone Kit

is perfectly tuned to panther smooth, soft grinding, fast cutting diamond, high durability, long service life, smooth surface finish. Panther Instruments are the perfect choice when minimal adjustments are required.

The Panther stone kit includes 6 pieces

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6 bur set

Panther Edition Kit

is a premium line for all-ceramic restorations, natural gloss polish in just two steps, high durability long service life, well tested selection of shapes, non contaminated binder for smooth grinding
The Panther Polishing Kit is perfect for a flawless gloss for zirconia restorations

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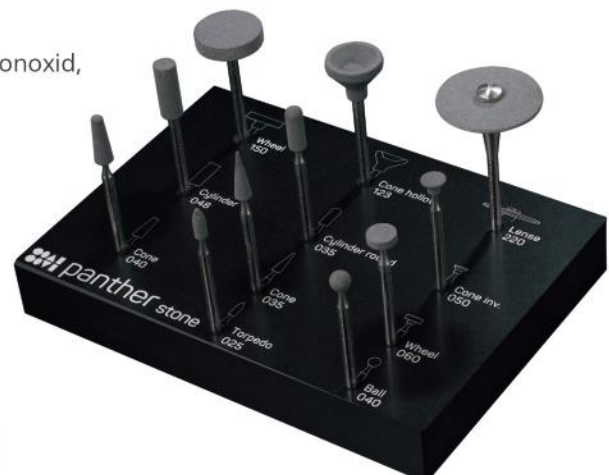
11 bur set

Panther Stone 11 Kit

is an 11-piece set for finishing sintered zirconium oxide, lithium disilicate and veneering ceramics

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Panther Edition XL Kit

the "crown" of the Panther line.

You now have the perfect solution for all areas of work: from optimal green part processing to the premium finish for the highest demands. Panther edition XL kit includes 31 pieces.

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31 bur set

Panther Stone

soft grinding, fast cutting diamond for every indication. Whether monolithic or partially veneered zirconia, fine-structure glass ceramic, or conventional veneering ceramic

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AP Grinding and Polishing Kit

AP Grinding and Polishing Kit for clinics and dental labs offers everything you need for restorations in a compact case to keep your workflow organized.

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9 bur set

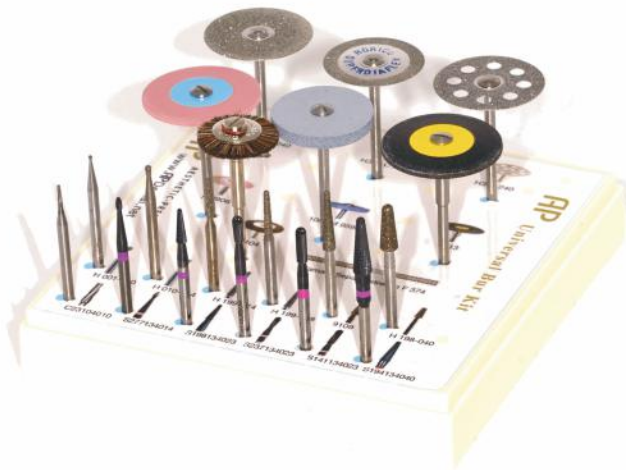
AP Universal Bur Kit

AP Universal Bur Kit is a selection of high quality dental burs for clinics and dental labs. Everything you need for restorations in a compact case to keep your workflow organised.

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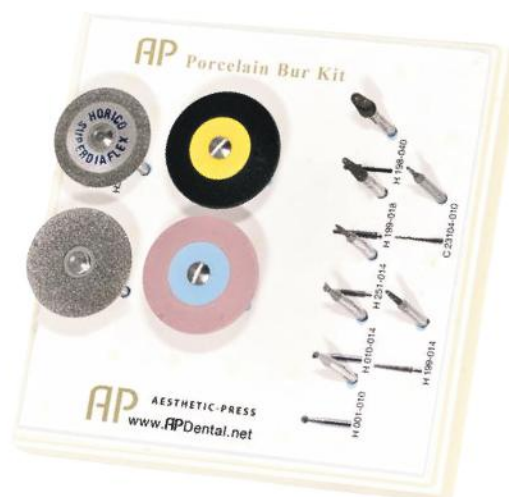
11 bur set

AP Porcelain Bur Kit

AP Porcelain Bur Kit - a premium selection of dental burs for clinics and dental labs. Everything you need for restorations in a compact case to keep your workflow organised.

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OSCAR Diamond Polish

"So good it deserves an Oscar!"
- Jürgen Auffarth, Salzburg

"We have been using Oscar Polishing Paste for many years, after trying many different paste we found this one to be the best bar none!"

Barry Johal, President, Dental Concepts, San Francisco

AP[®]volution M

Aesthetic Press Dental is a leading PFM manufacturer and supplier for over 2 decades

For nearly two decades, Aesthetic Press Dental has been at the forefront of dental innovation, establishing itself as a trusted leader in porcelain-fused-to-metal (PFM) restorations and advanced press-to-metal techniques. While many providers have abandoned PFM, we remain steadfast, perfecting this craft with the same passion as we embrace cutting-edge 3D innovations. Our advanced techniques and premium materials deliver durable, lifelike restorations with unmatched precision.

Ready to level up your skills?

Our dynamic in-house seminars invite dental professionals to dive into hands-on training with our expert master dental technician Jörg Müller.

Learn the art and science of creating stunning, long-lasting restorations while mastering the art of advanced press-to-metal.

MDT Dwight Vandaele,
Labo VANDERBEKEN,
Belgium

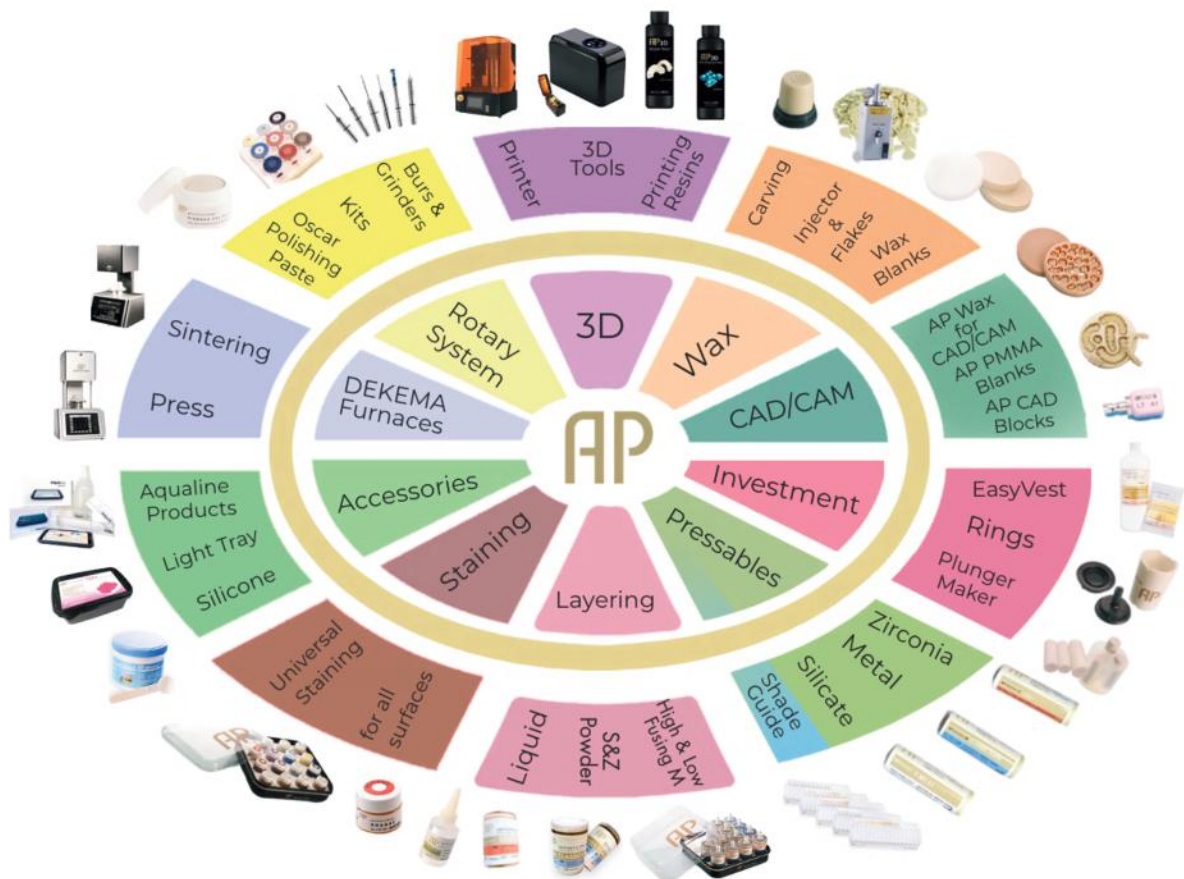
Contact us for Aesthetic Press Dental in-house seminars.

www.apdental.net

www.apdental.net

USA

Aesthetic-Press, LLC
7620 Massachusetts
Avenue, New Port Richey,
FL 34653
Tel: +1 (727) 493 4062
Fax: +1 (415) 723-7075
www.apdental.net

Email:

sales@apdental.net

Germany

Aesthetic-Press GmbH
Paulinzella 8 A
07426 Königsee
Tel: +49(0)211-540 14 772
Fax: +49(0)211-204-9131
www.apdental.shop