


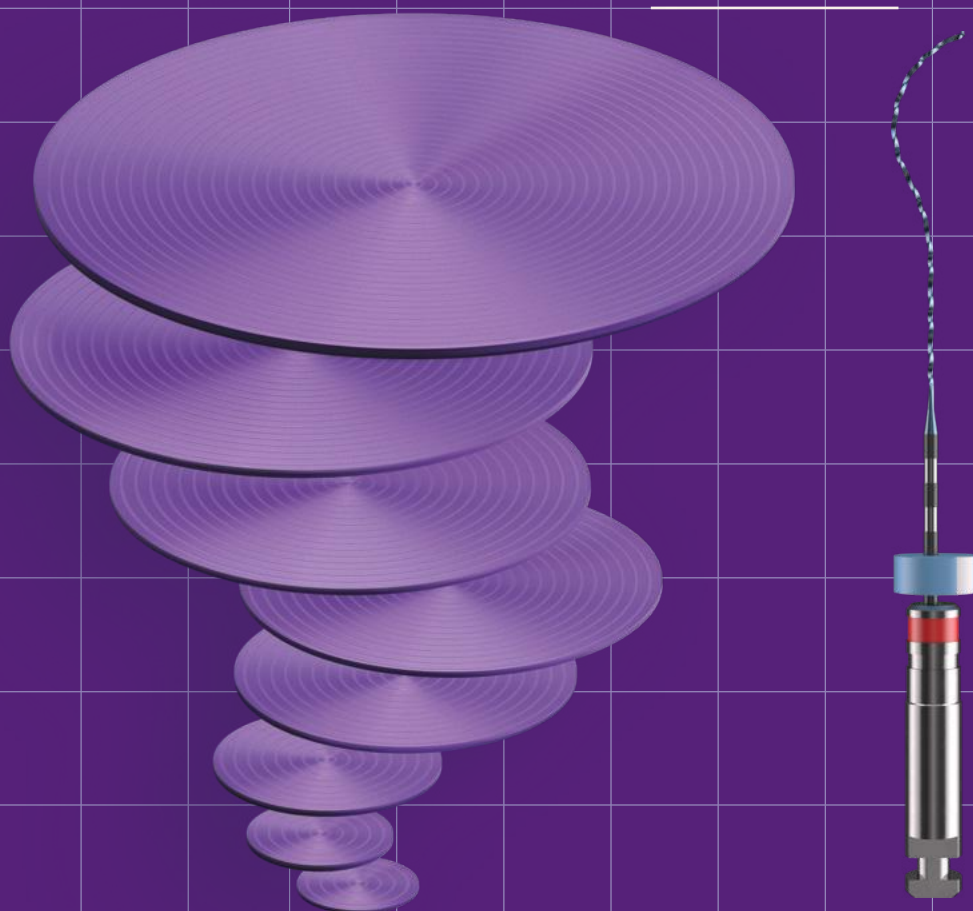

Cleaning

Power Tornado

Radical: 98% bacterial
removal. Ready to be a
cleaning fanatic?

Finisher

Innovation in the Heart of the Swiss microtech valley

Since 1931, in the heart of the Swiss microtech valley, we have been pushing the limits of endodontics by combining technology and innovation.

Our mission is to provide solutions that respect the complex anatomy of teeth while improving the treatment efficiency. With game-changers such as XP-endo Rise, RACE EVO, R-Motion, Finisher and TotalFill, we have transformed how endodontics is practiced, giving clinicians better tools to meet challenges with confidence.


Didier
Devaud
General
Manager




Cleaning

Power Tornado

Get a powerful clean like never before. This unique mechanical cleaning device effectively reduces bacteria up to 98%*. Impressive, isn't it? Finisher is able to remove filling material too. A universal solution so revolutionary that you'll become a cleaning fanatic. That's how we empower endo.

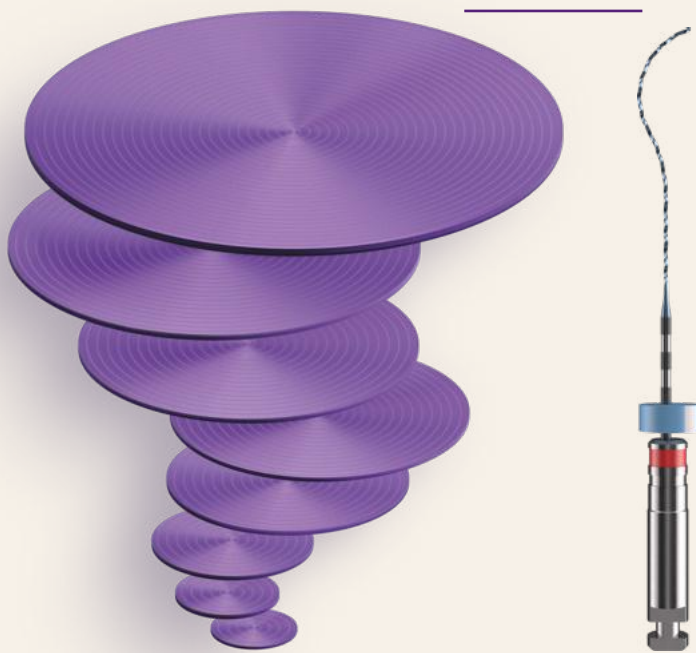


We provide a universal cleaning system that ensures a high level of disinfection for outstanding treatment results. It gently abrades canal walls, removes stubborn and/or hard-to-reach debris and effectively removes filling materials.

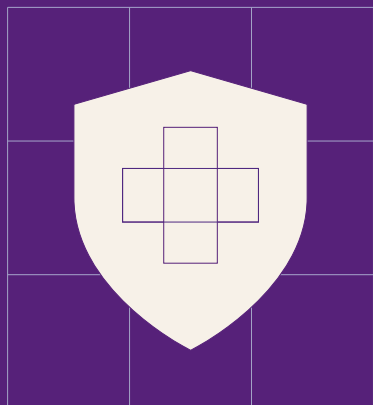


High level disinfection

Made to enhance disinfection, The Finisher removes up to 98% of bacteria and reduces the smear layer*. This leads to a high level of clinical outcomes and suitable preparation for receiving obturation material.

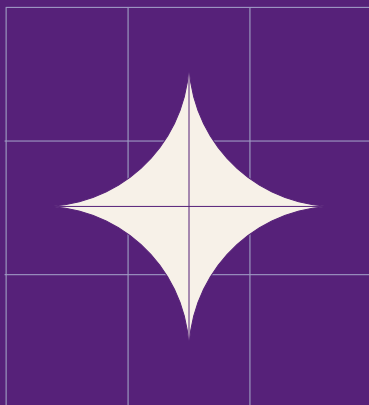

Universal cleaning system

No matter which shaping system you use, Finisher's universal cleaning system significantly enhances canal cleaning.

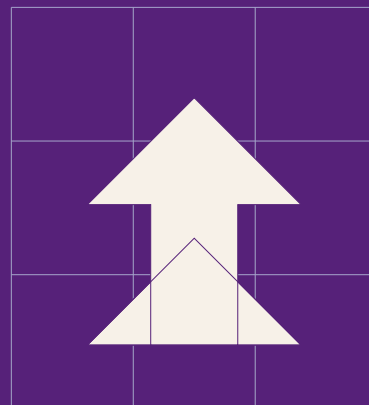

Removal filling material

Simply improve cleaning with the Finisher R, the advanced tool that efficiently removes filling material during retreatment procedures.



High level disinfection*


Finisher provides an advanced level of disinfection, as proven by numerous studies. This level of disinfection can achieve up to 98% of bacteria removal and reduces the smear layer, resulting in a cleaner canal system.

This reduction in smear layer and debris allows irrigant and sealer to penetrate deeper into the dentinal tubules, favoring the seal of the root canal filling.

* Azim AA, Aksel H, Zhuang T, Mashtare T, Babu JP, Huang GT. Efficacy of 4 Irrigation Protocols in Killing Bacteria Colonized in Dentinal Tubules Examined by a Novel Confocal Laser Scanning Microscope Analysis. J Endod. 2016;42(6):928-934. doi:10.1016/j.joen.2016.03.009

Universal cleaning system

Finisher is a universal cleaning system that enhances root canal procedures by ensuring high cleanliness, regardless of the shaping system used. Its use enhances the disinfection level, making it your ideal partner. Studies show that using Finisher can significantly improve canal cleaning, leaving an extremely clean canal ready for obturation.

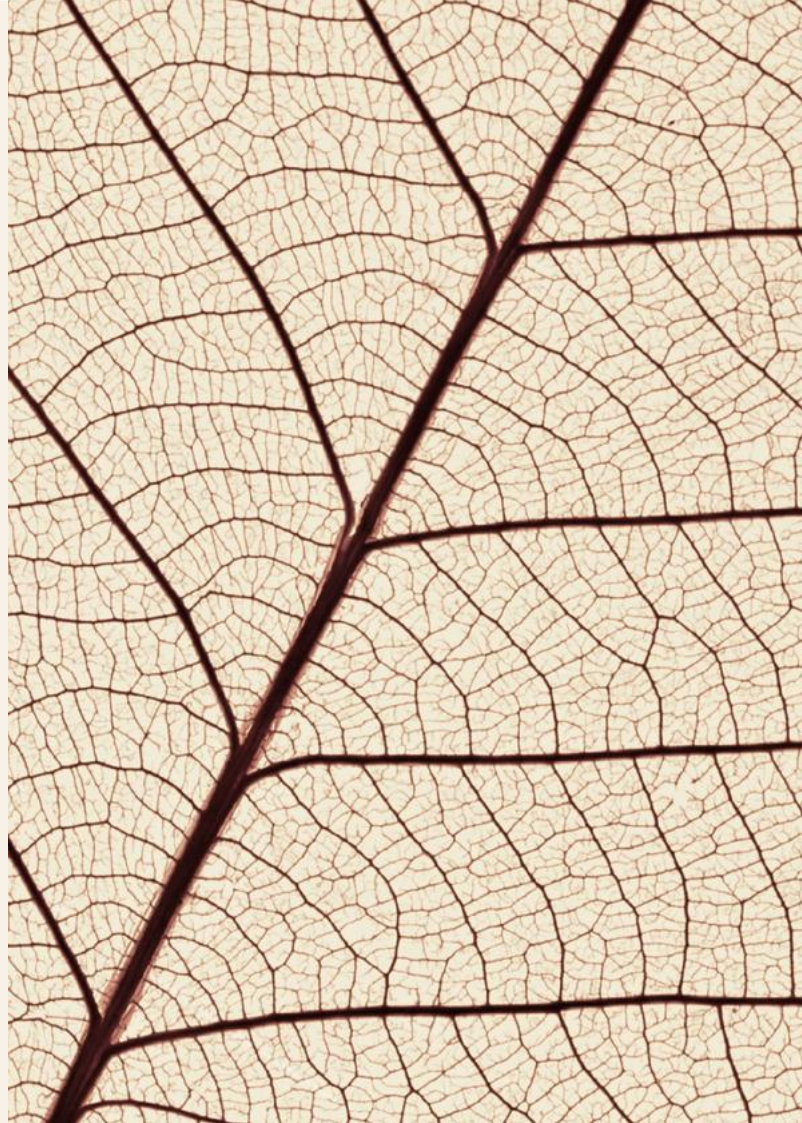

Removal filling material

The Finisher R is a powerful tool for removing filling material during retreatment procedures. With a stronger design compared to the original Finisher, the Finisher R ensures a cleaning process that adapts to materials requiring power to be removed, such as during retreatment procedures.

Expert's tip on inserting Finisher

"The curved tip of Finisher may create some difficulties to insert into the canal. My clinical advice to facilitate the insertion of this file into the canal, is to face the tip against the referal cuspid – it will slide into the canal by itself when activated by the motor."

To make this clinical advice understandable, you will find a short tutorial video by scanning the QR Code below.

"I've been using Finisher since its launch over 10 years ago. Based on scientific publication and my clinical experience, I'm convinced of its effectiveness for final cleaning in both primary and retreatment cases. It can be used after any NiTi file system as a final step before obturation. It's a real game changer in the world of endodontics."

Dr. Gilberto Debelian
DMD, PhD / Oslo
Norway




Curious? Here's more details about Finisher.

Irrigant activation

Gently abrade canal walls and dislodge debris in difficult-to-reach areas.

Finisher and Finisher R are compatible with all rotary or reciprocating endo systems.

Non-cutting and non-active tip.


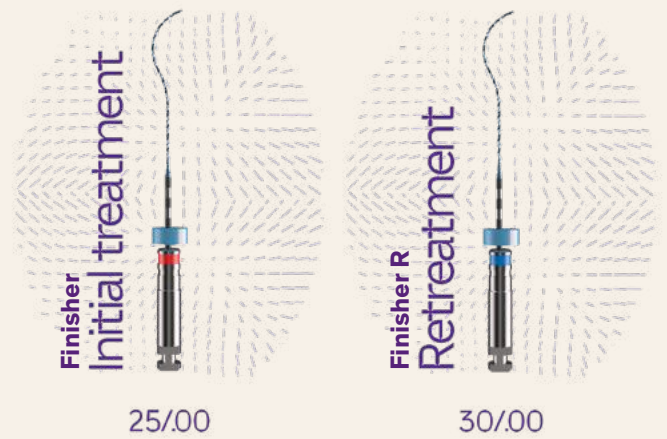


Finisher

Initial Treatment

Up to 98% bacterial reduction in the main canal*

Pre-op/Post-op NiTi : debris in the canal and isthmus areas. In Post-op F(inisher), after final cleaning, no debris are seen.

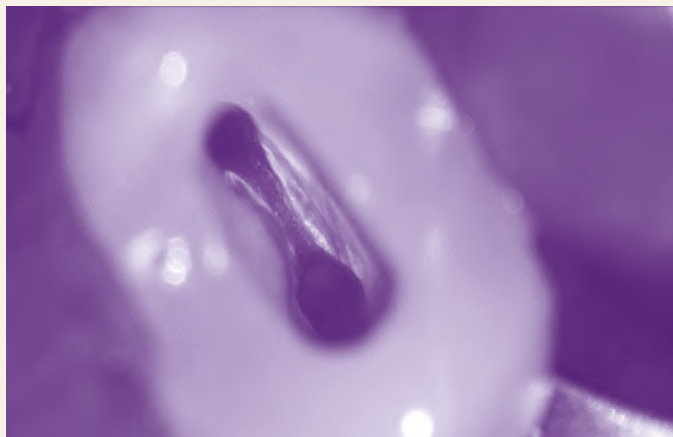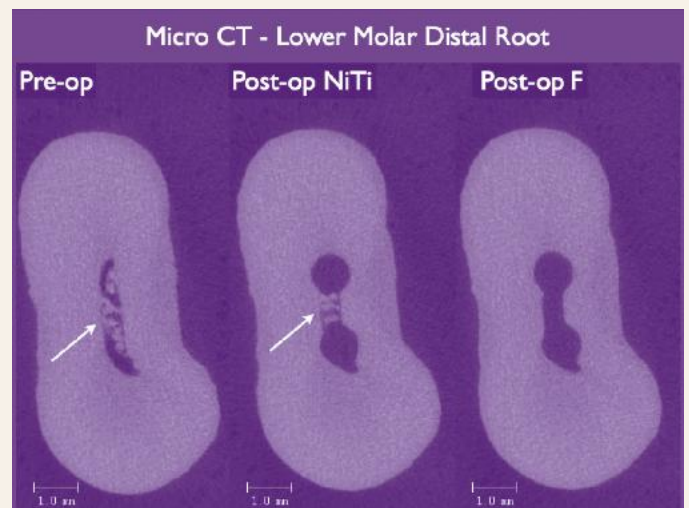

2x more debris removal with Finisher R vs. PUI **

After Instrumentation with standard NiTi files and final cleaning with Finisher R, no debris are seen.

* Azim AA, Aksel H, Zhuang T, Mashtare T, Babu JP, Huang GT. Efficacy of 4 Irrigation Protocols in Killing Bacteria Colonized in Dentinal Tubules Examined by a Novel Confocal Laser Scanning Microscope Analysis. J Endod. 2016;42(6):928-934. doi:10.1016/j.joen.2016.03.009

** De-Deus G, Belladonna FG, Zuolo AS, et al. XP-endo Finisher R instrument optimizes the removal of root filling remnants in oval-shaped canals. Int Endod J. 2019;52(6):899-907. doi:10.1111/iej.13077

References

Finisher


25/.00 3 pieces per Blister

	21mm	S1.XB0.00.0AX.FK
	25mm	S1.XB0.00.0AY.FK
	31mm	S1.XB0.00.0AZ.FK

Finisher R

30/.00 3 pieces per Blister

	21mm	S1.XB0.00.0BA.FK
	25mm	S1.XB0.00.0BB.FK
	31mm	S1.XB0.00.0BC.FK

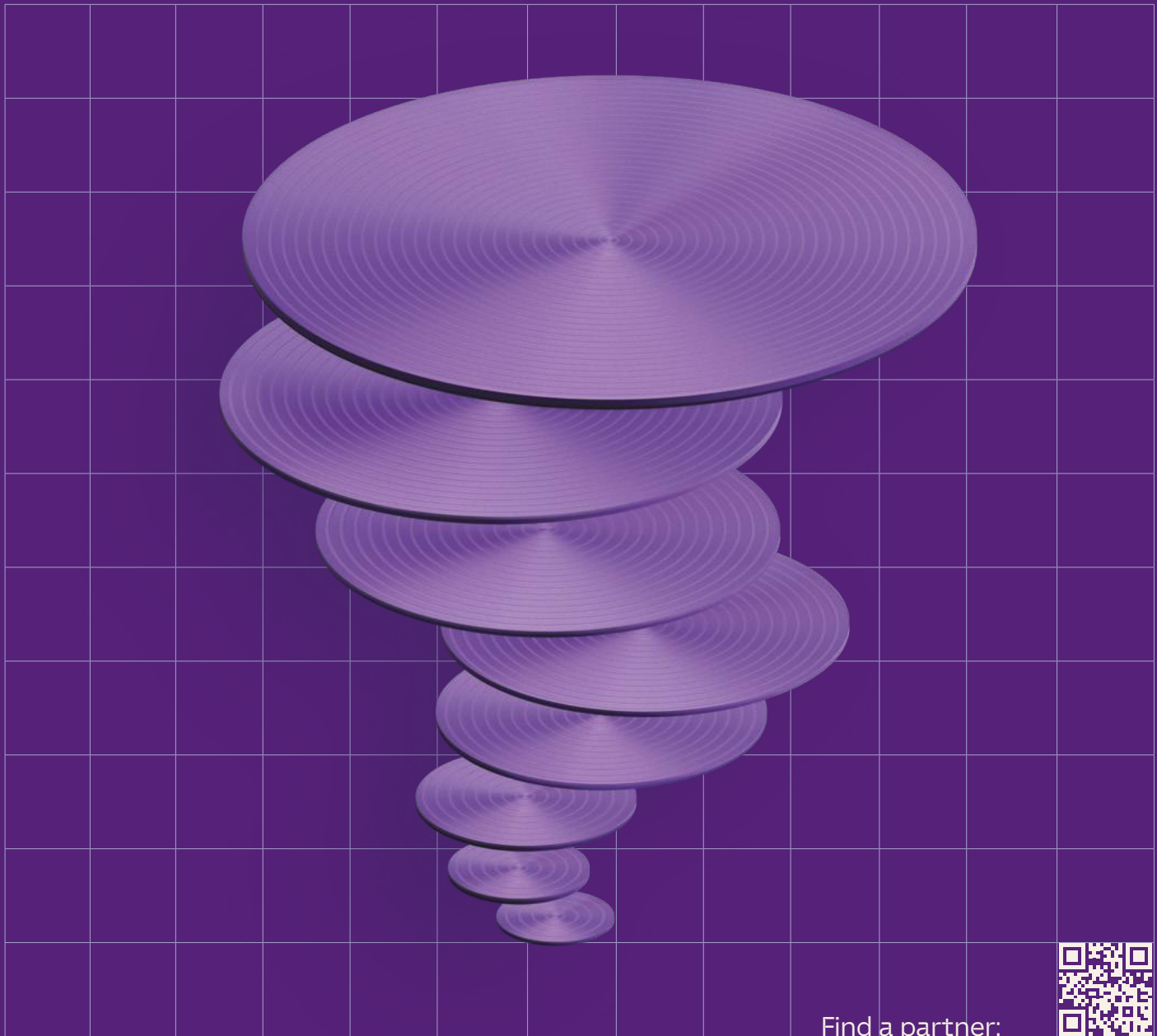

Find a partner:



Empower Endo



FKG REF 99.X000009AEN_REV1_2025-05-23 - © FKG Dentaire Sàrl - Pictures of our products are not contractual

Welcome to the heart of the Swiss microtech valley.

For nearly a century, we have been constantly re-inventing and challenging what we thought we knew.

We strive to create innovations for new ways of working: more respectful, more efficient, more agile, more ergonomic, and safer—serving new generations, promoting new visions of life, for us, for you, and for patients.

We embrace change that empowers endo.

FKG | Change the game.

Enter our game-
changing universe



Check out
our website



www.fkg.ch

FKG Dentaire Sàrl
Le Crêt-du-Loche 4
2322 Le Crêt-du-Loche
Switzerland

+41 32 924 22 44
info@fkg.ch